

- *Helps block out ambient light while holding all models of flexible probes on adult fingers or infant feet and toes.*
- *Replaces tape and protects delicate skin.*
- *Fits adults, children or infants.*



- *Hook and loop closure provides universal sizing.*
- *May be trimmed to fit.*
- *One dozen per package.*

## **PULSE OXIMETER PROBE WRAPS**

**Indication:** Patients requiring pulse oximeter monitoring.

**#6554**



**POSEY HOTLINE: 1-800-44-POSEY Fax: 1-800-767-3933**

## **PULSE OXIMETER PROBE WRAP**

**Indication:** Patients requiring pulse oximeter monitoring.

**#6554**

- *Secures all styles of flexible optical transducers and probe sensors without messy tape.*
- *Holds optical components firmly to help prevent misalignment; blocks excess ambient light for more exact readings.*
- *Universal design may be trimmed to fit neonatal foot, infant toe, pediatric or adult finger.*
- *One dozen per package.*

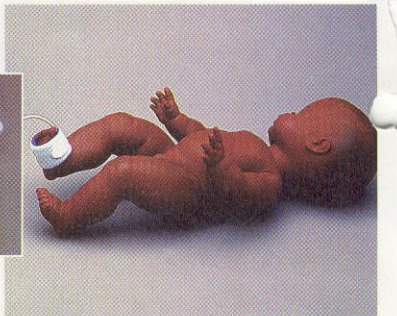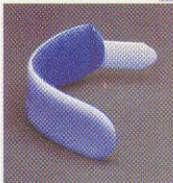





# Pulse Oximeter Probe Wrap

*The Posey Probe Wrap replaces adhesive tape to secure all models of flexible optical transducers or probe sensors. It holds it firmly to prevent the misalignment of optical components while blocking out excessive ambient light. The soft foam tapeless fastener does not stick to the skin, allowing for easy application or removal while preserving delicate skin integrity. The universal design will fit a neonatal foot, infant toe, pediatric or adult finger. Posey Probe Wraps may be cut to any size.*

**#6554** (1 Dz/box)

## **⚠ WARNING**

- Make sure your patient has stable peripheral vascular function before you place a pulse oximeter sensor.
- Secure the sensor loosely with the Posey Probe Wrap, never wrap it tightly.
- Check the site immediately after applying the sensor to ensure that circulation is adequate.
- Include circulation checks with all patient assessments.
- Change the pulse oximeter site every 2 hours or according to your institution's protocol.
- Closely monitor patients with fragile skin, such as infants or elderly adults.

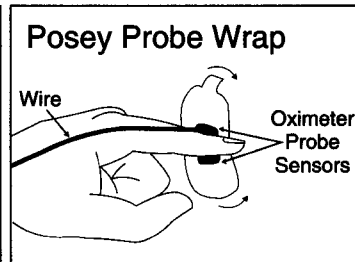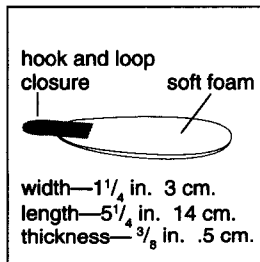


**J.T. Posey Company** 5635 Peck Road • Arcadia, CA 91006-0020 USA • Tel. 800-447-6739 or 626-443-3143 • Fax 800-767-3933 or 626-443-5014


EU AUTH. PERSON: (MDD 93/42/EEC) MDSS GmbH, Burckhardtstr. 1, D-30163, Hannover, Germany

© 1993 J.T. Posey Company. All rights reserved.

**CE**  
M1614 100998

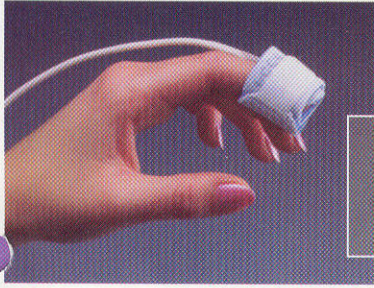

- Helps block out ambient light while holding all models of flexible probes on adult fingers or infant feet and toes.
- Replaces tape and protects delicate skin.
- Fits adults, children or infants.

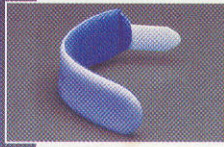

- Hook and loop closure provides universal sizing.
- May be trimmed to fit.
- One dozen per package.

## PULSE OXIMETER PROBE WRAPS

**Indication:** Patients requiring pulse oximeter monitoring.

#6554

**POSEY HOTLINE: 1-800-44-POSEY Fax: 1-800-767-3933**

## PULSE OXIMETER PROBE WRAP

**Indication:** Patients requiring pulse oximeter monitoring.

#6554

- Secures all styles of flexible optical transducers and probe sensors without messy tape.
- Holds optical components firmly to help prevent misalignment; blocks excess ambient light for more exact readings.
- Universal design may be trimmed to fit neonatal foot, infant toe, pediatric or adult finger.
- One dozen per package.

