Disposable Oximetry Sensor

Adult Models: **2311** through **2361**
 Pediatric Models: **2312** through **2362**
 Infant Models: **2313** through **2363**
 Neonatal Models: **2314** through **2364**

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Intended Use: Dolphin 2000™ oximetry sensors are indicated for use in continuous noninvasive monitoring of arterial oxygen saturation and pulse rate. They are fully compatible replacement sensors intended for use with major brands of pulse oximeters. ♦ The above tables list models of Dolphin 2000™ disposable sensors for the following four categories of patients: adult (weight greater than 30 kg), pediatric (10–50 kg weight), infant (3–20 kg weight), and neonatal (weight less than 3 kg). Select the sensor model number that matches the brand of oximeter you are using.

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Precautions: For further instructions, warnings and precautions, refer to the operating instructions provided with your oximeter. ♦ Do not autoclave or immerse the sensor in liquid. ♦ CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician. ♦ **Warnings:** Match the sensor model with your brand of oximeter. A mismatch may cause patient injury, inaccurate measurements or equipment damage. ♦ Apply the sensor firmly but not too tightly. A sensor applied too tightly may cause a blister. ♦ Do not apply the sensor for too long a time at one site. A sensor applied for too long at one site may cause a blister. ♦ Do not use the sensor if it is damaged. Use of a damaged sensor could cause patient injury or equipment failure. ♦ To prevent inaccurate measurements, avoid excessive ambient light, excessive patient motion, low perfusion or electrical interference at the sensor site.

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To Apply the Sensor



1. Connect the sensor cable to the Dolphin™ adaptor cable that fits your oximeter. Then turn on the oximeter and verify proper operation.

2. Select the sensor site on the patient. The preferred sensor sites are the index finger for adults and children, the great toe for infants, and on the foot below the toes for neonates.


3. Remove the plastic backing from the sensor adhesive wrap.

4. Apply the sensor on the patient:


- Position the emitter (☀) and detector (☿) directly opposite each other.
- Apply the sensor firmly but not too tightly.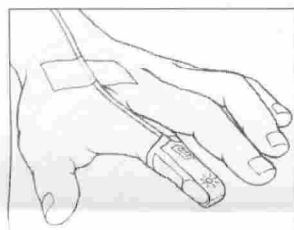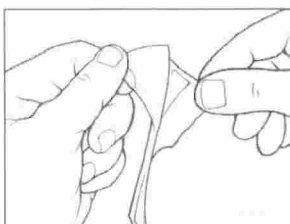


Adult Sensor

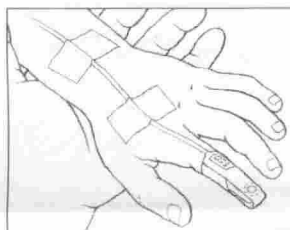Pediatric Sensor

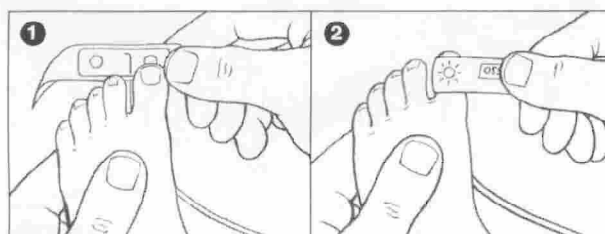Infant Sensor

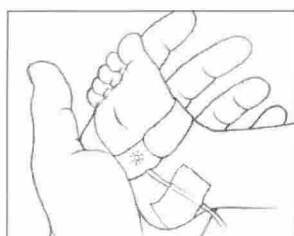Neonate Sensor

5. Visually monitor the sensor site to ensure the integrity of the skin.

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Environmental Requirements:

- Operating temperature range: 50–104° F (10–40° C)
- Storage temperature range: 32–122° F (0–50° C)

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©1999 by OSS Medical. OSS, Dolphin, Dolphin 2000 and the dolphin logo are trademarks of OSS Medical. Nellcor is a registered trademark of Nellcor Puritan Bennett, Incorporated. Ohmeda is a registered trademark of The BOC Group PLC. Datex is a registered trademark of Instrumentarium Corporation. Novamatrix is a registered trademark of Novamatrix Medical Systems, Inc.

OSS Medical
996 Bendemeer Road #04-02
Singapore 339944
Telephone: 65 296 7600