



WIRING DIAGRAM

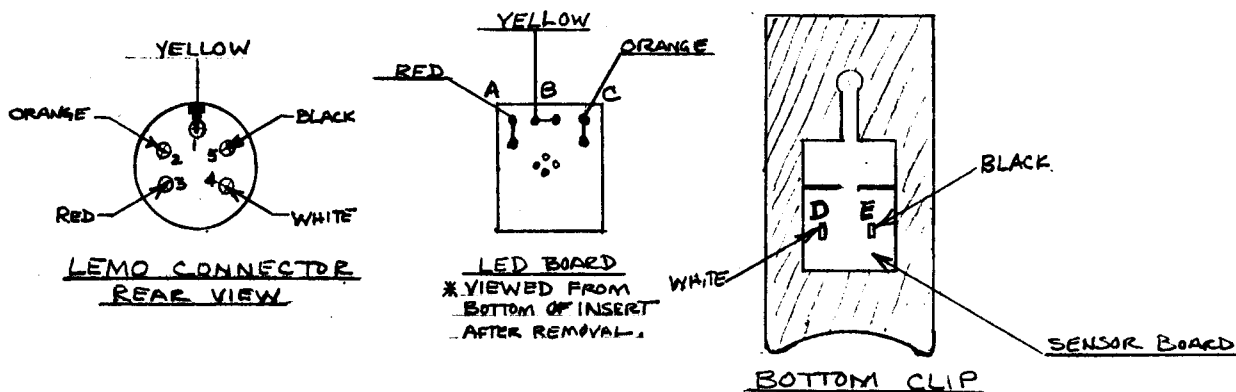

SCHEMATIC

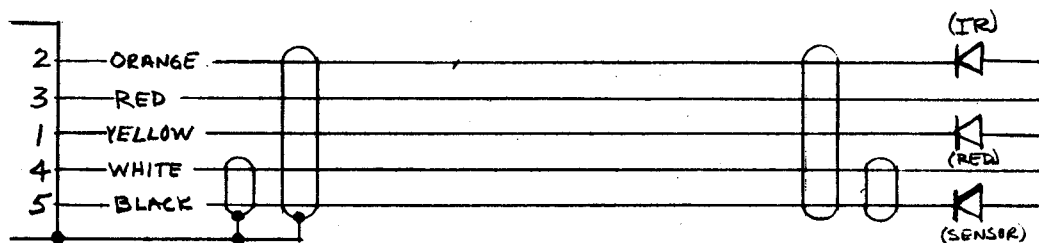

PARTS LIST

10' X2D055B0 CABLE
 X3C1B305 CONNECTOR, 5 PIN, LEMO
 X3C1B030 STRAIN RELIEF

SPECIAL INSTRUCTIONS

1. SUPER GLUE WIRES IN FINGER CLIP BEFORE ASSEMBLING
2. WHEN REPLACING CABLE USE THE FOLLOWING WIRING DIAGRAM.

RED - PIN 3 TO A
 ORANGE - 2 TO C
 YELLOW - 1 TO B
 WHITE - 4 TO D
 BLACK - 5 TO E
 SHIELDS - CASE TO NC



EPIC MEDICAL EQUIPMENT SERVICES, INC.

Dallas, Texas

SCALE: N/A

APPROVED BY:

Worley

DRAWN BY WORLEY

DATE: 11/17/95

REVISED 2/8/96

CRITICARE SpO₂ CABLE - 511-10L

REPAIR STANDARDS

DRAWING NUMBER
C-CARE 01