

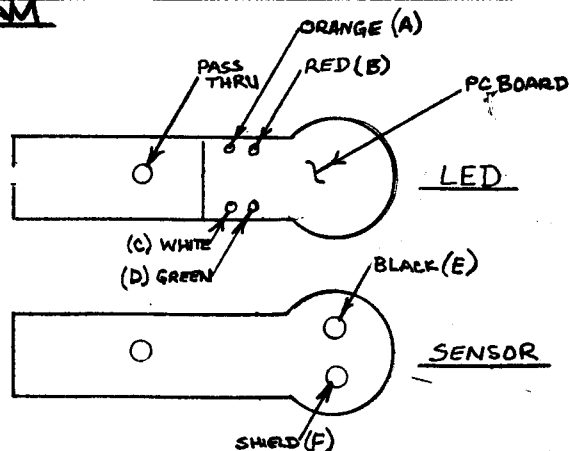


WIRING DIAGRAM

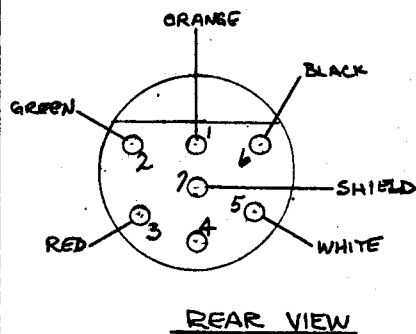

SCHEMATIC

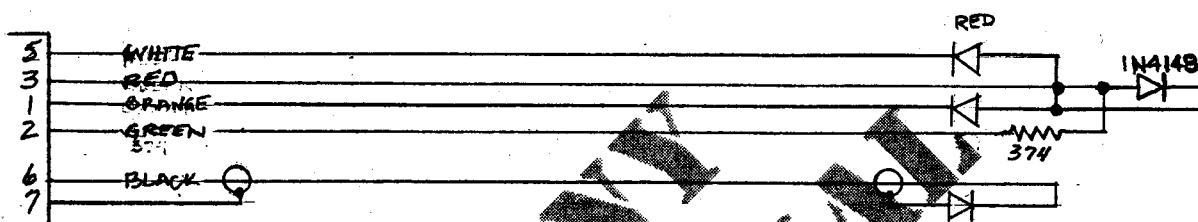

PARTS LIST

8' X2D05580 CABLE
X3C 00706 CONNECTOR IN HYP
X2FD1003 STRAIN REL

SPECIAL INSTRUCTIONS

WHEN REPLACING THE CABLE ASSY, USE EPIC PIN X2D05580 CABLE AND FOLLOW THE WIRING DIAGRAM BELOW:

RED	-	PIN 5	To	C
ORANGE	-	3		B
YELLOW	-	1		A
WHITE	-	2		D
BLACK	-	6		E
SHIELD	-	7		F



EPIC MEDICAL EQUIPMENT SERVICES, INC.

Dallas, Texas

SCALE:

N/A

APPROVED BY:

DRAWN BY

WORLEY

DATE:

1/22/96

REVISED



OHMEDA S₀O₂ CABLE - BIOX IVA EAR

REPAIR STANDARDS


DRAWING NUMBER

OHMEDA Ø2

WIRING DIAGRAM

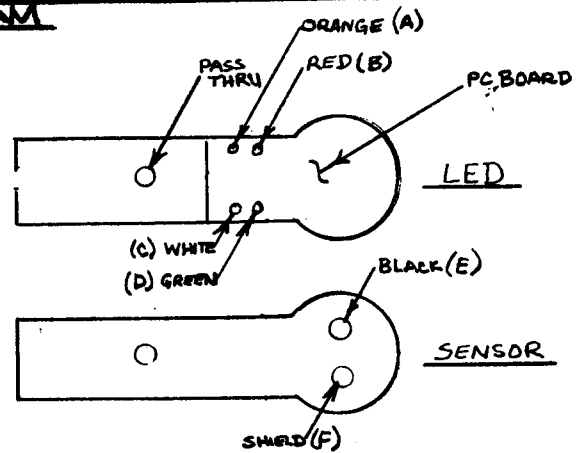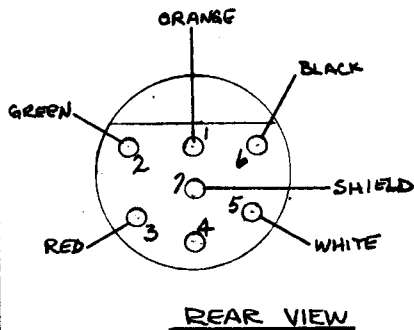

SCHEMATIC

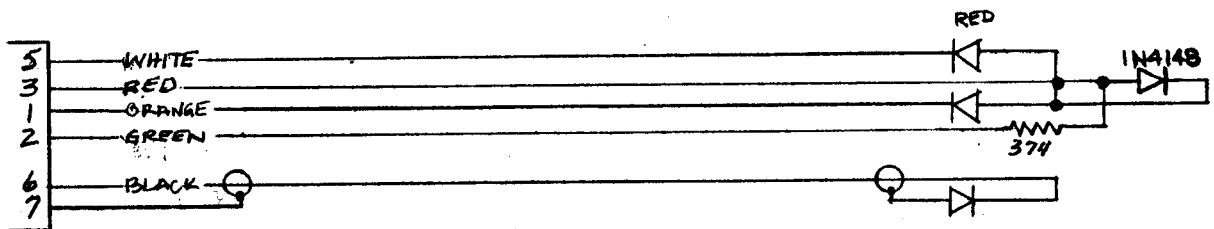

PARTS LIST

8' X2D05580 CABLE
X3C00706 CONNECTOR, 7 PIN HYP (INCL PIN)
X2FD1003 STRAIN RELIEF

SPECIAL INSTRUCTIONS

1. WHEN REPLACING THE CABLE ASSY, USE EPIC PIN X2D05580 CABLE AND FOLLOW THE WIRING DIAGRAM BELOW:

RED	-	PIN 3	To	B
ORANGE	-	1		A
YELLOW	-	2		C
WHITE	-	5		D
BLACK	-	6		E
SHIELD	-	7		F



EPIC MEDICAL EQUIPMENT SERVICES, INC.

Dallas, Texas

SCALE:

N/A

APPROVED BY:

Worley

DRAWN BY

WORLEY

DATE:

1/22/96

REVISED

2/8/96

OHMEDA S₆O₂ CABLE - BIOX IVA EAR

REPAIR STANDARDS

DRAWING NUMBER

OHMEDA Ø2