

QUALITY CONTROL PROCEDURE
REPAIRED DATEX SaO2 CABLES
Page 3 of 3

III. PERFORMANCE (record readings on WORKSHEET)

A. CABLE CONNECTOR

1. Connect the Nellcor Adapter Cable to the "A" connector of the Test Fixture and the Nellcor N200 Oximeter Patient Module.

B. SENSOR CLIP

1. Attach the sensor clip to the "RED" Nonin Saturation Test Unit.
2. Pulse the unit about once per second.
3. The Oximeter should read " % SaO2 (+/- 2).
4. Attach the Sensor Clip to the "BLACK" Nonin Saturation Test Unit.
5. Pulse the unit about once per second.
6. The Oximeter should read " % SaO2 (+/- 2).

IV. GENERAL

- A. Make sure all entries are recorded on worksheet.
- B. Indicate "Acceptance" or "Failure".
NOTE: If unit fails, return to repair technician.
- C. If accepted, record the date QC was performed.
- D. Send the unit, with the worksheet, to shipping for return to the customer.