

0110616 OXYGEN SENSOR

W/51002962

CE0086

**Contains caustic
substance exceeds
0.4% lead by weight**



VIAMED

15 Station Road, Cross Hills, Keighley
West Yorkshire BD20 7DT, United Kingdom
Tel: +44 (0) 1535 634542
Fax: +44 (0) 1535 635582
Email: info@viamed.co.uk
Web: www.viamed.co.uk



WARNING!

- 1) Sensor contains corrosive electrolyte.
- 2) Do not puncture or remove sealed membranes.
- 3) Use eye and hand protection if sensor leaks.
- 4) Flush and wipe electrolyte from all surfaces.
- 5) Electrolyte can be harmful or fatal if swallowed.

INSTALLATION INSTRUCTIONS:

- 1) If sensor is damaged during transit, return to Viamed.
- 2) Store at reduced temperature to extend sensor life.
- 3) Do not touch the sensor membrane or contacts.
- 4) If fitted, remove shorting device; i.e. red tab on PCB or spring on pins.
- 5) Install sensor immediately. Do not allow sensor to remain un-shortcd.