



HME Limited

ARLINGHAM HOUSE, 8Y ALBANS ROAD
SOUTH MIMMS, HERTS, EN6 3PH,
ENGLAND

Tel: +44 (0)1707 601 300 Fax: +44 (0)1707 646 250

Fax to: Viamed

Fax No: 01535 635582

FAO: Steve Nixon

Date: January 3, 2001

From: Bill Terry

PAGE 1 OF 2




Dear Steve,

Further to our phone call please find attached a drawing of the cable we need to our Part Number XC 0064.

Please note that if necessary we have some Redel plugs PAG-M07-GLAC39N if necessary.

Regards,

Bill

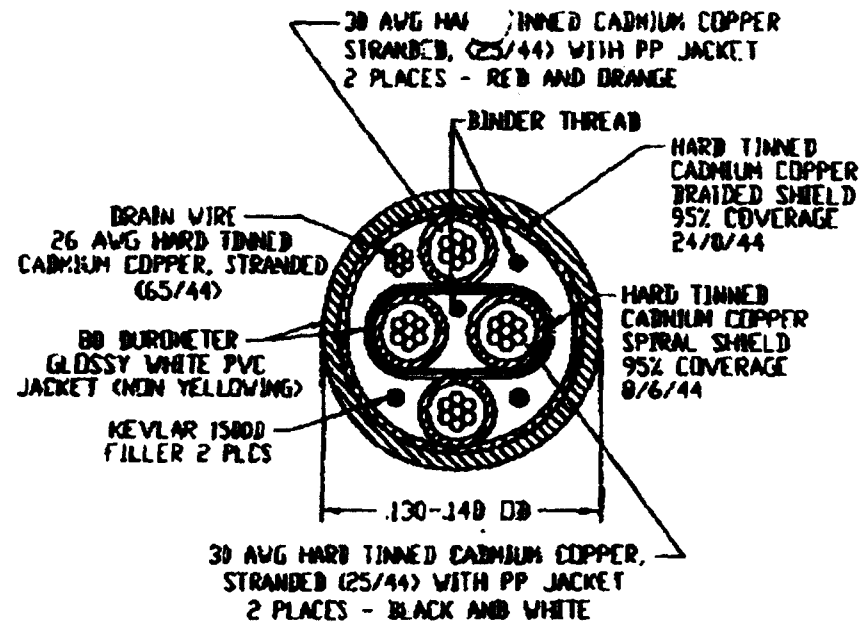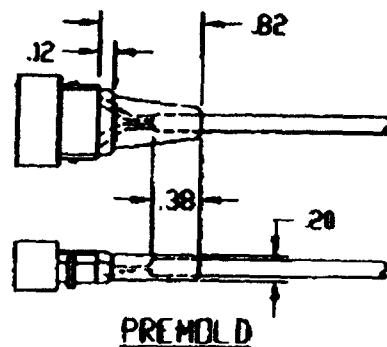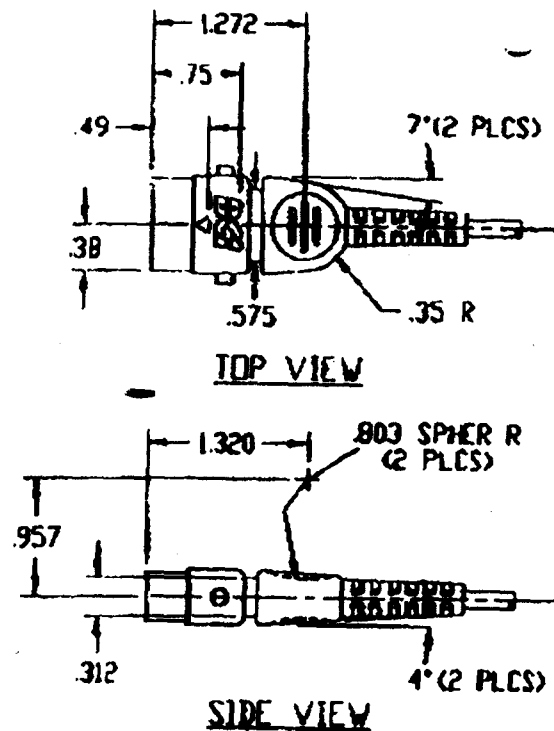

REV.	DESCRIPTION
0	ALL SHEETS 2-23-96
0	PRODUCTION RELEASE C/N 304 MER 5-12-97
1	ADDED BINDER THREADS. SEE SHEET 1. C/N 483 MER 6-8-97
2	SEE SHEETS 1 & 3. C/N 516 MER 7-18-97
3	SEE SHEET 1. C/N 416 MER 9-5-92
4	SEE SHEET 1. C/N 369 MER 10-1-97

UNCONTROLLED

JUN 01 1998

COPY WILL NOT BE REPLACED WHEN NEW REVISION IS RELEASED

ORIGINAL

		BCI INTERNATIONAL	
DRAWN BY: RJR	CHK. BY: M. H. Kinsey	ENG. APPR: H. S. Seltzer	
DATE: 10-9-96	DATE: 5-28-97	DATE: 5-28-97	
SCALE: NTS	DO NOT SCALE PRINT	S.A. APPR: J. A. S.	
THIS DOCUMENT CONTAINS CONFIDENTIAL AND/OR PROPRIETARY INFORMATION AND IS NOT TO BE REPRODUCED, DISTRIBUTED OR DISCLOSED WITHOUT PRIOR APPROVAL OF BCI INTERNATIONAL OR ITS DESIGNATED AGENT.		DATE: 5-28-97	
MATERIAL: 50 BURMETER PVC		MFG. APPR: M. S.	
FINISH: WHITE		DATE: 5-28-97	
TOLERANCE UNLESS OTHERWISE SPECIFIED: XX: ± .009 YD: ± .005 ANGLES: ± 1'		TOLERANCE UNLESS OTHERWISE SPECIFIED: XX: ± .009 YD: ± .005 ANGLES: ± 1'	
TITLE: CABLE, ASM SURE GRIP OVERMOLDED, FINGER PROBE 2		DRWG NO: 72035B3	REV: 4
		SHT. 2 OF 3	

BCI LEMO (VIAMED)

LED




RO



WB

SENSOR

CONN RV (ONE KEY REPEL)

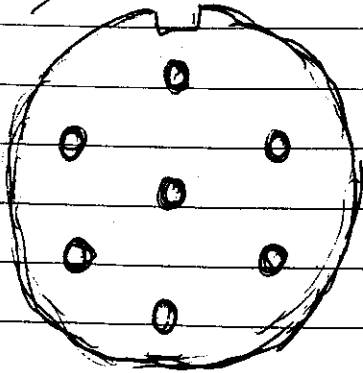



1340 Logan Avenue
Costa Mesa, CA 92626
Phone: (714) 545-3469
(800) 828-1599
Fax: (714) 545-7212

Pulse Oximeter Finger Probe Repair Procedure

Model: P861RA

1.1 P861RA Parts List

Item No.	Description	Quantity	Manufacture	Part Number
	Complete Clip Assembly	1	MCI	MC-P100A
1	Top Shell	1	MCI	MC-P101
2	Bottom Shell	1	MCI	MC-P102
3	Wire Spring	1	MCI	MC-P103
4	Buttons	2	MCI	MC-P104
5	Soft Finger Pad (LED)	1	MCI	MC-P105
6	Hard Finger Pad (Detector)	1	MCI	MC-P106
7	Top Pad Support Frame	1	MCI	MC-P107
8	Bottom Pad Support Frame	1	MCI	MC-P108
9	LED assembly	1	MCI	MC-P861-LED
10	Detector	1	MCI	MC-P861-DET
11	Complete Cable Assembly	1	MCI	MC-P861-CAB
	Cable Assembly			
12	Cable	1		
13	Amp Connector (male)	1		
14	Amp Cord Guard	1		
15	Amp Pins	7		
16	Probe Strain Relief	1		
17	Silicone (RTV)			
18	Glue			

Confidential and Proprietary Information
Not to be used without Written Authorization

1.2 P861RA Repair Procedure

1.2.1 Probe Disassembly:

1. Inspect cable, shell, pads and connector for damage or cracks. Note any damage and list part that need to be replaced.
2. Cut and discard cable (item 11) at the probe strain relief (item 16) to remove from the finger probe clip assembly.
3. Remove top pad support frame (item 7) with soft finger pad (item 5) from top shell. Remove soft finger pad from support frame. Remove bottom pad support frame (item 8) with hard finger pad (item 6) from bottom shell (item 2). Remove hard finger pad from support frame.
4. Separate shells (items 1 & 2) and remove spring (item 3) and buttons (item 4) to disassemble finger probe.

1.2.2 Probe Testing:

5. Remove LED assembly (item 9) from top pad. Inspect for physical damage. Test the individual RED and IR LEDs. The test specifications for the LEDs are listed in Table 1 below.
If item 9 is within test specifications then reuse LED assembly. Replace assembly if Red or IR LED failed to meet test specifications.
6. Remove Detector (item 10) from bottom pad. Inspect detector for physical damage and test the Detector. The test specifications for the LEDs are listed in Table 1 below.
If item 10 is within test specifications then reuse detector. Replace item 10 if detector failed to meet test specifications.

Table 1 LED and Detector Test Specifications

	Test		Typ.	Max.	Range	Units	Test Conditions
Red LED	Forward Voltage	V_F	1.8	2.4	± 0.6	volts	$I_F = 20 \text{ mA}$
IR LED	Forward Voltage	V_F				volts	
Detector	Forward Voltage	V_F				volts	


Confidential and Proprietary Information
Not to be used without Written Authorization

1.2.3 Probe Reassembly:


7. Inspect cable, connector and strain relief for damage. Replace cable assembly if damaged or cracked with the complete cable assembly (item 11). To build cable assembly attach cord guard (item 14) onto a 3 foot length cable (item 12). Attach wires (solder) onto pins (item 15). Attach main shield to pin 9. Attach jumper wires to pins 5, 8 and 9. Place pins into amp connector receptacle contact housing (item 13) according to connector diagram. Snap cord guard onto contact housing.

P861RA

CONNECTOR REAR-VIEW

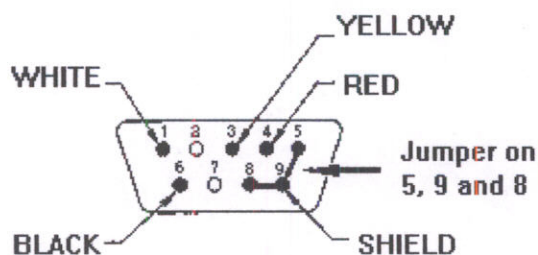

8. Attach probe strain relief (item 16) to cable (item 11). Strip wires back. Connect wires (solder) to LED and Detector. Connect the red and yellow wires to the Led terminals. Connect the black and white wires to the Detector terminals. Test continuity.

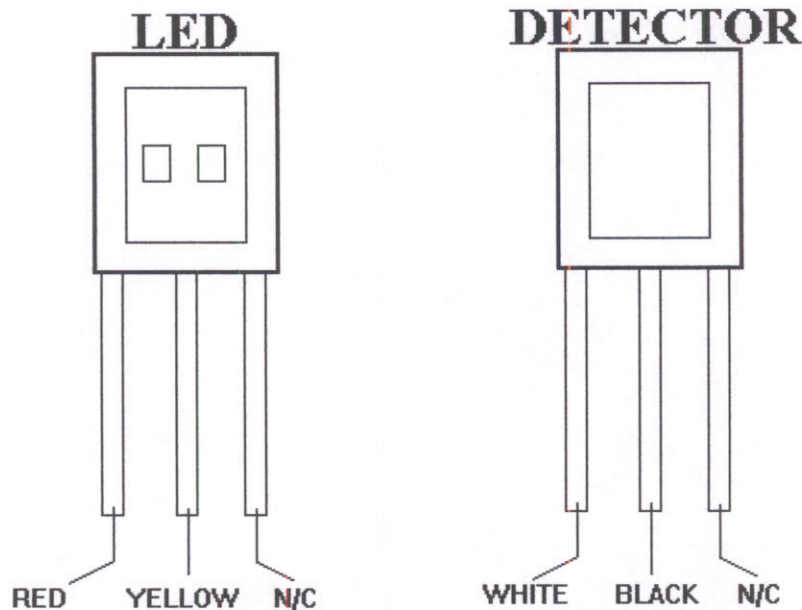

Confidential and Proprietary Information
Not to be used without Written Authorization

P861RA Finger Probe Repair continued

9. Inspect finger pads (item 5 & 6) for damage or cracks. Replace finger pads if damaged or cracked. Slide the LED into the top pad and silicone (item 17) into place. Slide the Detector into the bottom pad and silicone (item 17) into place. Let silicone dry.
10. Inspect top and bottom shell, button, and spring for damage or cracks. Replace items if necessary. Attach spring and buttons to top and bottom shell.
11. Inspect support frames and replace if damaged. Attach the top pad to the top support frame with glue (item 18). Attach the bottom pad to the bottom support frame with glue. Snap and glue top and bottom pads into top and bottom shell respectively.
12. Test probe for SpO₂ measurement.
13. Send probe to Quality Control for testing

Confidential and Proprietary Information
Not to be used without Written Authorization

Date	24/03/97	Type	BCI (EQUIVALENT E344)
bci.cmx		WORKS ON NELLCOR MACHINE & TEST BOX	

REAR VIEW

CUT HERE

DISCARD STRAIN RELIEF

On small porcelin clip style Sensor:

White goes onto small contact on black coloured sensor surface

Black goes to larger contact on reverse (Gold coloured)

SW 1

SW 4

Drawn By:	D.Lamb	Signed	
-----------	--------	--------	--