



Matt Shippam

 CID24160 Opera CID24160 Type:Vandagraph Retail

Matt Shippam 10 Harfield Court, , Bognor Regis, West Sussex, PO21 1EE, United Kingdom

Tel **07928481805** GBP Retail Price List Vandagraph (0010)

 Log Call

Add Issue

Proforma Credit Limit 0.00GBP

 Account Home

 Main Details

Account Memos

 Sales History

Stock References

○ Cust./Supp. Returns

 Service Visits

Distributor Agreements

Documents
Files

 Linked Issues

Regulatory Agreements

SRS68432

SRS Created : 29 Mar 2023

Created By Ryan Swaine

Add Watch

[Print Barcodes](#)

[Print A4 Sheet](#)

Split Items into Different SRS Numbers

Upload Document to SRS

Handle with care

Add Items to Supplier Returns Box

Add

☐ Return for Credit / Return to Stock
Unopened - will apply to all items on srs

☐ Clear From System - Force from System.
- will apply to all items on srs

Add notes to SRS

Add Notes

ADD DOCUMENT TO SRS : Document Description	Choose a file to upload: Choose file No file chosen
Next	

Service Repair Sheet 68432	
Company / Hospital	Matt Shippam
Contact Name	Matt Shippam
Department	
Position	
Company Phone	07928481805
Direct Phone	07928 481805
Opera Account	CID24160
Email	matt.shippam@sky.com
Order Number	
Viamed ORD Number	
Date Received Into building	29 Mar 2023
Type	Repair Approved
Issues?	291548

Notes
29/Mar/2023 Ryan Swaine
29/Mar/2023 Ryan Swaine
TEK-OX sn: TX2349
Sold: 2017
Not turning on, has had new batteries.
May also require new sensor.
29/Mar/2023 Robert Connor
Received 1 x tek-ox s/n TX2349, with blue tee and yellow plastic case.
<i>LINKED ISSUE NOTES:</i>
<u>291548:</u>
AutoGenerated Issue SRS68432 Appears to be Complete

Items for Repair

SRN	Equipment	Stock Reference	Barcode	Serialnumber	Description	Warranty New	Warranty Repair	Status	Repair Code	Time (H:M)
-----	-----------	-----------------	---------	--------------	-------------	--------------	-----------------	--------	-------------	------------

SRN35727	Oxygen Analyser TEK-OX	7910211	 967082	TX2349	Oxygen Analyser Tek-Ox	No	No	Complete - Repaired 06 Apr 2023 Michael Green	Not Switching On	00:30
SRN35727 Engineer Notes: INTERNAL NOTES not for customer discussion 05/Apr/2023 Michael Green The Tek-Ox analyser has been tested. The unit fails to power up. 05/Apr/2023 Michael Green The Tek-Ox analyser has been repaired. The battery contacts have been cleaned. The PCB has been reset in the case. The R-17VAN Oxygen sensor has been replaced with SN:107730. The batteries have been replaced. 05/Apr/2023 Michael Green The unit will be given a longer test as a precaution. 06/Apr/2023 Michael Green The unit has been tested numerous times. The unit is working as expected.										

Currently Viewing ITEM SRN35727 Oxygen Analyser TEK-OX 7910211 TX2349 Oxygen Analyser Tek-Ox

[Create a Meeting Issue](#)

Items Requested for Repair	Item notes	Engineer Notes	Notes To Customer
	The Tek-Ox analyser has been repaired. The battery contacts have been cleaned. The PCB has been reset in the case. The R-17VAN Oxygen sensor has been replaced with SN:107730. The batteries have been replaced.	05/Apr/2023 Michael Green The Tek-Ox analyser has been tested. The unit fails to power up. 05/Apr/2023 Michael Green The Tek-Ox analyser has been repaired. The battery contacts have been cleaned. The PCB has been reset in the case. The R-17VAN Oxygen sensor has been replaced with SN:107730. The batteries have been replaced. 05/Apr/2023 Michael Green The unit will be given a longer test as a precaution. 06/Apr/2023 Michael Green The unit has been tested numerous times. The unit is working as expected.	The Tek-Ox analyser has been repaired. The battery contacts have been cleaned. The PCB has been reset in the case. The R-17VAN Oxygen sensor has been replaced with SN:107730. The batteries have been replaced.

-