


**Tom Thumb
Infant T-Piece Resuscitator
Model: TT490-15 Series
Instructions for Use**

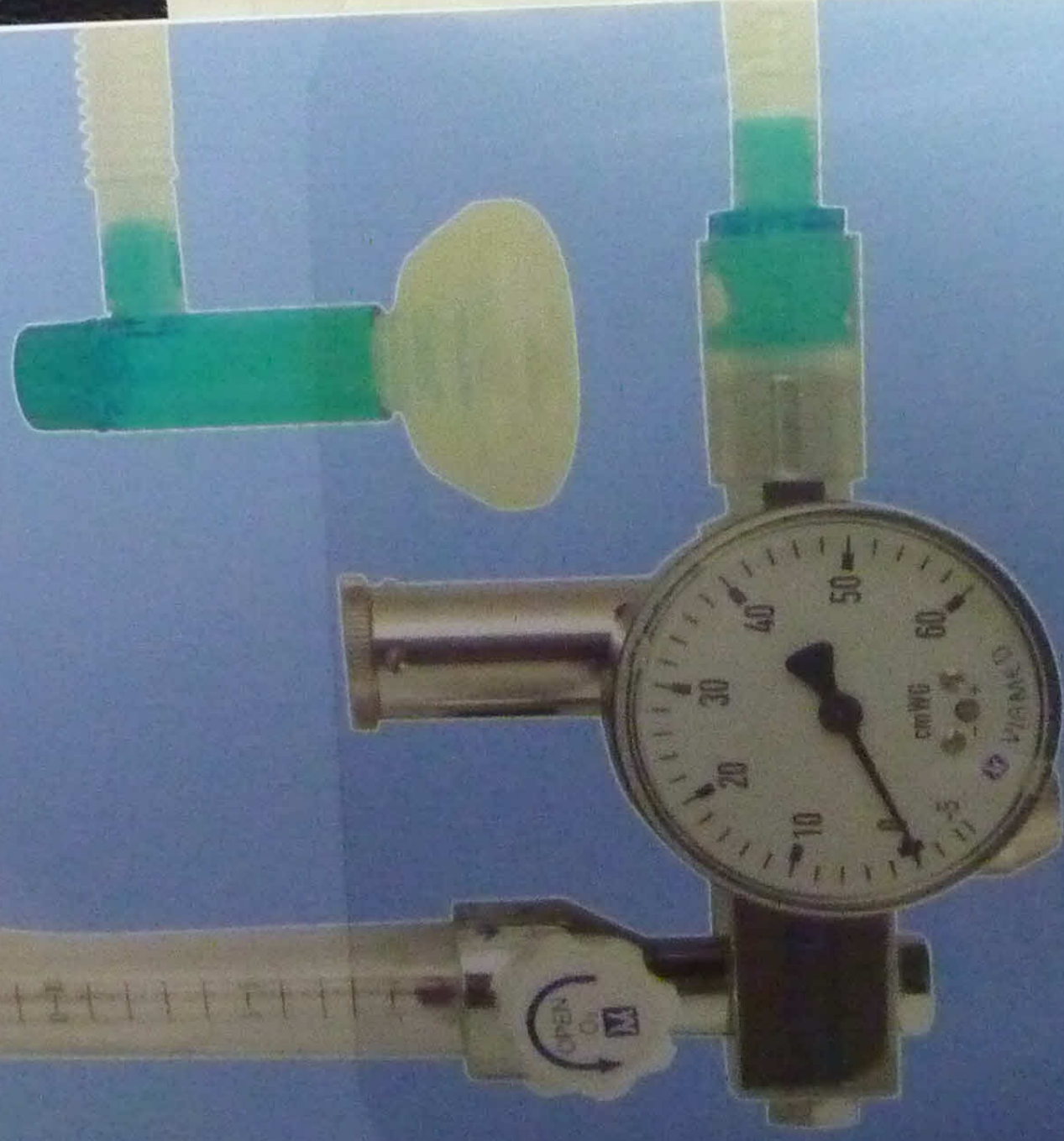

CE0086

 VIA MED

For use by qualified trained personnel only.
Recommended gas inlet pressure of 4 bar.
Flow rates within the range of the flowmeter.
Outlet pressure after altering the flow rate.
at $H_2O = 0.981mb$

8


- ### Guidelines for Use During Resuscitation
1. Follow the pre-use set up procedure and set the flow rate and outlet pressures, as described in the hospital's protocol for resuscitation. Ensure that the flowmeter and outlet pressures are checked prior to administering PEEP.
 2. Apply the mask to the patient's face. Ensure that the flow valve outlet to the patient's face is open and the T-piece is closed.
 3. Uncover and press the T-piece. Breathe into the T-piece and observe the patient's lungs to determine if the resuscitation steps to deliver PEEP are correct.
 4. Repeat steps 2 and 3 until the patient is breathing.

Tom Thumb

Infant T-Piece Resuscitator

Model: TT490-15 Series

Instructions for Use

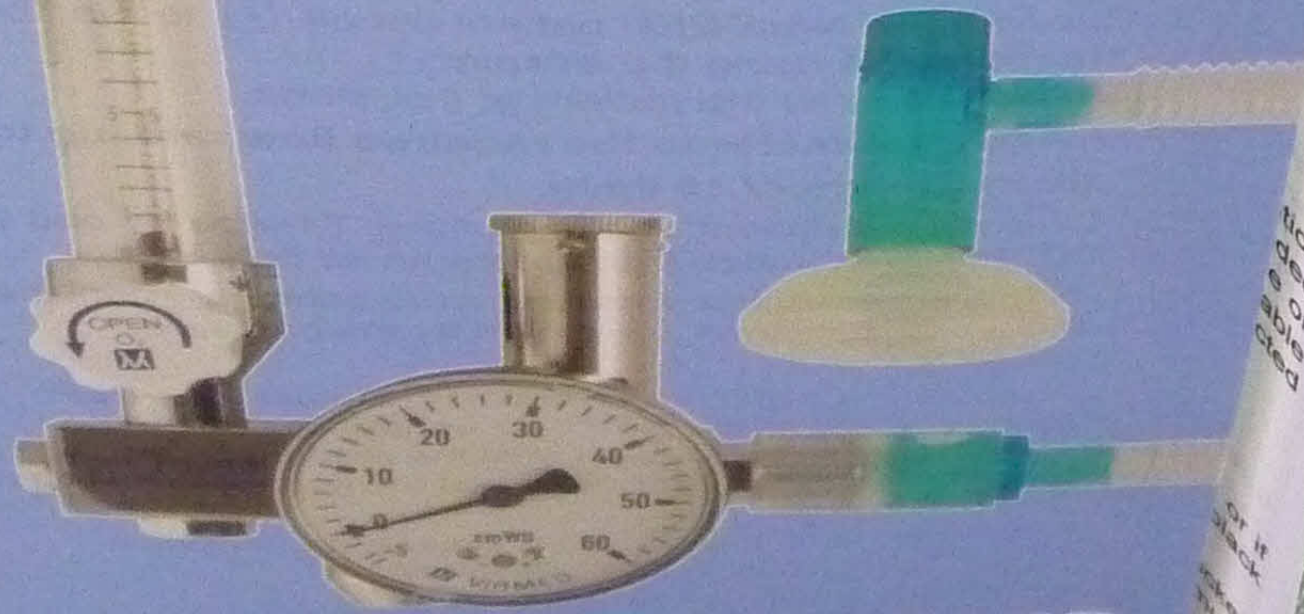

CE 0086

VIAMED