

VANDAGRAPH COMPANY OPERATING PROCEDURE**Final Test Procedure Vandagraph VN202 mkII
VM3/COP/40.17**

Date: 28 Jun 2007

Revision Date: 10-Oct-12

Issue 1

Only to be attempted by trained personel

Parts Required			
Quantity	Description	Part No.	
1	Oxycal		

1. Connect Oxycal to VN202mkII
2. Switch Oxycal to 100% and set VN202 mkII calibration to read 100% +/- 0.2
3. Switch Oxycal to 21% VN202 mkII should read 20.9 +/- 0.2
4. Switch Oxycal to 100% . Turn calibration until 88.8 or all the segments have been visible.
5. Autoswitch off versions should switch off after about 10 minutes