bluepoint MEDICAL	Compatibility of Components	Rev. 2
Internal Document	Multi Parameter Monitors	Page
internal bocament	VM-2500-M / S	1 of 3

Compatibility of Components - VM-2500-M / S:

Compatibility of Components 1111 2000 in 7 C.				
Hardware main / power PCB (Modification)	Firmware	PC- Software	Product CD	Remark
V1.9 / V1.1 (M1,M2)	V4.5 - V5.0	V1.3 & V1.2 - V1.4 Move power on/off to confirm key alarm reset to alarm-mute key.		Move power on/off to confirm key. Move alarm reset to alarm-mute key.
V1.9 / V1.1 (M1,M2,M3)	V5.1	V1.3 & V1.2 - V1.4 Changed the RTC routine for the RTC IC (PCF8563)		Changed the RTC routine for the new RTC IC (PCF8563)
V1.9 / V1.1 (M1,M2,M3)	V5.2 – V5.8	V1.6 V1.5 Updated the alarms log list. Ma alarm start and end time		Updated the alarms log list. Masked the alarm start and end time
V1.9 / V1.1	V 5.9	V1.7 – V1.8	V1.6 - V1.7	Added second to the alarms log. If Firmware V5.9 is used with PC software V1.6, the alarms are not displayed in the "Patient Info" Tab
V1.9 / V1.1 (NEW Boot Load Code)	V 7.5	V1.7 – V1.8	V1.6 - V1.7	1) Needs new Boot Load Code (solves switch on issue which resulted due to bug fixes and changes see below) 2) Only use sensors with Viamed sensor ID 3) Included "Home Care Mode" and "Hide Delete Data" and "Check gas accuracy" 4) Changed Start up Settings" 5) Included Finish&Polish 6) solved some cosmetic bugs
V1.9 / V1.1 (PCB date 20111212)	V 7.8	V1.7 – V1.8	V1.6 - V1.7	Solved "flash error" bug which is connected only to PCB with PCB date 20111212
V1.9 / V1.1	V 8.0	V1.8	V1.7	1) Solved timing problem with new Sensor ID chips 2) Solved switch on bugs related to "check gas accuracy mode" 3) Add switch off apnoe alarm in the Alarm settings by selecting " " 4) Data send via RTM is send every second (instead of every 4 seconds as in v7.5) >> NEW PC-Software required, is backward compatible down to v5.9

bluepoint MEDICAL	Compatibility of Components	Rev. 2
Internal Document	Multi Parameter Monitors VM-2500-M / S	Page 2 of 3

Compatibility of Components - VM-2500-MG:

Hardware main / power PCB	Firmware	PC-Software	Product CD	Remark
V1.9 / V1.1	1.3.0	V1.2	V1.0	Initial version based on CapnoTrue MG

Compatibility of Components	Rev. 2
Multi Parameter Monitors	Page 3 of 3

Document History Log

Rev.	Date	Modifications
0	16.11.2009	Initial version
1	19.07.2011	Included the Parameter "Product CD" and updated Firmware V5.9
2	16.08.2012	Added: Firmware update V7.5 - V8.0
		Included VM-2500-MG