


5/6/00 ~~857~~ 47


- 1. take lines out of shield.
- 2. take orange off. tin rest.
- 3. heatshrink connector. shields together. Pins on end. leave Black.

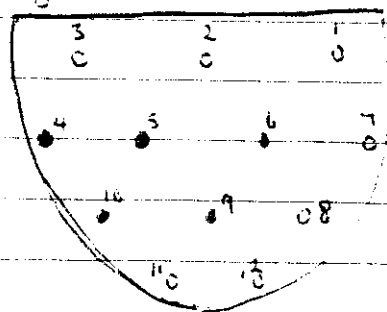


- 1. white
- 2. Black
- 3. shield
- 4. N/C
- 5. N/C
- 6. N/C
- 7. 50K
- 8. 50K
- 9. N/C
- 10. N/C
- 11. Red
- 12. Yellow

Hypertronics Male 12 Pin

Finger Probe

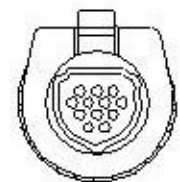



Enlarged front view

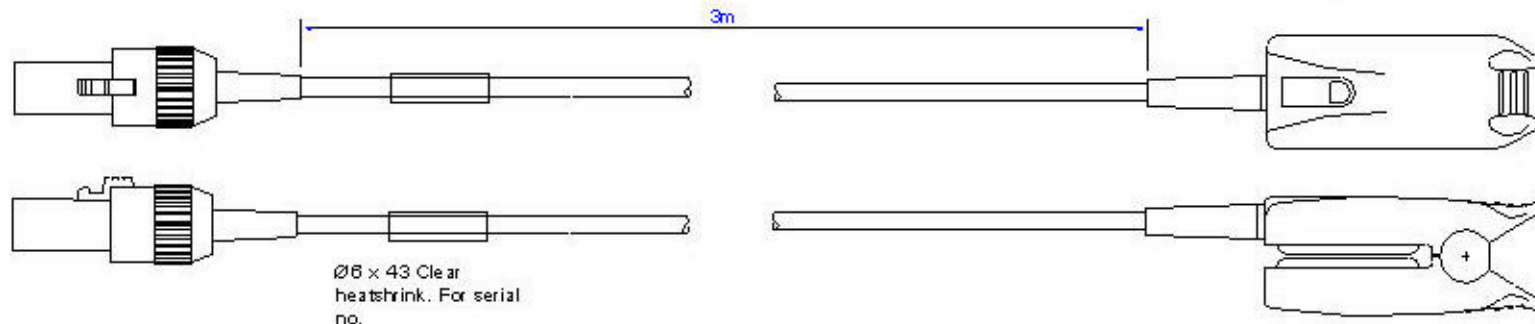

Pin Housing Rear View

(3,2) R1 = 10 Ω
(3,7) R2 = 80.4 K Ω
(3,8) R2 = 7.5 K Ω

(3)
Combined
outer and
inner
shields
(s+ os)

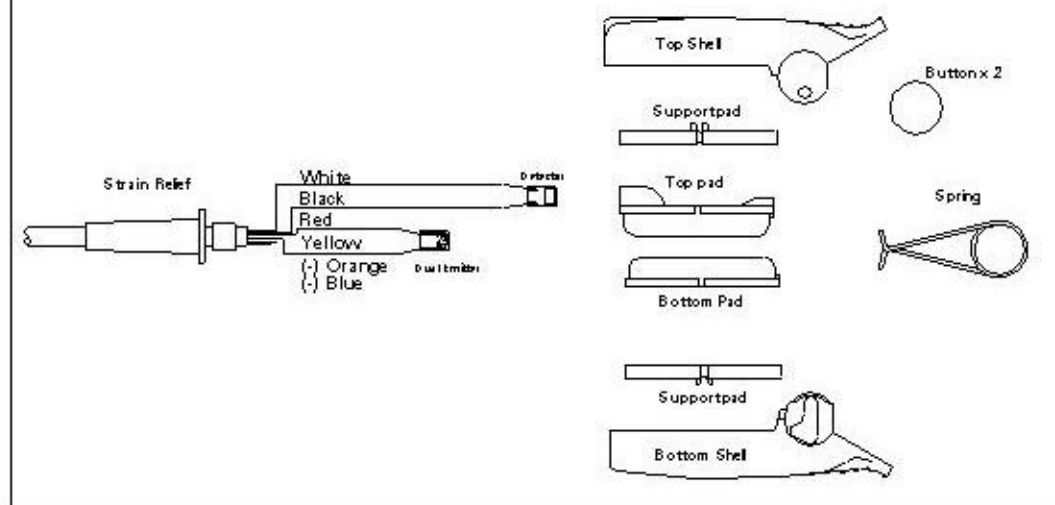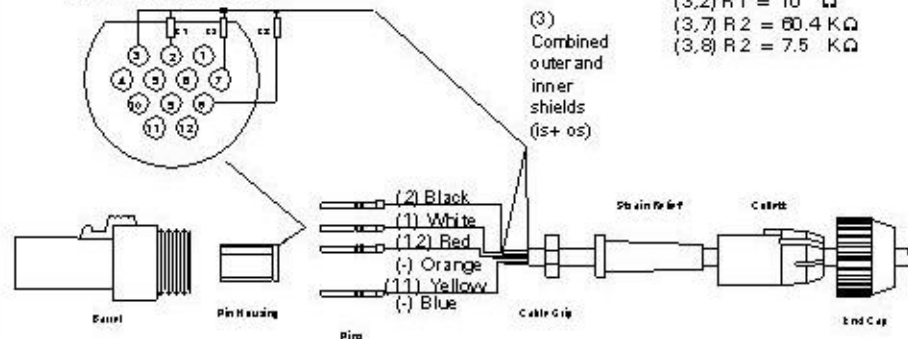

Title P857RA
Spacelabs



Dim in mm
Tol ± 0.2
Drawn J.Nirwan

Scale Not To Scale

Date 29/01/02

Dwg No.

SPF-857



VIAMED Ltd.
15 Station Rd
Cross Hills, Keighley
West Yorkshire
BD20 7DT

Material:

REV Date N° Drawn Approv Part No. 0018570

