SpiroTrue® A

Flow sensor

REF 1030132006

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EN Instructions for use

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bluepoint MEDICAL GmbH & CO. KG
An der Trave 15
23923 Selmsdorf, Germany
www.bluepoint-medical.com

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SpiroTrue® A – Flowsensor

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SpiroTrue® A – Flow sensor

Intended Use

Hot wire anemometric flow sensor, for measuring gas flow in Dräger anaesthesia and ventilation equipment. The sensor is compatible to:

- Spirolog - REF 8403735-06

Operation

- Due to fire risk - do not introduce any medicaments or other substances based on combustible solvents, such as alcohol, into the patient systems.
- Do not use explosive anaesthetic agents (e.g. ether or cyclopropane).
- After using with medicament nebulisers, immediately clean and treat the flow sensor as specified. Alternatively, install a filter upstream of the flow sensor.

Care

- Alignment, operation and sensor change must be carried out in accordance with the relevant device-specific instruction manual.
- Do not clean in automated cleaning or disinfection equipment. Do not clean with compressed air, water jets, brushes etc.
- Reusing the flow sensor may affect patient safety due to the possible contamination of the sensor.
- **Life cycle of the sensor is limited. It can be used as long as calibration in the anaesthesia or ventilation equipment is possible.**

Cleaning and Disinfection

The following cleaning agents and disinfectants can be used:

- Korsolex Basic/Extra/Plus (Bode Chemie, Hamburg, Germany)
- Gigasept FF/AF (Schülke&Mayr, Norderstedt, Germany)

Strictly follow the manufacturer's instructions on the disinfectant.

- Disinfect the flow sensor by immersion.
- Inspect the flow sensor for soiling and damage.
- Rinse the flow sensor under water (preferably sterile water) and then allow to dry in air for at least 30 minutes.

Sterilization

The flow sensor **cannot** be autoclaved or sterilized.

Disposal

The flow sensor must be disposed of as infectious waste.

Measuring range: 0 to 200 L/min, depending on the gas type and operating conditions

Accuracy: DIN EN ISO 80601-2-12,
DIN EN ISO 80601-2-13
Typically $\pm 8\%$ of measured value under calibration conditions depending on Ventilator(evaluation electronics)

Recommended operating conditions

- Temperature: 10°C to 40°C
- Relative humidity: 5% to 95%, non-condensing
- Atmospheric pressure: 700hPa to 1060hPa

Recommended storage and transport conditions

- Temperature: -20°C to 50°C
- Relative humidity: 5% to 95%, non-condensing
- Atmospheric pressure: 700hPa to 1060hPa

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