





# Pulse Oximeter Probe Wrap

*The Posey Probe Wrap replaces adhesive tape to secure all models of flexible optical transducers or probe sensors. It holds it firmly to prevent the misalignment of optical components while blocking out excessive ambient light. The soft foam tapeless fastener does not stick to the skin, allowing for easy application or removal while preserving delicate skin integrity. The universal design will fit a neonatal foot, infant toe, pediatric or adult finger. Posey Probe Wraps may be cut to any size.*

**#6554** (1 Dz/box)

## **⚠ WARNING**

- Make sure your patient has stable peripheral vascular function before you place a pulse oximeter sensor.
- Secure the sensor loosely with the Posey Probe Wrap, never wrap it tightly.
- Check the site immediately after applying the sensor to ensure that circulation is adequate.
- Include circulation checks with all patient assessments.
- Change the pulse oximeter site every 2 hours or according to your institution's protocol.
- Closely monitor patients with fragile skin, such as infants or elderly adults.

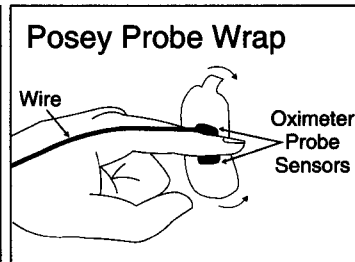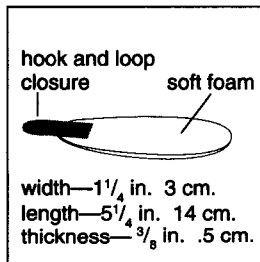

**J.T. Posey Company** 5635 Peck Road • Arcadia, CA 91006-0020 USA • Tel. 800-447-6739 or 626-443-3143 • Fax 800-767-3933 or 626-443-5014

EU AUTH. PERSON: (MDD 93/42/EEC) MDSS GmbH, Burckhardtstr. 1, D-30163, Hannover, Germany

© 1993 J.T. Posey Company. All rights reserved.

**CE**  
M1614 100998