Flowsensor A

P/N: 4310001

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P/N: 4310001

Intended Use

Hot wire flow sensor for measuring the flow in Dräger anaesthesia and ventilation units compatible with:

Description	Order-No.
Spirolog FlowSensor	8403735-06

Operation

- Do not introduce any medicaments or other substances based on combustible solvents, such as alcohol, into the patient systems. Fire risk!
- Do not use explosive anaesthetic agents (e.g. ether or cyclopropane)!
- After using with medicament nebulisers, immediately clean and treat the flow sensor as specified.
- Alternatively: Install a filter upstream of the flow sensor.

Care

- Alignment, operation and sensor change must be carried out in accordance with the relevant device-specific instruction manual
- Do not clean in an automatic cleaning/disinfecting machine. Do not clean with compressed air, water jet, brush etc.
- Reusing the sensor may affect patient's safety due to the possible contamination of the sensor.
- **Life cycle of the sensors is limited. It can be used as long as calibration in the evaluation unit is possible.**

Disinfecting



- Disinfect for approx. 1h in 70% ethanol or isopropanol solution.
- Then allow to dry in air for at least 30 minutes.

The following instruments disinfectants can be used:

- Korsolex Basic/Extra/Plus (Bode Chemie, Hamburg, Germany)
- Gigasept FF/AF (Schülke&Mayr, Norderstedt, Germany)

Sterilising

- Flowsensor A can not be autoclaved or sterilised

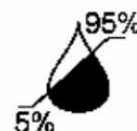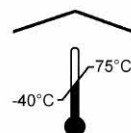Viamed Ltd

15 Station Road

Cross Hills

Keighley

West Yorkshire, BD20 7DT

United Kingdom