SpiroTrue® A  
P/N: 1030132006

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SpiroTrue® A  
P/N: 1030132006

#### Intended Use

Hot wire flow sensor for measuring the flow in Dräger anaesthesia and ventilation units compatible with:

| Description         | Order-No.  |
|---------------------|------------|
| Spirolog FlowSensor | 8403735-06 |

#### Operation

- Do not introduce any medicaments or other substances based on combustible solvents, such as alcohol, into the patient systems. Fire risk!
- Do not use explosive anaesthetic agents (e.g. ether or cyclopropane)!
- After using with medicament nebulisers, immediately clean and treat the flow sensor as specified.
- Alternatively: Install a filter upstream of the flow sensor.

#### Care




- Alignment, operation and sensor change must be carried out in accordance with the relevant device-specific instruction manual
- Do not clean in an automatic cleaning/disinfecting machine. Do not clean with compressed air, water jet, brush etc.
- Reusing the sensor may affect patient's safety due to the possible contamination of the sensor.
- **Life cycle of the sensors is limited. It can be used as long as calibration in the evaluation unit is possible.**

#### Disinfecting

- Disinfect for approx. 30 minutes in 70% ethanol or isopropanol solution.
  - Then allow to dry in air for at least 30 minutes.
- The following instruments disinfectants can be used:
- Korsolex Basic/Extra/Plus (Bode Chemie, Hamburg, Germany)
  - Gigasept FF/AF ( Schülke&Mayr, Norderstedt, Germany)

#### Sterilising

- SpiroTrue® A can not be autoclaved or sterilised

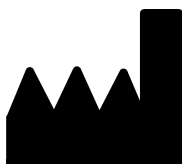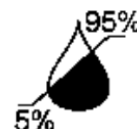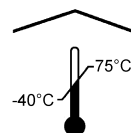bluepoint MEDICAL GmbH&Co.KG  
An der Tarve 15  
D-23923 Selmsdorf, Germany  
Fon.: +49 (38823) 548 8000  
Fax.: +49 (38823) 548 8029