



Superior Eye Protection for Neonatal Phototherapy


INSTRUCTIONS

Regular Size; Head Circumference over 30 cm
Premature Size: under 30cm.

- 1) Place the eye pads over the eyes.
- 2) The upper strap goes over the forehead.

- 3) **Pass the Biliband™ strap around and BELOW the bulge on the back of the head to oppose the forehead strap.**

- 4) Secure with the hook section (NOTE: it does not need to be tight).
- 5) Change the Biliband™ for a new one if this becomes over-stretched
- 6) The mid-line tabs are used to adjust the Biliband™ when secured.

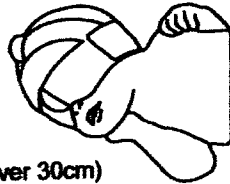

B 001 Regular Size (HC over 30cm)
B 002 Premature Size (HC under 30 cm)

DESIGNED & MANUFACTURED IN AUSTRALIA by

CAUTION:

- Avoid obstructing the nose and check regularly that this does not occur
- The Biliband™ should not be tightly stretched. It should be gently comfortable against the baby.

N
NASCOR
NEONATAL
AUSTRALIA

Single Patient Use Only

