

NeoMask

Phototherapy Eye Mask



Instructions for Use

1. The NeoMask is not a bandage; cover any existing skin abrasions appropriately before use.
2. Remove the NeoMask from the packaging and disconnect the rear fastener.
3. Position the NeoMask such that the blue eye pad covers the patient's eyes. The other tab (upper) is positioned across the patient's forehead.
4. Wrap the NeoMask around the head, so that it covers the ears - then secure together with the rear fastener.
Do not over tighten. Ensure that the mask is not hooked over the ears and that the ears are not folded over.
5. Positioning of the NeoMask can be adjusted by using the front: upper and lower tabs.
6. If necessary - for optimal fit, the upper tab can be adjusted. The two parts of this tab can be pulled part, then laid flat against the forehead: the end with the fastener wrapped over on top of the other.

 Monitor the position of the NeoMask on a regular basis and ensure that it does not obstruct the patient's nose. If the NeoMask becomes over-stretched, replace with a new one.



Large


1114015: Occipital-Frontal Circumference **32 - 38cm.** Pack of 20

Medium

1114016: Occipital-Frontal Circumference **26 - 32cm.** Pack of 20

Small

1114017: Occipital-Frontal Circumference **20 - 26cm.** Pack of 20

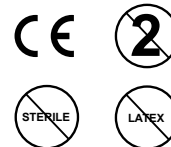

VIAMED

Viamed Limited · 15 Station Road · Cross Hills
Keighley · West Yorkshire · BD20 7DT · United Kingdom
Tel: +44 (0)1535 634 542 Fax: +44 (0)1535 635 582
Email: info@viamed.co.uk Website: www.viamed.co.uk