




### Superior Eye Protection for Neonatal Phototherapy

#### INSTRUCTIONS

Regular Size; Head Circumference over 30 cm  
Premature Size; under 30cm.

- 1) Place the eye pads over the eyes.
- 2) The upper strap goes over the forehead.
- 3) **Pass the Biliband™ strap around and BELOW the bulge on the back of the head to oppose the forehead strap.**
- 4) Secure with the hook section (NOTE: it does not need to be tight).
- 5) Change the Biliband™ for a new one if this becomes over-stretched
- 6) The mid-line tabs are used to adjust the Biliband™ when secured.

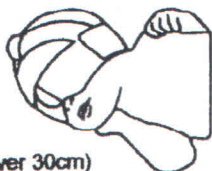

B 001 Regular Size (HC over 30cm)  
B 002 Premature Size (HC under 30 cm)

#### CAUTION:

- Avoid obstructing the nose and check regularly that this does not occur
- The Biliband™ should not be tightly stretched. It should be gently comfortable against the baby.

DESIGNED & MANUFACTURED IN AUSTRALIA by

**N**  
**NASCOR**  
**NEONATAL**  
AUSTRALIA

Single Patient Use Only



**NASCOR PTY. LTD.**

A.B.N. 81 002 965 960



Ground Floor, Unit 9  
252, Allambie Road  
Allambie Heights  
NSW 2100 Australia  
Tel: +61 2 9452 6244  
Fax: +61 2 9958 6383

## **NASCOR BILIBAND™**

This product is manufactured in Australia by Nascor Pty Ltd. Nascor Pty Ltd is certified ISO 9001/ EN 46001 by TUV Product Service.

The Biliband is a Class 1 (FDA exempt) product and is CE marked by Nascor Pty Ltd

The material used is a patented Elastic Nonwoven Fabric. It is manufactured in the USA and marketed as a bandage. It contains no natural latex, has been tested extensively for dermal irritation and has been cleared as a secondary wound dressing. It has been tested extensively in contact with babies' skin and is used in disposable nappies.

The Biliband™ has been tested for light transmission and the eyepads have been shown to attenuate more visible light than the other phototherapy masks on the market. The headband however does allow blue light to penetrate so the head continues to receive phototherapy.

The material however remains breathable and cannot obstruct respiration.

Signed

Dr Howard Chilton  
Managing Director  
NASCOR Pty Ltd

12/5/01

