SpiroTrue® A
P/N: 1030132006

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SpiroTrue® A
P/N: 1030132006

Intended Use

Hot wire anemometric flow sensor, for measuring gas flow in Dräger anaesthesia and ventilation equipment. The sensor is compatible to:

Description
Spirolog flow sensor

Operation

- Due to fire risk - do not introduce any medicaments or other substances based on combustible solvents, such as alcohol, into the patient systems.
- Do not use explosive anaesthetic agents (e.g. ether or cyclopropane).
- After using with medicament nebulisers, immediately clean and treat the flow sensor as specified. Alternatively, install a filter upstream of the flow sensor.

Care

- Alignment, operation and sensor change must be carried out in accordance with the relevant device-specific instruction manual.
- Do not clean in automated cleaning/disinfection equipment. Do not clean with compressed air, water jets, brushes etc.
- Reusing the flow sensor may affect patient safety due to the possible contamination of the sensor.
- **Life cycle of the sensor is limited. It can be used as long as calibration in the anaesthesia or ventilation equipment is possible.**

Disinfection

The following instrument disinfectants can be used:

- Korsolex Basic/Extra/Plus (Bode Chemie, Hamburg, Germany)
- Gigasept FF/AF (Schülke&Mayr, Norderstedt, Germany)
- Incidin Liquid (Fa.Ecolab Deutschland GmbH, Germany)

- Strictly follow the manufacturer's instructions on the disinfectant.
- Disinfect the flow sensor by immersion.
- Inspect the flow sensor for soiling and damage.
- Rinse the flow sensor under water (preferably sterile water) and then allow to dry in air for at least 30 minutes.

Sterilization

The flow sensor **can not** be autoclaved.

bluepoint MEDICAL

bluepoint MEDICAL GmbH&Co.KG
An der Trave 15
D-23923 Selmsdorf, Germany
Fon.: +49 (38823) 548 8000
Fax.: +49 (38823) 548 8029