SpiroTrue® APC
P/N: 3030131003

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SpiroTrue® APC
P/N: 3030131003

Intended Use

Hot wire anemometric flow sensor, for measuring gas flow in Dräger anaesthesia and ventilation equipment. The sensor is compatible to:

Description	Order-No.
Dräger Spirolife flow sensor	MK01900

Operation

- Due to fire risk - do not introduce any medicaments or other substances based on combustible solvents, such as alcohol, into the patient systems.
- Do not use explosive anaesthetic agents (e.g. ether or cyclopropane).
- After using with medicament nebulisers, immediately clean and treat the flow sensor as specified. Alternatively, install a filter upstream of the flow sensor.

Care

- Alignment, operation and sensor change must be carried out in accordance with the relevant device-specific instruction manual.
- Do not clean in automated cleaning/disinfection equipment. Do not clean with compressed air, water jets, brushes etc.
- Reusing the flow sensor may affect patient safety due to the possible contamination of the sensor.
- Life cycle of the sensor is limited. It can be used as long as calibration in the anaesthesia or ventilation equipment is possible.**

Disinfecting

The following instruments disinfectants can be used:

- Korsolex Basic/Extra/Plus (Bode Chemie, Hamburg, Germany)
- Gigasept FF/AF (Schülke&Mayr, Norderstedt, Germany)
- Incidin Liquid (Fa.Ecolab Deutschland GmbH, Germany)


- Strictly follow the manufacturer's instructions on the disinfectant.
- Disinfect the flow sensor by immersion.
- Inspect the flow sensor for soiling and damages.
- Rinse the flow sensor under water (preferably sterile water) and then allow to dry in air for at least 30 minutes.

Sterilising

- The sensor can be autoclaved (134°C).
- The sensor can be sterilized with ETO prior to use.

Disposal

- The flow sensor must be disposed of as infectious waste.

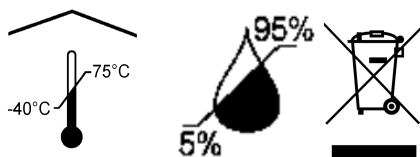bluepoint MEDICAL GmbH&Co.KG
An der Trave 15
D-23923 Selmsdorf, Germany
Fon.: +49 (38823) 548 8000
Fax.: +49 (38823) 548 8029