

VM3COP53.08 Ceratherm QA Test Procedure

- 1) Carefully remove the Ceratherm heater from the packaging.
- 2) Check the package contains:
 - A Ceratherm heater
 - Power cable – Euro lead
 - User manual/ ISO certificate/ mounting instructions – USB card
 - Quick start guide
 - Test record
- 3) Remove the Euro lead. A UK power lead will be added before the unit is dispatched.
- 4) Ensure the master power switch on the rear of the unit is in the off position.
- 5) Using a suitable power cable, connect the Ceratherm heater to a power outlet.
- 6) Turn the master power switch on.
- 7) Activate the unit using the on/off button on the front panel. The unit should start at power level 1.
- 8) Check the heating element indicator is illuminated and that the element begins to warm.
- 9) In turn, press each of the power level buttons and check the power level indicator is illuminated.



- 10) Press the left light on button. The Ceratherm light should come on at the 20% level.
- 11) Press the right light button to cycle through the light intensity levels.



- 12) With the unit set at power level 4, switch the mains power off. The unit should have an audible alarm and the alarm cancel button should be illuminated.
- 13) Press the alarm cancel button. The audible alarm should be silenced, but the illuminated button should flash on and off every few seconds.



- 14) Check the unit returns to the previous setting after leaving the mains power off for 15 minutes.
- 15) Power down the Ceratherm heater. Allow the unit to cool before repackaging.



- 16) Repackage the Ceratherm in the original packaging.
- 17) Record the QA results in the intrastats system.