


Cabinet Assembly.

Parts required..

- 1*Resus cabinet either (Pt No 0330005 or 0330105)
- 1*Bed base (Pt No 0320020 or ?))
- 1*Bed Mattress (Pt No 00320035 or 0320005)
- 6*Screws (Pt No 0330065)
- 1*Tom Thumb.
- 6*Viamed sticky labels
- 1*Viamed logo label.
- 1*patient circuit (Pt No 035005)
- 3*Drawers (Pt No 0310020)

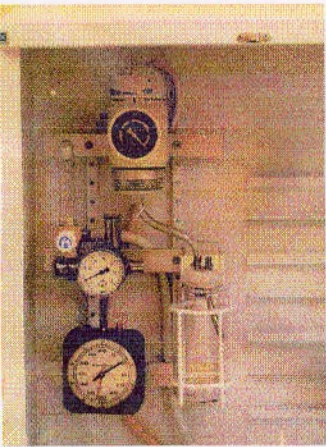

1. Fit bed base (Pt No 0320020) to drop down front panel ,300mm from bottom of front panel and 50mm from the left hand side whilst looking into cabinet, using 6 screws (Pt No 0330065). Check that mattress (Pt No 0320035) is the correct size for the bed.
2. Fit Viamed logo sticker (Pt No) to top left hand front corner of cabinet surround. So that it can be seen when front panel closed up.
3. Fit Viamed sticky labels to rear of suction unit and clock.
- 4.Remove CE label from the front of cabinet carefully and refix it to the bottom of the right hand side of the cabinet about 1 cm from the front and 3cm from the bottom, place Viamed sticky label just below it.
- 5.Last Viamed sticky label to be fitted inside the cabinet 1 cm above the top mounting rail and half its length.
6. Locate 3 drawers (Pt No 0310020) and put onto mountings on the right side oif the cabinet. Add patient circuit (Pt No 035005)
- 7.Fit completed timer, Suction assembly and Tom Thumbinto cabinet together with suction jar and check everything looks straight and that door closes.
- 8.Check that unit is clean and free of scratches.



Completed Cabinet with all parts fitted.