Disposable EEG Sensor Pediatric for measurement of the bispectral index

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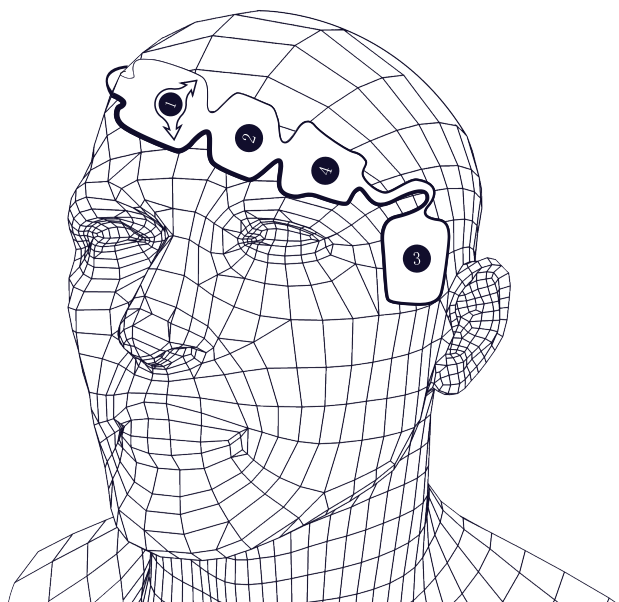
EN:

1. Intended Use:

The disposable EEG Sensor is used together with the Anesthesia Depth Monitoring for collecting and transmitting EEG biological signals to monitor the anesthesia depth of patients.

2. Warning:

To reduce the hazard of burns during use of brain - stimulating devices (e.g., transcranial electrical motor evoked potential), place stimulating electrodes as far as possible from the disposable bispectral index sensor and place sensor according to instructions.

3. Cautions:

Patient position may increase the risk of skin irritation on the forehead, with patients in prone position, consider minimizing pressure on the sensor.

4. Note:

Do not use if sensor is dry, to avoid dry out, do not open pack until ready for use. Due to intimate skin contact, reuse may pose risk of infection. If skin rash or other unusual symptoms develop, stop use and remove. Limited to short - term use (maximum 24hours). Do not cut sensor, as this will result in improper operation. Upon removal, slight redness of skin may occur and typically resolves within a short period of time.

5. Using Instructions :

1. Wipe the skin with wet gauze by alcohol and dry

2. Place the sensor diagonally on the forehead as shown

3. Squeeze sensor edges to ensure adhesion

4. Press ① ② ③ ④ sequence for 5 seconds to ensure adhesion

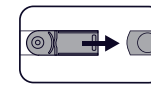
2.1 place ① position 5cm directly above the bridge of nose in the center of the forehead

2.2. Place ② directly above the eyebrows

2.3. Place ③ in the temple position

5. Insert the sensor plug into the interface cable

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Manufacturer: Medker Medical Electronic Tech (Shenzhen) Co., LTD
EU Represetative: Sungo Europe B.V.