



Jonathan Connor &lt;viamed.jonathan.connor@gmail.com&gt;

---

**AW: VM2500 Firmware**

1 message

**Froehlich, Heidi** <h.froehlich@bluepoint-medical.com>

3 June 2014 08:45

To: Jonathan Connor &lt;jonathan.connor@viamed.co.uk&gt;

Cc: Steve Nixon &lt;steve.nixon@viamed.co.uk&gt;

Good morning Jonathan,

Please find attached the firmware v8.6.

Changes:

- Included storage of frame error and checksum error (before we still had bugs in this feature which we implemented based on Risk Analysis review. Now it works fine)
- Solved a small minor rounding bug which occurred when changing from vol% to mmHg in CO2. It caused the PC software value in RTM to be different by 1 digit compared to the device in some limited measurement ranges
- In the service menu changed the name "Home care mode" to be "alarm delay mode". We found this requires less explanation regarding the function

During software update some bugs were solved which occurred based on the changes (that's why it is v8.6 and not v8.2;)

Do you like the new case for VM-2500? Please let me know if you have questions in this regard.

The top inlay had a hole unintendedly. You can see we needed to repair it fast. The next devices will have a closed surface top inlay.

In future we also want the Inlay which shows which model version is in the case to be a little larger to have less issues regarding tolerances at the edges.

Let me know if you have questions.

Mit freundlichen Grüßen

Best regards

Heidi Fröhlich

Bluepoint MEDICAL GmbH & Co. KG

An der Trave 15

23923 Selmsdorf, Germany

tel: +49 (0) 38823 - 548 8032

fax: +49 (0) 38823 - 548 8029

email: [h.froehlich@bluepoint-medical.com](mailto:h.froehlich@bluepoint-medical.com)

web: <http://www.bluepoint-medical.com>

Trade register: Schwerin, Reg.-No.: HRA 2600

Diese E-Mail enthaelt vertrauliche und/oder rechtlich geschuetzte Informationen. Wenn Sie nicht der richtige Adressat sind oder diese E-Mail irrtuemlich erhalten haben, informieren Sie bitte sofort den Absender und vernichten Sie diese E-Mail. Das unerlaubte Kopieren sowie die unbefugte Weitergabe dieser E-Mail sind nicht gestattet.

This message may contain confidential and /or privileged information. If you are not the addressee or authorized to receive this for the addressee, you must not use, copy, disclose or take any action based on this message or any information herein. If you have received this message in error, please advice the sender immediately by reply e-mail and delete this message. Thank you for your cooperation.

---

**Von:** Jonathan Connor [<mailto:jonathan.connor@viamed.co.uk>]

**Gesendet:** Dienstag, 3. Juni 2014 09:32

**An:** Froehlich, Heidi

**Betreff:** VM2500 Firmware

Good Morning Heidi

Please can you send me the 8.6 Firmware and what has changed since 8.1A

Kind regards

Jonathan Connor

<http://www.viamed.co.uk>

Email [jonathan.connor@viamed.co.uk](mailto:jonathan.connor@viamed.co.uk)

Tel: +44 (0)1535 634542

Fax: +44 (0)1535 635582

Twitter: [twitter.com/ViamedLtd](https://twitter.com/ViamedLtd)

Facebook Page: Search for Viamed Ltd

Company registered in England, No. 01291765.

## E-mail Disclaimer

-----

The information in this email is confidential and may be legally privileged.

It is intended solely for the addressee. Access to this email by anyone else is unauthorized.

If you are not the intended recipient, any disclosure, copying, distribution or any action taken or omitted to be taken in reliance on it, is prohibited and may be unlawful.

If you feel you have received this email in error or would like to be removed from our mailing list, please reply with delete in the subject line or call [+44 \(0\)1535 634542](tel:+441535634542)

-----

**VM\_2500\_v8.6.zip**

204K