

Version: 1.1	
<b>Digital Apgar Timer Service Sheet</b>	
Hospital / Organisation:	
Location (department, room no.):	
Engineer (print):	
Service Date:	
<b>Action</b>	<b>Further action required</b> <b>OK</b>
Serial number:	
Batteries replaced Comments:	<input type="checkbox"/> <input type="checkbox"/>
Check for overall signs of damage Comments:	<input type="checkbox"/> <input type="checkbox"/>
Check START button enters <i>Counting Mode</i> Comments:	<input type="checkbox"/> <input type="checkbox"/>
Check FREEZE button enters <i>Freeze Mode</i> Comments:	<input type="checkbox"/> <input type="checkbox"/>
Check FREEZE button resumes <i>Counting Mode</i> Comments:	<input type="checkbox"/> <input type="checkbox"/>
Re-enter <i>Freeze Mode</i> , check START button resumes <i>Counting Mode</i> Comments:	<input type="checkbox"/> <input type="checkbox"/>
Check RESET button enters <i>Stand-by Mode</i> Comments:	<input type="checkbox"/> <input type="checkbox"/>
Start a stopwatch whilst simultaneously pressing START (note: there may be a slight pause before the timer begins counting, ensure they are running in synchronisation before proceeding)	
Check single beep audible indication at 1 minute Comments:	<input type="checkbox"/> <input type="checkbox"/>
Check two beeps audible indication at 5 minutes Comments:	<input type="checkbox"/> <input type="checkbox"/>
Check three beeps audible indication at 10 minutes Comments:	<input type="checkbox"/> <input type="checkbox"/>
Check count is synchronized with the stopwatch at 10 minutes, $\pm 1$ sec Comments:	<input type="checkbox"/> <input type="checkbox"/>
Check mounting clamp and bracket are securely fastened Comments:	<input type="checkbox"/> <input type="checkbox"/>
Clean unit as detailed in the Operator's Manual Comments:	<input type="checkbox"/> <input type="checkbox"/>
<b>Additional notes or comments:</b>	