

## Service Repair Sheet 68798

Contact Name	Dean Young
Company/ Hospital Name	Hull Royal Infirmary
Department	Clinical Engineering
Position	Senior Medical Equipment Technician
Direct Phone	01482608973
General Phone	01482608783
Opera Account	00002265
Email	dean.young3@nhs.net
Order Number	SRS68798
Date Received	29/May/2024
Booked in By	Robert Connor

Repair	Ref	S/N	Equipment Type	Complete - Repaired
SRN36471	0310030	040895	Tom Thumb	Time :0 Hour(s)

## Parts Replaced

Qty: 1	0380000
Qty: 1	0330236

Tom Thumb TT480 040895 has been stripped and cleaned then reassembled with new O rings. The two valves have been checked and reset before QA.

I have replaced the gauge face and set the gauge back to zero. I have confirmed the gauge is accurate with my calibrated manometer.

Repair	Ref	S/N	Equipment Type	Complete - Repaired
SRN36472	0310030	440060	Tom Thumb	Time :0 Hour(s)

## Parts Replaced

Qty: 1	0380000
--------	---------

Tom Thumb TT480 440060 has been stripped and cleaned then reassembled with new O rings. The two valves have been checked and reset before QA.

Repair	Ref	S/N	Equipment Type	Complete - Repaired
SRN36473	0310030	VMCJ48	Tom Thumb	Time :0 Hour(s)

## Parts Replaced

Qty: 1	0380000
--------	---------

Tom Thumb TT480 VMCJ48 has been stripped and cleaned then reassembled with new O rings. The two valves have been checked and reset before QA.

I have freed up the adjusting screw on the precision valve. I used my hot air gun to heat it up and turned the screw clockwise, it's far easier than trying to undo the screw into more Permabond. I've cleaned off as much of the Permabond as I could, the screw is easy to adjust now.