


BE 117 Non-Rebreathing 'T'

15mm I.D./22mm O.D. x 22mm O.D. x 15mm O.D./22mm O.D.

- ◆ Designed for use with a spontaneously breathing patient where back flow must be avoided
- ◆ Intended to be used with a spirometer
- ◆ Two silicone one-way valves allow inhaled gas to be delivered exclusively through proximal port and exhaled through the distal port

Directions for use:

1. Connect spirometer to a clean BE 117 'T'.
2. Place BE 117 in-line.
3. Take measurement.
4. Remove BE 117 from circuit.
5. Remove BE 117 from spirometer.
Clean BE 117 after each use.
6. Use a clean BE 117 for next patient.
(See sterilization procedures on other side of sheet)

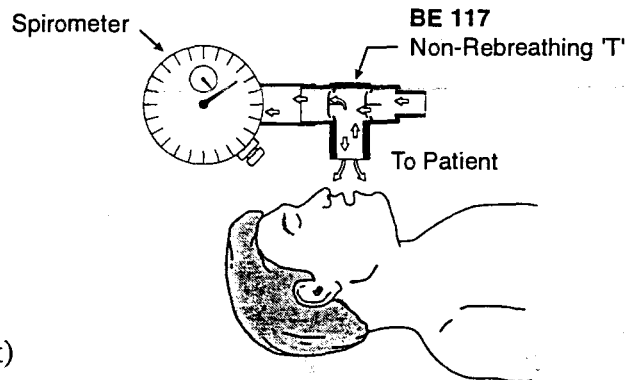

INSTRUMENTATION INDUSTRIES, INC.

2990 Industrial Boulevard Bethel Park, PA 15102
Phone: 1-412-854-1133 Customer Service: 1-800-633-8577

