

capnography

FLOCAP[®]

Flow Indicator & EtCO₂ Detector



- High definition colour changing material displays CO₂ %
- Patented flow indicator spinner
- Highly visible
- Long shelf life, currently tested to 36 months
- A quick way to determine end expiratory breath in addition to waveform CO₂ monitors
- If "waveform CO₂ monitors are not available, a nonwaveform carbon dioxide detector device in addition to clinical assessment is an alternative" hence the Maxtec FLOCAP

Source: NAP4 Report 4th National Audit Project of The Royal College of Anaesthetists




The FLOCAP is a single-use EtCO₂ & flow indication device designed for visualisation of a patient's exhaled CO₂ and expiratory flow. The EtCO₂ indication assists the user in verifying proper ET tube placement. The flow indicator helps the caregiver identify end of exhalation which may help prevent breath-stacking or overdistention and its associated health issues during resuscitation.



FLOCAP®

Flow Indicator & EtCO₂ Detector

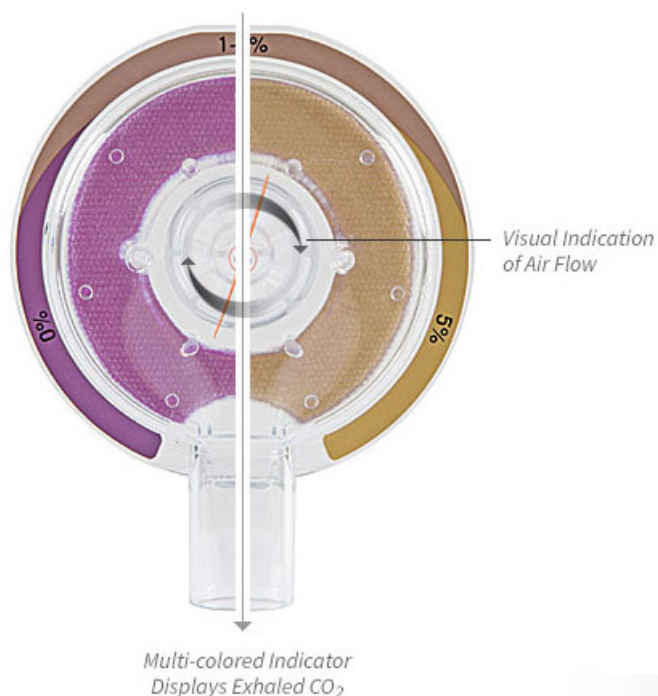

Colour indications

Purple (0%)

No carbon dioxide detected

Beige (1-2%)

Exposed to CO₂ approx 1-2%

Yellow Beige (2-5%)

At intermediate CO₂ concentrations of 2-5%

Yellow (5%)

Exposed to 5% or greater CO₂

Permanent Yellow

Exhausted or damaged indicator

Spinning*

Patient is exhaling

Not spinning

Exhalation flow < 1 LPM

* At the end of exhalation, the flow indicator's momentum will cause it to spin briefly after exhalation has ceased.

Flow Sensitivity

1 LPM

CO₂ detection 0%, 1-2%, >5%

Use duration 24 hours

Storage temperature 5°C (41°F) to 25°C (75°F)

Use temperature 10° C (50°F) to 40°C (104°F)

Internal volume <25ml

Leak rate according to ISO 9360-1 0.00 ml/min

Compliance according to ISO 9360-1 0.44 ml/kPa

Device weight 23g

Pressure drop 0.7 cmH₂O @ 30 LPM, 2.7 cmH₂O @ 60 LPM, 5.7 cmH₂O @ 90 LPM

Connector port (patient end) 15 mm Inner Diameter/22 mm Outer Diameter

Connector port (ventilator end) 15 mm Outer Diameter

Patient size >15 Kg (33 lbs)

Part Number

Maxtec FLOCAP: Pack of 24 4410100

Specifications Subject to Change

CE0123



VIAMED

Viamed Limited · 15 Station Road · Cross Hills
Keighley · West Yorkshire · BD20 7DT · United Kingdom
Tel: +44 (0)1535 634 542 Fax: +44 (0)1535 635 582
Email: info@viamed.co.uk Website: www.viamed.co.uk



Part Number: 4490000
Date: 10/15