patient warming




Ceratherm 600-3 Radiant Warmers

For maintaining the body temperature of infants, children and adults

- Ceramic 600-Watt heating element with fast warm-up
- Four programmable warming settings
- Computer-controlled heat regulation and alarm monitoring
- LED lighting for dazzle-free illumination of the treatment area
- Two programmable levels of illumination intensity with soft transitional on/off



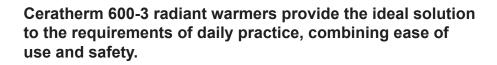

Ideal for use in delivery suites, special care baby units, clinics, paediatric wards, accident & emergency and operating theatres.

Available in several configurations including: wall mounted, ceiling mounted and mobile floor standing.








0310300

Warmer only no mounting bracket

Model: 521A-60020-3-EX



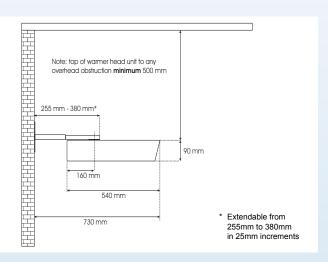


With fixed wall mounting bracket, not pivotal

Model: 521A-60023-3-EX



Standard Radiant Warmer used in conjunction with Viamed Infant Resuscitation Cabinets

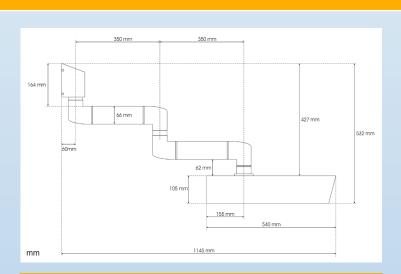


With double hinged arm & wall mount bracket, with integrated electrics

Model: 521A-60022-3-OND

0310302

0310301




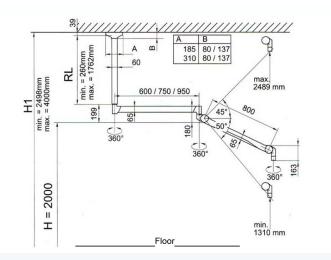

Ceiling mounted and with height adjustment* Model: 521A-60024-3-EX

0310303




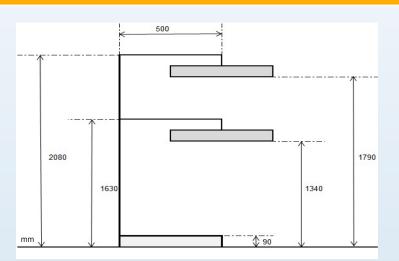

On mobile floor stand Model: 521A-60035-3-EX

0310304



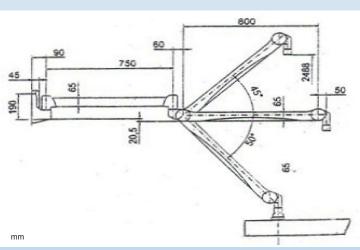

0310305 With double-jointed wall mounting & with height adjustment

Model: 521A-60025-3-EX

Ceratherm 600-3 Radiant Warmers

Controlled radiator performance for optimal warming and maximum patient safety.

The Ceratherm 600-3 range of radiant warmers facilitate temperature stabilization, and by assisting in maintaining a neutral thermal environment - they counter the loss of body heat to the environment by convection. The overhead radiant warmers offer unrestricted access to the patient and also allow several people to care for the patient at the same time - such as during infant resuscitation or other emergency procedures.

The operating controls are arranged in a clear format and are easily activated. There are four pre-set levels of warming and each level can be programmed separately. Dazzle-free LED lighting with soft transitional on/off provides natural colour temperature illumination of the treatment area. There are two illumination levels and each one can be programmed in 10% increments, from to 10 –100% illumination. The lighting also switches off automatically after 20 minutes in standby mode; this function can be disabled if required.

Technical Data

Mains voltage
Heating element
Lighting
Radiator performance
Colour
Certified according to
Conformity
IP Protection Class
Heating element guarantee
Contamination Level Class
Electrical Protection Rating
Medical Product Class
Applied Standards

230 V ±10%, 50-60 Hz Ceramic, 600 W LED lamp 12 V / 1 W 4 steps, 0 - 99% RAL 9010 IEC 601-2 TÜV/CE

CE (Directive 93/42/EEC, Annex II)
IP20
5 years

Class I (with protective earth connection)
llb (Directive 93/42/EEC, Annex IX, Rule 9)
EN 60601-1:2007, EN 60601-2-21:2010
EN 60601-1-2:2006

Alarms

- Acoustic and visual 15-minute reminder alarm. As a safety feature if it is not acknowledged, then the heat output is automatically switched to 20% of the maximum output
- Mains Failure
- Technical malfunction (with error code)

Warmer Dimensions Weight	55 cm (L) x 21 cm (W) x 9 cm (H) 4.3 kg (without accessories)
Weight	4.5 kg (without accessories)

weight	4.3 kg (without accessories)	
Part Numbers		
Ceratherm 600-3	warmer only - no mounting bracket	0310300
Ceratherm 600-3	with fixed wall mounting bracket, not pivotal	0310301
Ceratherm 600-3	with double hinged arm & wall mounting bracket, with integrated electrics	0310302
Ceratherm 600-3	ceiling mounted and with height adjustment	0310303
Ceratherm 600-3	on mobile floor stand	0310304
Ceratherm 600-3	with double-jointed wall mounting & with	0310305



Front panel controls and indicators:

Heat setting 1 (illuminated button).

Output setting programmable - default 20%

Heat setting 2 (illuminated button). Output setting programmable - default 50%

Heat setting 3 (illuminated button). Output setting programmable - default 75%

Heat setting 4 (illuminated button). Output setting programmable - default 98%

Heat output LED indicator

Alarm reset (illuminated button)

Lamp (low setting). Illumination level programmable – default 20%

Lamp (high setting). Illumination level programmable – default 100%

ON/Standby (illuminated button)

C€0123

Specifications Subject to Change All drawings are not to scale

Distributed by:



VIAMED

Viamed Limited · 15 Station Road · Cross Hills Keighley · West Yorkshire · BD20 7DT · United Kingdom Tel: +44 (0)1535 634 542 Fax: +44 (0)1535 635 582 Email: info@viamed.co.uk Website: www.viamed.co.uk



Part Number: 0390102 Date: 03/15

