

VM-2500 CO₂/SpO₂ Monitor

High performance capnography with either mainstream or sidestream measurement

- Robust and ergonomic design
- Fast warm-up time
- Rapid response time
- Extremely accurate and reliable measurement
- CO₂ and SpO₂ combined
- Displays EtCO₂, FiCO₂, SpO₂, respiration and pulse rate, capnogram and plethysmogram
- Maintenance and calibration-free technology
- Clear and bright, colour display offers superior clarity
- For intubated and non-intubated patients



VM-2500-M
Mainstream Unit



VM-2500-S
Sidestream Unit



The ultra-compact, ergonomic design of the VM-2500-M and VM-2500-S combine outstanding performance and reliability in mainstream or sidestream CO₂ and SpO₂ monitoring.

The VM-2500-M and VM-2500-S combine innovative technologies and design with the latest materials, to create extremely accurate and reliable measurement of intubated or non-intubated patients.

The mainstream unit utilises the advanced IRMA™ CO₂ Analyser; to precisely determine gas concentration in the mixture.

The sidestream unit utilises the advanced ISA™ CO₂ Analyser; with a range of sampling solutions to eliminate problems caused by water secretions affecting the reading.

These units incorporate a clear and bright colour display technology, offering unrivalled clarity, in even the most difficult of lighting conditions.

The VM-2500-M and VM-2500-S can be used in a variety of applications, from use in emergency rescue situations, during surgery, intensive care, resuscitation and patient transportation.

The ultra-compact, ergonomic design of the VM-2500 combines outstanding performance and reliability in for either mainstream or sidestream CO₂ and SpO₂ monitoring.

Mainstream Standard Package

- Mainstream Unit
- IRMA™ CO₂ analyser
- IRMA™ Airway adapter (Adult and Paediatric)
- Reusable SpO₂ sensor*
- USB data cable
- Power supply (UK and EU Plug)
- 4 x AA batteries
- 1 x Li-Poly battery (CT-2500)
- Silicone protective cover
- PC software
- Instruction manual on CD

* Please select from the following sensor styles:
Silicone Soft Sensor (Adult or Paediatric), Finger Sensor, Silicone Wrap Sensor or Ear Sensor, and indicate at the time of ordering.



Key Features of the VM-2500-M

- Mainstream CO₂ measurement with the **IRMA™ CO₂ Analyser**
- Fast warm-up time of < 10 seconds to full specification
- 'PLUG-IN and MEASURE...™' technology - **no integration procedures** or associated expenses required
- **No occlusion by water or mucus**
- **IRMA airway adapter with XTP™ non-condensing** light transmission window
- Small, lightweight and shock-resistant; the **IRMA™ CO₂ Analyser** weighs less than 30g
- Direct measurement **without time delay**
- For intubated and non-intubated patients
- Maintenance and calibration-free technology
- Wide range of accessories and SpO₂ sensors available



Mainstream Measurement

The **IRMA™ CO₂ Analyser** sets new standards in CO₂ mainstream monitoring and provides reliable, safe and easy CO₂ monitoring by direct measurement, without time delay.

- Small and lightweight
- 'PLUG-IN and MEASURE...™' technology
- Shock resistant

The **IRMA™ Airway Adapters** are supplied in two versions; Adult/Paediatric and Infant.

- Non-condensing light transmission XTP™ window
- Single patient use



Applications

The VM-2500 with Silicone Finger Sensor is suitable for the following applications:

- Patient transport within and between hospitals
- Rescue services and emergency medicine
- Resuscitation
- General and Paediatric ICU
- Ventilator management and weaning



Nomo Adapter



Sampling Line



Airway Adapter
(15 mm)

Sidestream Standard Package

- Sidestream Unit
- Nomo adapter
- Sampling Line
- Sidestream airway adapter (Adult/Paediatric)
- Reusable SpO₂ sensor*
- USB data cable
- Power supply (UK and EU Plug)
- 4 x AA batteries
- 1 x Li-Poly battery (CT-2500)
- Silicone protective cover
- PC software
- Instruction manual on CD

* Please select from the following sensor styles:

Silicone Soft Sensor (Adult or Paediatric), Finger Sensor, Silicone Wrap Sensor or Ear Sensor, and indicate at the time of ordering.



Key Features of the VM-2500-S

- Sidestream CO₂ measurement with the **ISA™ CO₂ Analyser** and Nomo technology
- Fast warm-up time of < 10 seconds to full specification
- The **Nomo Adapter** uses Nomo technology which **removes water** and **vapour** from the sampled gas
- The Nomo Adapter includes a hydrophobic bacterial filter to **protect against water intrusion and cross contamination**
- Specially designed for **low flow applications**; 50 ml/min sampling flow rate
- For intubated and non-intubated patients
- Maintenance and calibration-free technology
- Wide range of accessories and SpO₂ sensors available

Sidestream Measurement

The VM-2500-S offers two sampling solutions; a reusable Nomo adapter used in conjunction with single patient use sampling lines, or VersaStream; a complete one-piece single patient use solution.

Both Nomo adapter and VersaStream technologies incorporate a unique water separation polymer with an integrated hydrophobic bacterial filter, passing water vapour and condensed water through its surface and evaporating into the surrounding air whilst leaving the carbon dioxide concentration unaffected.

The Nomo adapter allows for up to two weeks of continuous monitoring with high moisture content breathing gases when used with single patient use sampling lines; VersaStream sampling lines offer a fully disposable one-piece sampling solution for short-term (≤ 24 hours) or long-term (≤ 72 hours) use. This offers greater flexibility of choice for the end-user.

Additional Sidestream Accessories




Sidestream
airway adapter
available in
Adult/Paediatric



Airway adapter
and sampling
line



Nasal
sampling line

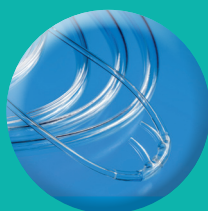

Nasal sampling
cannula available in
Adult and Paediatric




Nasal/oral
sampling cannula



15mm airway
adapters range
available,
I.D 2.0mm to
10.0mm


Screen Display Options

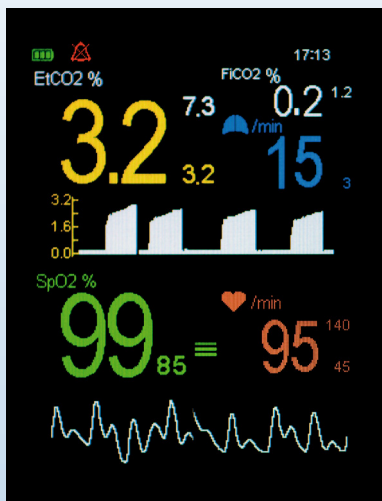


Standard Display Format



Large Digit Display

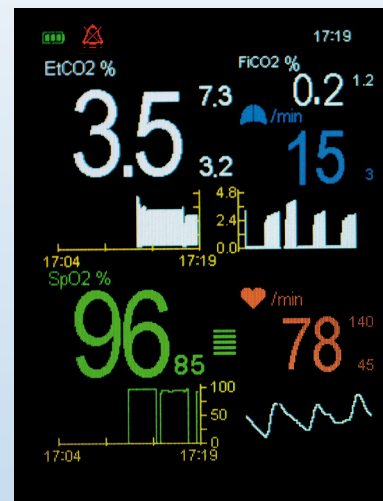

Trending Digit Display Format

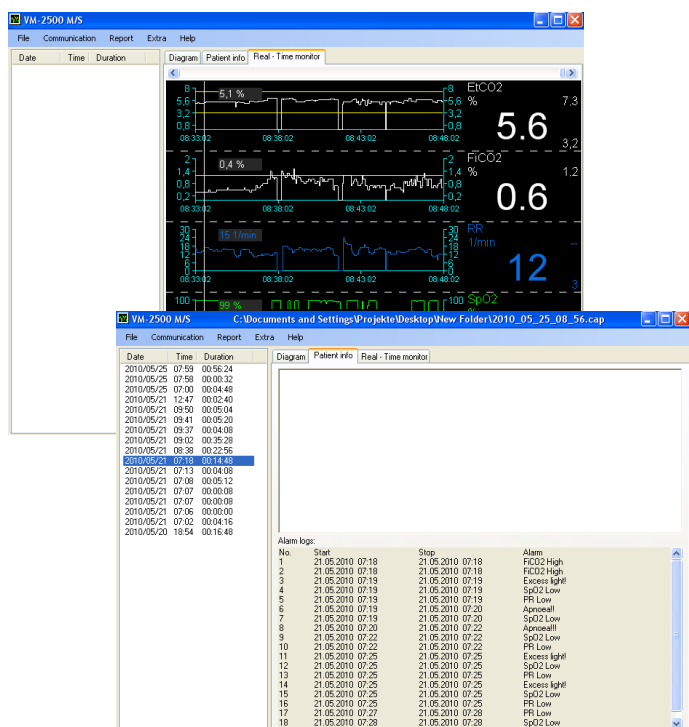

Powerful Capnograph PC Software

With the convenient Capnograph PC Software all measured values, selected alarm limits and alarm messages can be transferred to the computer via the USB interface. The data can be viewed on the computer and patient information added. The PC software enables basic statistical evaluation of the stored data.

The software can also be used to display and store the measurement values and alarm messages on the computer in parallel to ongoing measurements. This function is activated by setting the real-time mode of the Capnograph. When this mode is activated, the device transfers the actual measurement values for etCO_2 , FiCO_2 , SpO_2 , Respiration and Pulse Rate measurement values every second to the computer via the USB interface.

Available Sensors

A Also available in Autoclavable version



NB. When ordering the VM-2500 please select from the following reusable sensor styles: Soft Sensor (Adult or Paediatric), Finger Sensor, Silicone Wrap Sensor, Ear Sensor and indicate at the time of ordering. Autoclavable sensors are available.

VM-2500 CO₂/SpO₂ Monitor

Measuring Range	
EtCO ₂ and FiCO ₂	0 - 15 Vol%, or expressed as kPa or mmHg
SpO ₂	0 - 100%
Respiration Rate	0 - 150 breaths/min
Pulse Rate	20 - 300 bpm
Accuracy	
EtCO ₂ and FiCO ₂	± (0.2 vol% + 2% of reading) ± (0.3% vol % + 4% of reading) inc. interfering gases
SpO ₂	± 2% (70 - 100%)
Respiration Rate	± 1 digit at 60 breaths/min
Pulse Rate	± 1 digit (≤ 100/min) or ± 1% (> 100/min)
Display	
Features	Colour OLED, 262,000 colours, 240 x 320 pixels
Data displayed	End-tidal CO ₂ and inspired CO ₂ in volume %, kPa or mmHg, oxygen saturation, respiration rate, pulse rate, capnogram, plethysmogram, bar graph, short term trends
Indicators	Signal strength and signal quality, pulse amplitude, battery status, alarm silenced, volume silenced, neonatal mode, time, storage capacity full
Trend Information	
Short Term Trend	15 min/1 hour/6 hours (stored and displayed)
Long Term Trend	Approximately 400 hours (stored)
Environmental Conditions	
Operating temperature	0 - 50°C, 15 - 95% R.H. (non-condensing), 60 - 120 kPa (excl. Li-Poly battery)
Storage temperature	-30 - 70°C, 10 - 95% R.H. (non-condensing), 60 - 120 kPa (excl. Li-Poly battery)
Regulatory Approvals	
Standards	CE Class IIb in accordance with MDD 93/42/EEC BS EN 60601-1:2006, BS EN 60601-1-1:2001, BS EN 60601-1-2:2007, BS EN 60601-1-8:2004, EN 60529, EN ISO 21647:2004, BS EN ISO 9919:2005 Class II, Type BF
Safety	
Assembly	
Construction	IPX1
Dimensions	150 mm (L) x 75 mm (W) x 35 mm (H)
Weight	Approximately 400g (complete device with batteries)
Power Supply	Power supply 90 - 260 VAC/6 VDC, 50 - 60 Hz, 2.5 Ah Li-Poly battery or 4 x AA alkaline batteries
Battery Life	AA batteries: with full functionality VM-2500-M > 6 hours, VM-2500-S > 3 hours Li-Poly battery: with full functionality VM-2500-M > 7 hours, VM-2500-S > 5 hours
PC interface	USB 2.0
Part Numbers	
VM-2500-M Mainstream Capnograph* EAN-13 (GTIN-13) Barcode: 5051826002711	4410500
VM-2500-S Sidestream Capnograph* EAN-13 (GTIN-13) Barcode: 5051826002728	4410520
* Language Package: Central European, including: English, Dutch, French, German, Italian, Spanish, Swedish and Russian (additional language versions available on request)	
VM-2500-M - Asia Languages (English, Chinese and Japanese)	4410501
VM-2500-S - Asia Languages (English, Chinese and Japanese)	4410521
Additional Accessories Available	
Pulse Oximetry Extension Cable, 1.2m cable length (XT6500)	0014895
Pulse Oximetry Extension Cable, 2.4m cable length (XT6501)	0014896
Silicone Protective Cover	0022160
Carrying Case	4420620
Carrying Case with clear front	4420621
Protective Rubber Boot	4420615
Universal Mounting Plate Rev. 1	4420600
Universal Mounting Plate Rev. 2	4420601
'V' mount wall plate	0121180
'V' mount pole clamp (Small: 14 - 25mm span)	0121181
'V' mount pole clamp (Large: 16 - 40mm span)	0121182
'V' mount rail clamp	0121184
'V' mount pole clamp, multi-position	0121197
Universal Mounting Clamp, with vertical and horizontal adjustments	0121200
IRMA Holder	4420613
ISA Medirail Holder	4420614
Silicone Cover for use with IRMA	4420612

NB. When ordering either the VM-2500-M or VM-2500-S please select from the following reusable sensor styles:
Soft Sensor (Adult or Paediatric), Finger Sensor, Silicone Wrap Sensor, Ear Sensor and indicate at the time of ordering.
Autoclavable sensors are available.

Specification Subject to Change

Mounting Options

For use with VM-2500 range

 4420612 Silicone cover for use with IRMA	 4420615 Protective Rubber Boot includes carrying strap	 4420600 Rev. 1 4420601 Rev. 2 Universal Mounting Plate
 0121180 'V' Wall Mount	 0121197 'V' Mount Pole Clamp Multi-Position	 0121184 'V' Mount Rail Clamp
 0121182 'V' Mount Pole Clamp Large: 16 - 40mm span	 4420613 IRMA Holder	 4420614 ISA Medirail Holder

Carrying Cases for VM-2500



- Fluid repellent
- Non rot
- U.V. stable
- Washable at 60°C

- Self healing and waterproof zips
- micAgard silver-lining inners
- Abrasion and tear resistance
- High visibility and reflective properties



Distributed by:



Viamed Limited · 15 Station Road · Cross Hills
Keighley · West Yorkshire · BD20 7DT · United Kingdom
Tel: +44 (0)1535 634 542 Fax: +44 (0)1535 635 582
Email: info@viamed.co.uk Website: www.viamed.co.uk



Part Number: 0092190
Date: 03/16