



Viamed Limited - 15 Station Road - Cross Hills
Keighley -West Yorkshire BD20 7DT - United Kingdom
Tel: +44 (0)1535 634542 Fax: +44 (0)1535 635582
Email: info@viamed.co.uk Website: www.viamed.co.uk

Microstim DB3 Supramaximal Nerve Stimulator

Part number: 2510001



Caution:

This is a non-standard version unit and is easily identified as it has green switches; the standard version (part number 2510000) has red switches.

Functionality of the 2510001 stimulation is as follows:

T.O.F. Now latching switch – Continuous cyclic output, T.O.F. which repeats after a pause of 8 seconds - for a total period of 5 minutes.

D.B.S. Momentary switch - Functionality as with standard DB3

P.T.C. Momentary switch - Functionality as with standard DB3

1 Hz . Now latching switch – Continuous output repeats for a total period of 5 minutes.

NOTE: With either of the revised T.O.F. and 1Hz functions the latching switches need to be reset to the middle position otherwise another function cannot be selected.

After use - if either of the T.O.F and 1 Hz functions are left in the latched (On) position there is still a small current drain, even after the five minute time-out period. This current consumption will drain the battery unless the switches are returned to the middle (Off) position.

19th January 2010