


233

PRESS-STUD
FITTINGS FOR
DETACHABLE
FRONT

PERSPEX HOOD

OXYGEN INLET


HOOD PIVOT

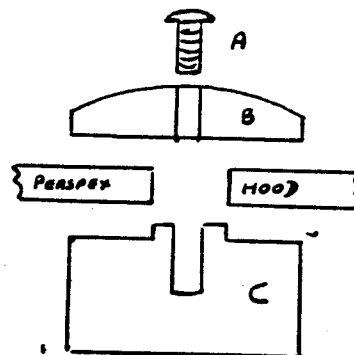

HOOD PIVOT

Assembly Instructions:

1. Remove screws 'A' and white washers 'B' from both sides of chair Hood Pivots.
2. Fit Perspex Hood to chair, locating Hood Pivot holes on to the small diameter raised section of Pivot Part 'C'
3. Screw 'A' via 'B' and the Hood into the screw hole in the centre of Part 'C'

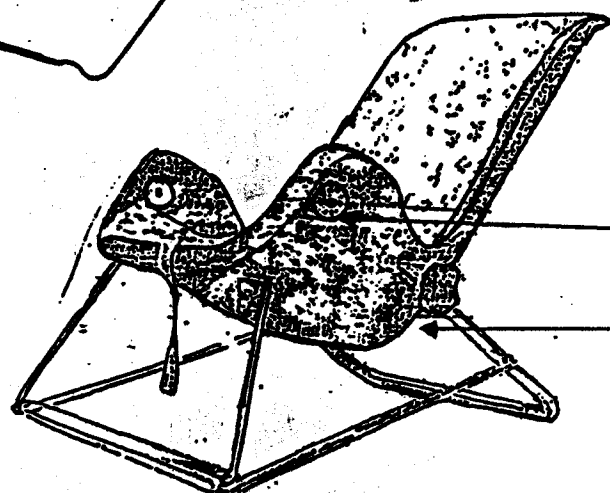

HOOD PIVOT

STANDARD BABY CHAIR
WITH VARIABLE TILT



VIAMED
15 Station Road, Cross Hills,
Keighley, West Yorkshire.
Tel. Cross Hills (0535) 34542