

**COMPANY OPERATING PROCEDURES**

Incoming Sensor Testing R-30V (PN: 0110040)  
VM3/COP/37.55

Date: 3-Feb-03

Revision date: 2-Feb-17

Issue: 3

**Test & Equipment Required**

Oxygen sensor gas barrier bag 3.5 x 5 inch (PN: 0150030)

**Method.**

1. Open the packaging and remove the O<sub>2</sub> sensor.
2. Check the O<sub>2</sub> sensor for damage and signs of leaking electrolyte.
3. Scan the Barcode into the QA form in intrastats, and complete the form.
4. Reseal in oxygen sensor gas barrier bag, 3.5 x 5 inch (PN: 0150030)
5. Seal in oxygen sensor packing box (PN: 0150012) and label with Viamed blue sensor label