

Instruction for Use



Product name

Neonatal Phototherapy Eye Mask

Intened use:

The product is intended to protect the eyes during the phototherapy treatment.

Materials:

Nonwoven fabric, composite bonding fabric, spandex bandage, Velcro tab, medical silicone gel.

Product description

It consists of mask body and shield pad. It is made up of nonwoven fabric, composite bonding fabric, spandex bandage, Velcro tab, medical silicone gel. The mask body is divided into fixed and movable types according to the forehead fixing method. The product is non-sterile, for single use only.

Models and Specifications

Type I Large / Type I Medium / Type I Small

Type II Large / Type II Medium / Type II Small

Type III Large / Type III Medium / Type III Small

Type IV Large / Type IV Medium / Type IV Small

Type IV-N Large / Type IV-N Medium / Type IV-N Small

Type V Medium / Type V Small

Type V-N Medium / Type V-N Small

Type VII Large / Type VII Medium/Type VII Small

Large specification is applicable to the neonate with head circumference of 30-38 cm.

Medium specification is applicable to the neonate with head circumference of 24-33cm.

Small specification is applicable to the neonate with head circumference of 20-28cm.

Shelf time: 3 years

Duration of time: Less than 24 hours

Intended patient population: Neonate

Intended users: The user shall be trained medical staff

Contraindications:

1. Neonate with allergic constitutions is prohibited from using this product, should use it with caution or follow the doctor's advice.
2. Neonate with eye infections or unhealed infections should use this product after recovery and follow the doctor's advice.

Warnings:

1. Please read this instruction for use and the contraindications, precautions and usage methods carefully before using this product.
2. The mask body and shield pad of this product should be used together.
3. This product is for one-time use only by a single patient. Do not cross-use or reuse.
4. Do not pull hard before use. Pay attention to adjusting the position and tightness of the product to ensure that the protective position is correct and the tightness is appropriate.
5. Please pay attention to the use of this product and strengthen the inspection during use. If you

find that the mask body is loose or loses its stickiness and cannot be used continuously, please replace it with a new eye mask immediately; if you find that the shield pad is contaminated by the neonate's eye secretions, please replace it with a new eye mask.

6. During the use of this product, slight marks, local redness or red spots may appear. These are normal phenomena and will heal or disappear quickly after discontinuation of use.

Instruction for use:

1. Select appropriate specification according to the neonate's head circumference.
2. When using, please ask the medical staff to gently place the product on the neonate's head and fix it, and at the same time apply the shield pad to the neonate's eyes, and then adjust it to a reasonable and appropriate position.
3. When using Type II, Type III, Type IV, Type IV-N, Type VII, medical staff should press the fixed mask body and the Velcro tab as tightly as possible at the contact part with Velcro tab. During use, please avoid the Velcro tab from contacting the neonate's skin to avoid rubbing the neonate's skin.
4. When using Type V and Type V-N, please ask the medical staff to clean the skin around the neonate's eyes and keep the skin dry; when using, please ask the medical staff to first remove the release paper on the back of the product, apply the shield pad to the neonate's eyes, and press the medical silicone gel patch except the shield pad with your fingers to make it completely adhere to the skin; during use, please pay attention to observation. If the neonate is found to have severe spontaneous sweating, turn off the phototherapy device first, remove the product, clean the sweat and sebum on the neonate's skin surface, and then put on the product again.

Storage and Transportation

1. The packaged product can be transported by the means of transportation. During transportation, they should be protected from sunlight, rain, harmful gases and liquid contamination.
2. The packaged product should be stored at -10°C~40°C, in a dry, cool and clean room without corrosive substances.

Production date:

See package

LOT No:


See package

Manufacturer Information



Ontecklong(Nanjing) Medical Supply Corporation Limited
Address: Bldg 5, No. 2 Future Road, Park Zhongshan, Liuhe district, Nanjing, 210048
Jiangsu, P.R China
Tel: 86-25-51552233 Fax: 86-25-84233533

**European
Authorized
Representative**

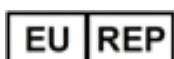

SUNGO Europe B.V.
Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel, The Netherlands