

VM3COP51.30 Resus Cabinet Assembly

Parts list		
Qty	Description	Part No.
1	Resuscitation Cabinet	0330105
320mm	Edging Grommet Strip 1.6mm to 2.0mm	0330012
1	Resuscitation Bed 14" flat	0320021
6	Self-Tapping Screw No. 6 x 0.5 inch	0330064
1	Mattress large - infant resuscitation	0320043
1	Resuscitation Cabinet Rail - 0.3m length	0332058
2	Resuscitation Cabinet Rail - 0.4m length	0332059
8	Resuscitation Cabinet Rail - Spacer 20mm	0332039
8	Resuscitation Cabinet Rail - Bolt 35mm	0332041
8	Resuscitation Cabinet Rail - Lock nut	0332043
3	Storage bins: Size 3 (blue)	0310020
1	Label: Viamed (domed)	0390056
1	Label: Support door (domed)	0390057
1	Label: Caution	0390054
1	Label: Serial number	0390053

Tool list
Pozi-drive No.2
Pozi-drive No.3
Drill with 2mm bit
10mm open ended spanner
Ruler
Pencil
Hand held vacuum cleaner

Revision date: 16-Feb-17

- 1) Unpack and inspect the cabinet for damage. Faults or damage must be reported to the warehouse team leader.



- 2) Affix the front labels as shown. These must be central and straight.



- 3) Affix the serial number label to the right-hand side of the cabinet as shown. This should be central and 90mm from the base of the cabinet.



- 4) Affix the rubber door stoppers to either side of the door opening.



- 5) Cut a length of edging strip to 320mm.
- 6) Insert the protective strip around the edges of the inlet access hole located in the base of the cabinet.



- 7) Inside the cabinet, on the rear panel, are three rail mounts. Place a nut behind the rail mount.



- 8) Place piece of folded card behind the nut to hold it in place.

The standard arrangement for these is: Top, bottom and fifth from bottom.

The rails are of two lengths. The top one is short, the two lower rails are long.



- 9) Place a spacer on the top side of the rail mount. Ensure that the spacer aligns with the nut.

- 10) Repeat this process on each rail.



Revision date: 16-Feb-17

- 11) Place a counter sunk bolt through each hole in the rail.

Note: The rail has channel on the lower side.




- 12) Affix each rail to the rail mounts.

The rails can be fitted in several positions to suit customer requirements. The standard positions are:

Top – short rail
One below middle – Long rail
Bottom – Long rail



- 13) Using a pencil, mark 50mm from the left side of the cabinet door. This measurement must be taken from the panel and not the rubber edge strip.

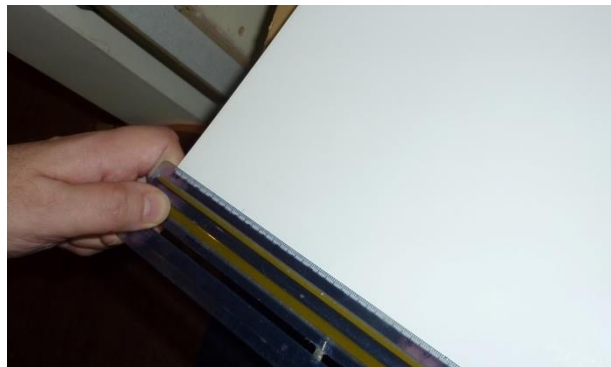

Revision date: 16-Feb-17

- 14) Place the bed on the cabinet door.
And align with the pencil mark.



- 15) Align the top of the bed with the top of the door panel. This alignment must be from the panel and not the rubber edge strip.



- 16) Using a pencil, mark through each of the six holes in the bottom of the bed.



- 17) Using a 2mm drill bit, drill pilot holes to a depth of 10mm at each mark.

- 18) Using a hand-held vacuum cleaner, remove any swarf.



Revision date: 16-Feb-17

19) Using the six fixing screws, affix the bed in place.



20) Ensure that the door closes with the bed in place.

21) Leave the mattress in the original packaging and place on top of the bed.



22) Wrap the three plastic storage bins in bubble wrap and place them on the shelf in the top right corner.



23) Affix the 'Caution' label on the inside of the door. The label should be 5mm from the edges.

