

Revision date: 16-Aug-17

VM3COP37.59 0110436 MAX-250B Oxygen Sensor Packing Procedure

Tools required:**Description:**

Heat sealer

Parts list:**Part number:**

0150012

0150020

9992515

Description:

Oxygen sensor box

Oxygen sensor barrier bag

Label – Medical Oxygen sensor Viamed


- 1) Take the MAX-250B Oxygen sensor and place it in an Oxygen sensor barrier bag. Use the heat sealer to seal the bag.



- 2) Take the sealed sensor and package it in a sensor box.



Revision date: 16-Aug-17

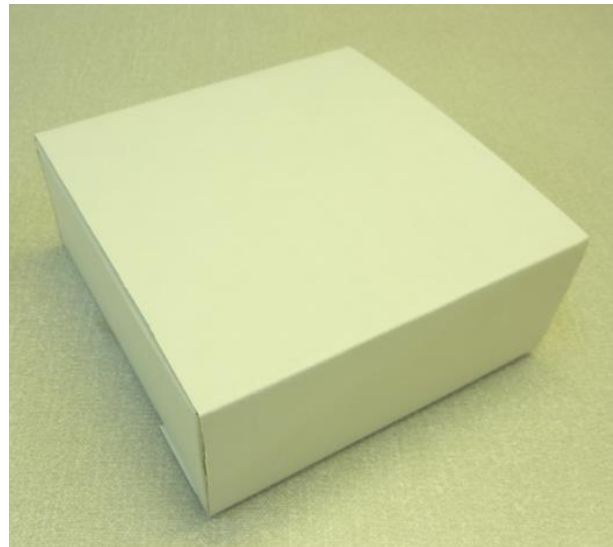

- 3) Affix the box label to the box by wrapping from the top to the bottom, ensuring the label is straight and without air bubbles or creases.



- 4) Affix the serial number label to the front of the box in the space provided.

