





# VIAMED



## Testing Spring P/N 9901-26

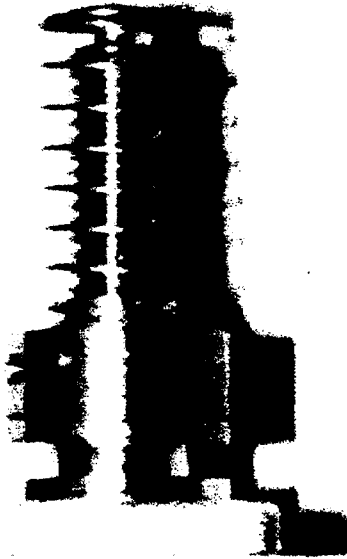

Pass

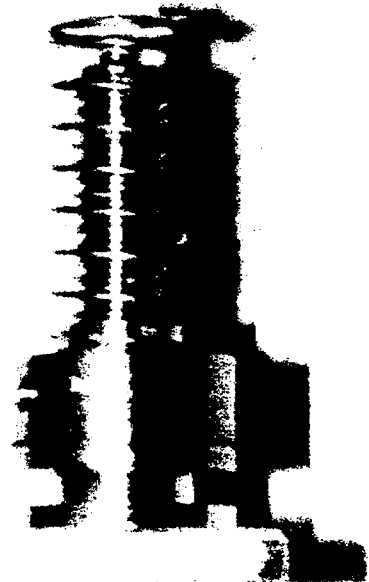

Fail

Place Spring onto stem of 9901-09, with 9901-10 fully screwed down until it touches the base of 9901-09.

The top of the stem should be almost touching the top of the spring..

If a gap can be seen between the top of the stem and the top of the spring the spring Fails.

Springs are not batch tested. Every Spring must be tested prior to being inserted in to 9901-40.



RS 28343

RS 28344

**Viamed Limited, 15 Station Road, Cross Hills,  
Keighley, West Yorkshire BD20 7DT**

**Tel +44 (0)1535 634542/636757 Fax +44 (0)1535 635582**

**Email [info@viamed.co.uk](mailto:info@viamed.co.uk)**

**Registration No 12917565 in England**

TT 125