

VM3COP51.40 Resus Cabinet QA

- 1) Check Viamed logo label is located on the front of the cabinet and above the door. The label must be central and straight.
- 2) Check the 'Support door' label is located on the front of the cabinet door. This must be central, straight and at the top of the door.
- 3) Check the serial number label is located to the lower right-hand side of the cabinet as shown. This must be central and straight.



- 4) Check the Caution label is on the inside of the door. The must be straight and on the right-side corner.



- 5) Check the rails are in place.

The rails can be fitted in several positions to suit customer requirements. The standard positions are:

Top – short rail
One below middle – Long rail
Bottom – Long rail



- 6) Check the mattress is present and stored on top of the bed.



- 7) Check the three plastic storage bins are stored in the top right corner.



- 8) Ensure that the door closes with the bed in place.
- 9) Check both catches latch and sufficiently prevent the door from opening.

