

Label part number 2590100

**Microstim DB3 Supramaximal Nerve Stimulator**

Part number: 2510001

Serial number: H000xxxx



**Caution:**

This is a non-standard version unit and is easily identified as it has green switches; the standard version (part number 2510000) has red switches.

**It is essential that the user reads the supplementary instructions – Appendix I, page 20.**

1 February 2010

Label part number 2590101

**Appendix I**

Applies to DB3 Supramaximal Nerve Stimulators, part number **2510001**

Functionality of stimulation is as follows:

**T.O.F.** Latching switch – Continuous cyclic output, T.O.F. which repeats after a pause of 8 seconds - for a total period of 5 minutes.

**D.B.S.** Momentary switch - Functionality same as with standard DB3

**P.T.C.** Momentary switch - Functionality same as with standard DB3

**1 Hz** . Latching switch – Continuous output repeats for a total period of 5 minutes.

**NOTE:** With either of the revised T.O.F. and 1Hz functions the latching switches need to be reset to the middle position otherwise another function cannot be selected.

After use - if either of the T.O.F and 1 Hz functions are left in the latched (On) position there is still a small current drain, even after the five minute time-out period. This current consumption will drain the battery unless the switches are returned to the middle (Off) position.

1<sup>st</sup> February 2010