



E NO.	DA	REV	/ E	MBEF
I.	9-10-85			

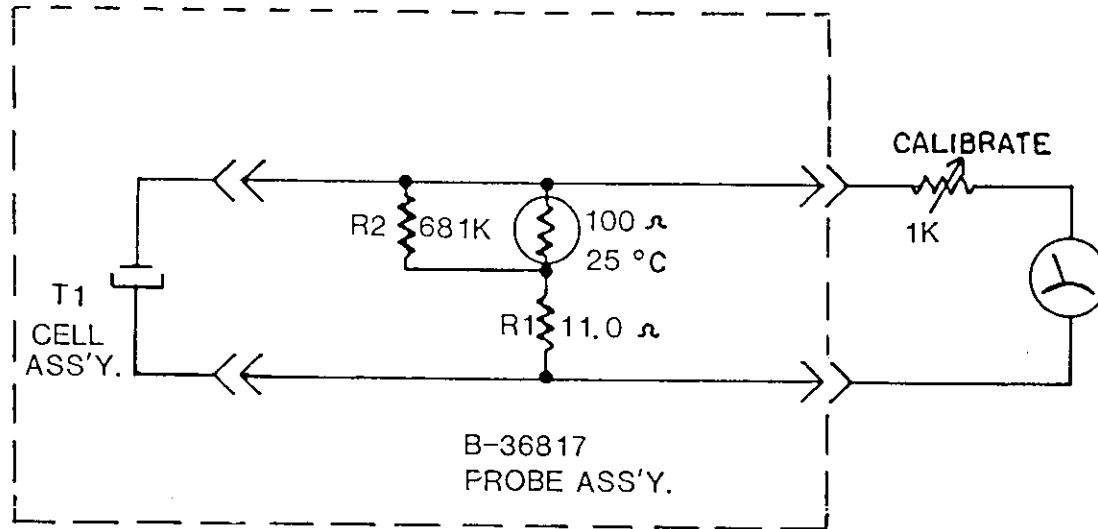


FOR T-6 CELL
PROBE ASS'Y IS B-38012

ITEM	PART NO.	REQ	DESCRIPTION		
TOLERANCE UNLESS OTHERWISE SPECIFIED		THIS DRAWING IS THE PROPERTY OF TELEDYNE ELECTRONIC DEVICES AND CONTAINS CONFIDENTIAL INFORMATION. IT IS NOT TO BE COPIED REPRODUCED OR USED WITHOUT WRITTEN PERMISSION			
FRAC - 1/64"		 TELEDYNE ELECTRONIC DEVICES CITY OF INDUSTRY CALIFORNIA 91748		SCALE _____	DATE 9-6-85
DEC - .005"				MAT'L _____	
ANGULAR - 1/2"		MDLTED 120 SCHEMATIC		FINISH _____	
F/C-34744				DR D.C. ENGR APP <i>SP</i>	
S.O.				A-38749	

ISSUE NO	DATE	REVISION	ECN NUMBER	APP
1	2-23-84			
2	8-23-84	REV'D, ADDED P1, R19, C1, AND CHANGE VALUE - S1 ON, R2, R4 PER ECO 84-0113M		
3	11-30-84	ECO 84-0149M		

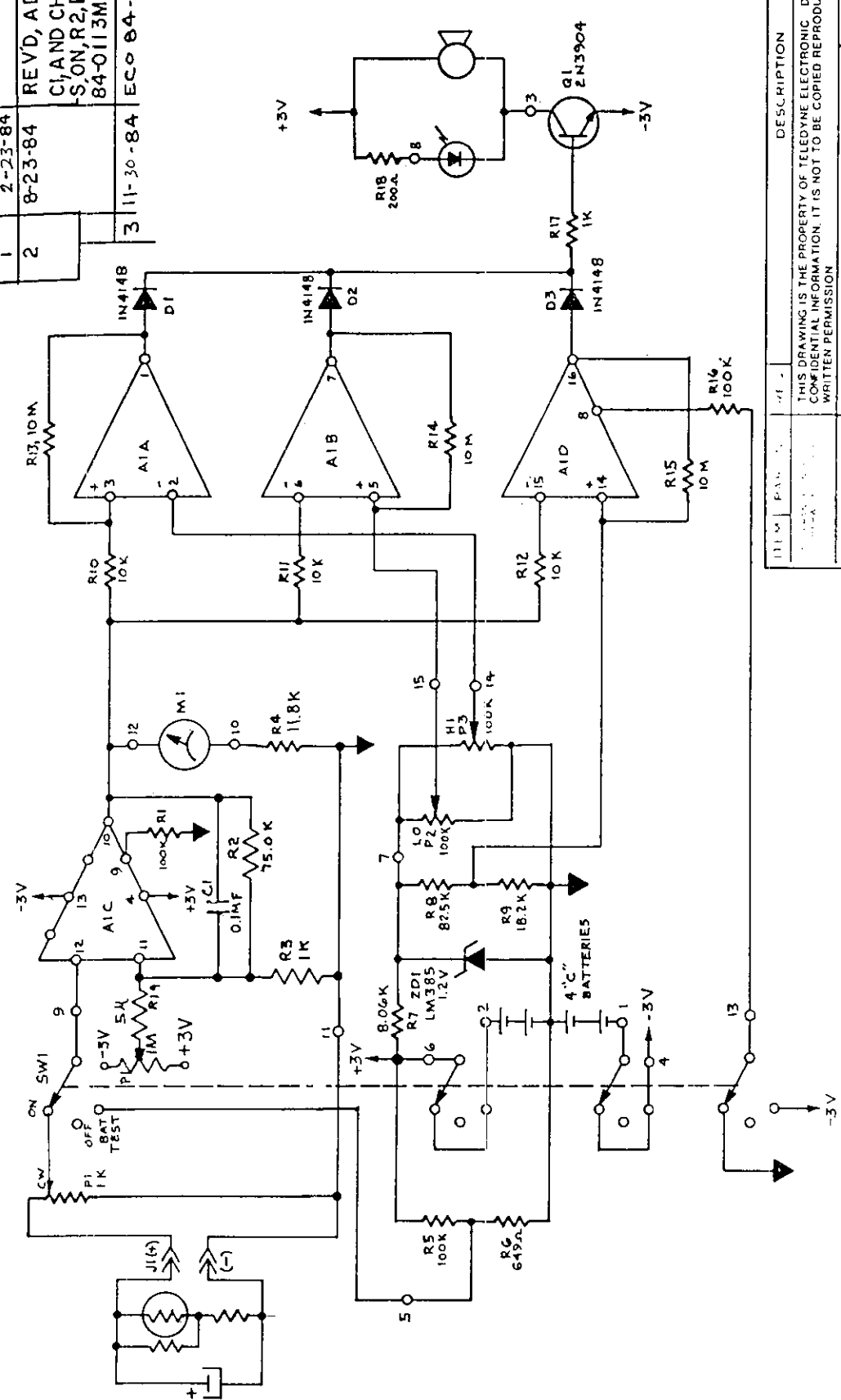



DESCRIPTION	
THIS DRAWING IS THE PROPERTY OF TELEDYNE ELECTRONIC DEVICES AND CONTAINS CONFIDENTIAL INFORMATION. IT IS NOT TO BE COPIED, REPRODUCED OR USED WITHOUT WRITTEN PERMISSION	
TELEDYNE ELECTRONIC DEVICES CITY OF INDUSTRY CALIFORNIA 91748	SCALE ~ DATE 1-24-84
MDL. TED 140 OXYGEN ALARM SCHEMATIC	MATL ~ FINISH ~
N/C-34620	DR NANCY ENGR APP JNZ
50	B34518

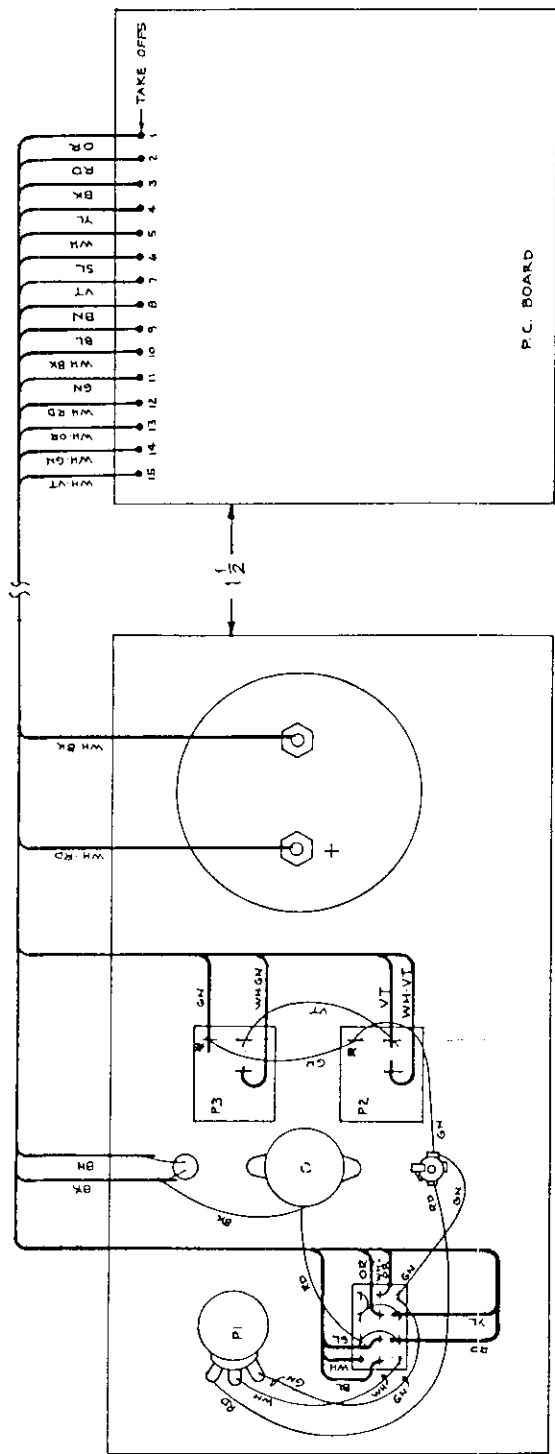

PCB T/O N°	COLOR	LENGH
1	OR	
2	RD	
3	BK	
4	YL	
5	WH	
6	SL	
7	VT	
8	BN	
9	BL	
10	WH-BK	
11	GN	
12	WH-RD	
13	WH-OR	
14	WH-GN	
15	WH-VT	

- NOTE:
- 1) ALL WIRES TO BE 22 AWG. WIRES BETWEEN P.C.B.D. & PANEL.
NOTE: WIRING SHOWN IN FINE LINES IS EXISTING FROM FRONT PANEL WIRING DIAGRAM. (B-34685)
 - 2) CONNECTIONS MARKED WITH (*) REQ'D TWO WIRES SOLDERED TO ONE TERMINAL (2 PLC'S)

DESCRIPTION		SCALE	DATE
TELEDYNE ELECTRONIC DEVICES		~	2-13-84
MDL. TED 140			
OXYGEN ANALYZER			
WIRING DIAGRAM			
F/C 34682			
C-34684			

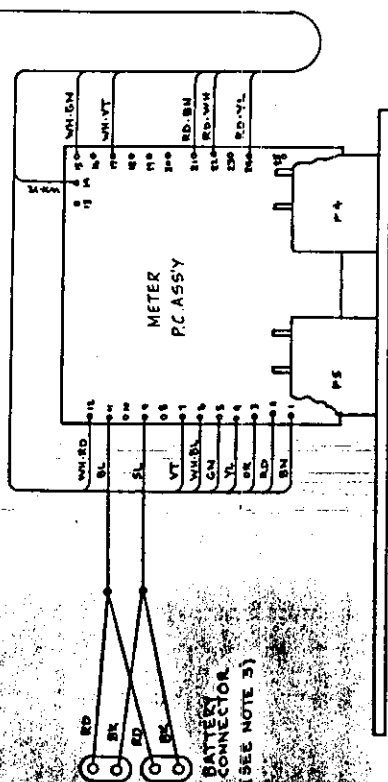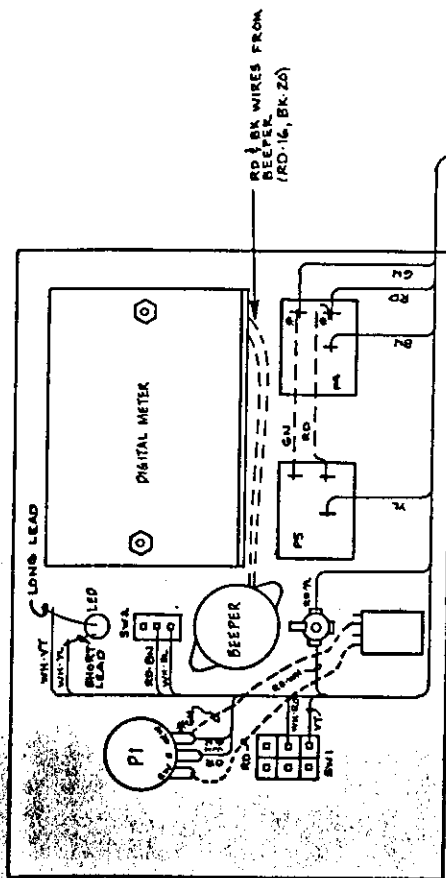

BOTTOM VIEW


NOTE:

- 1) ALL WIRES TO BE 22 AWG. WIRES BETWEEN P.C. BD & PANEL NOTE: WIRING SHOWN IN DOTTED LINES ARE EXISTING FROM FRONT PANEL WIRING DIAGRAM B-34771.
- 2) CONNECTIONS MARKED WITH (*) REQ'D TWO WIRES SOLDERED TO ONE TERMINAL. (3 PLACES)
- 3) ADD SPOT TIES TO FORM 'CABLE' TO BATTERIES, MAKE BL & SL WIRES LONG ENOUGH TO ALLOW SERVICE LOOP. (SEE DWG B-34771 FOR BATTERY CONNECTOR P/N)


ITEM	PART NO.	REV.	DESCRIPTION
1	TELEDYNE ANALYTICAL INSTRUMENTS		SCALE ~ DATE 2-23-84
2	ANALYTICAL INSTRUMENTS		DATE ~
3	CITY OF INDUSTRY CALIFORNIA 91748		FINISH ~
MDL TED 160 (DIGITAL)			DATE ~
WIRING DIAGRAM			DATE ~
P/C34767			DATE ~
S/C34769			DATE ~
C-34770			DATE ~



VIAMED
15 Station Road, Cross Hills,
Kesteven, West Yorkshire,
Tel. Cross Hills (0535) 34542

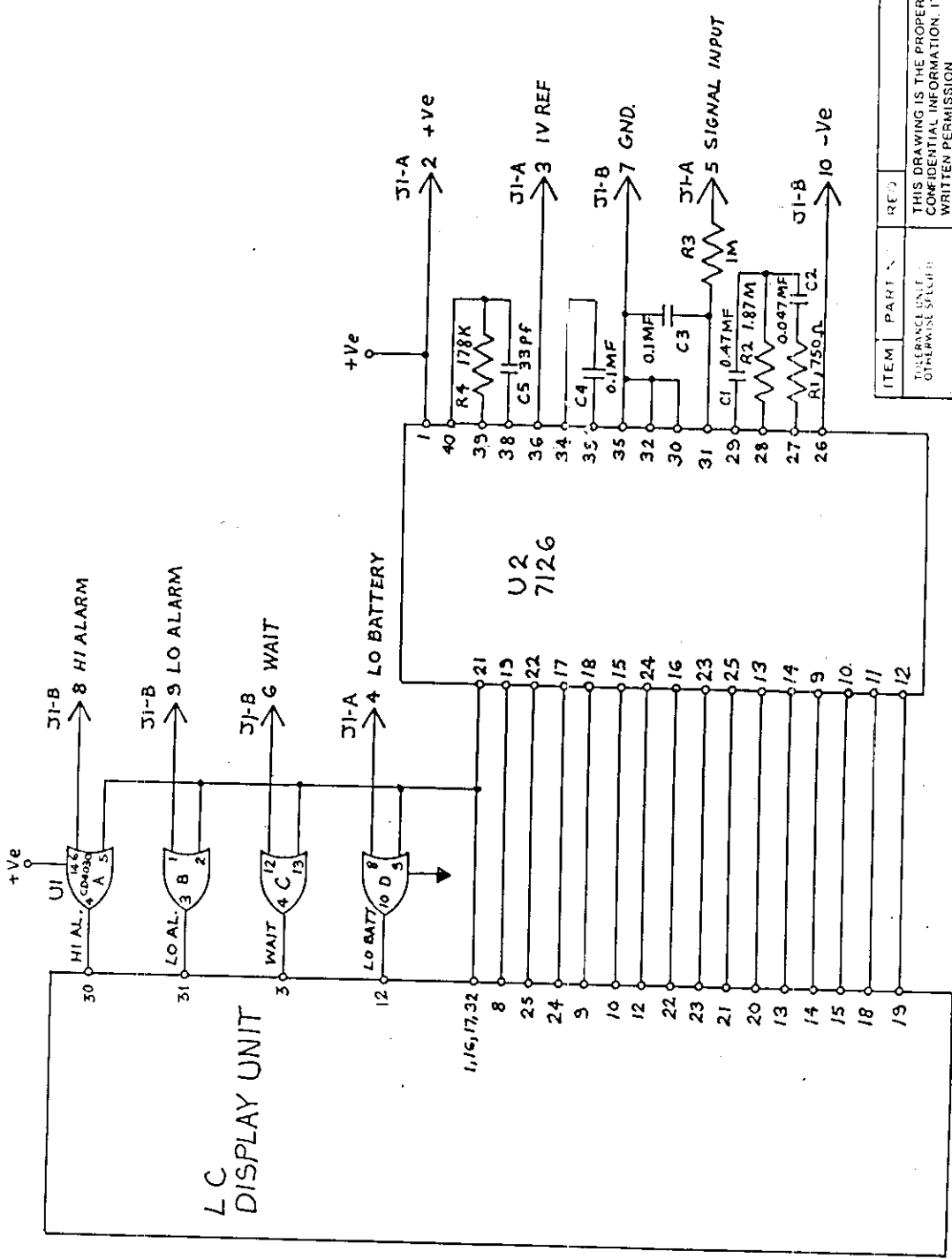

ITEM	PART	REV	DESCRIPTION
TOLERANCE UNLESS OTHERWISE SPECIFIED			THIS DRAWING IS THE PROPERTY OF TELEDYNE ELECTRONIC DEVICES AND CONTAINS CONFIDENTIAL INFORMATION. IT IS NOT TO BE COPIED, REPRODUCED OR USED WITHOUT WRITTEN PERMISSION
FRACTION	1/64		TELEDYNE ELECTRONIC DEVICES CITY OF INDUSTRY CALIFORNIA 91748
DEC	005		MDL. TED 100
ANGULAR	1/2		MEDICAL O2 ANALYZER
			LCD P.C. BD. SCHEMATIC
			SCH.
			DATE 3-22-84
			SCALE MATL
			FINISH
DR	VEg	ENGR	APP
			B 30662

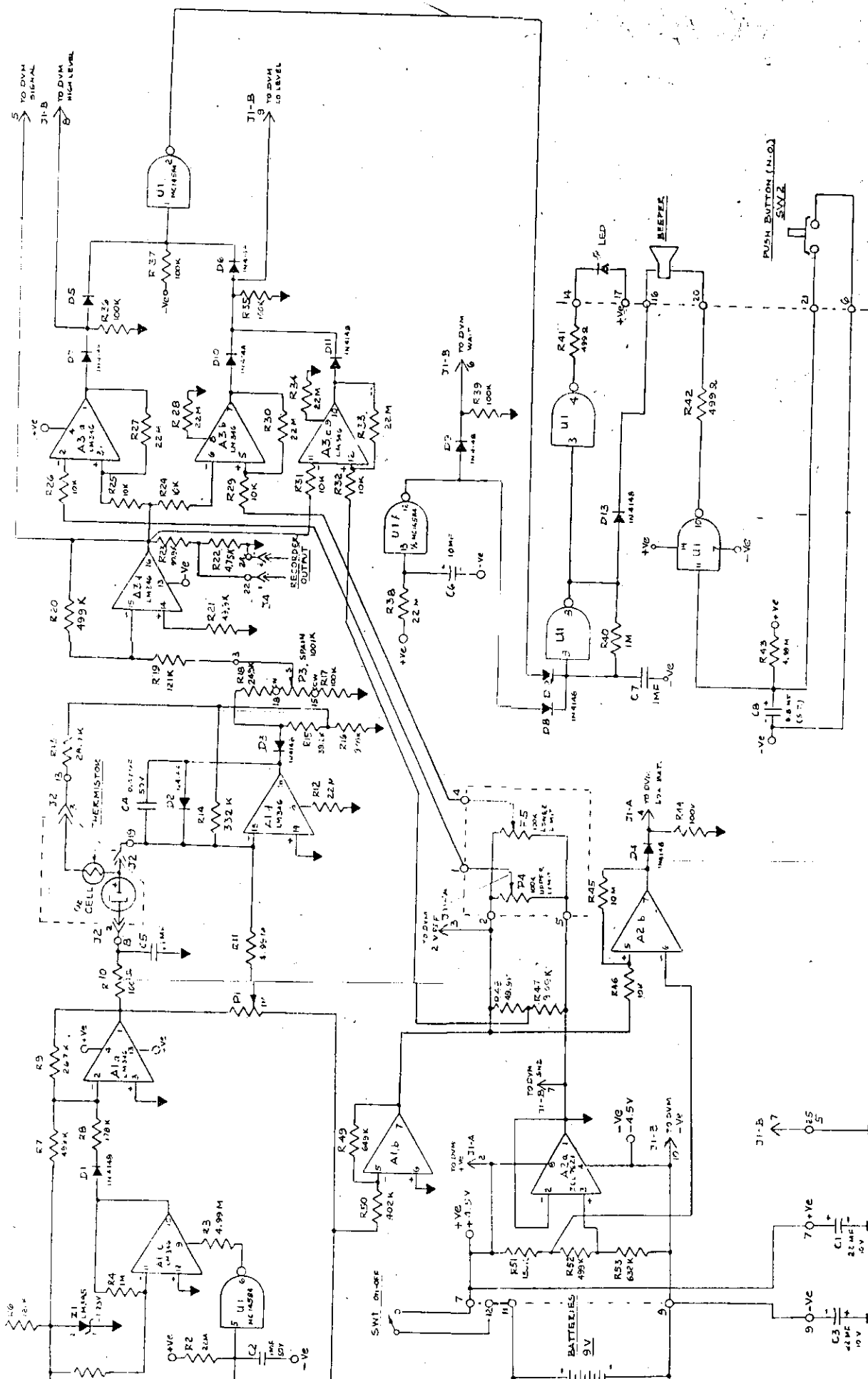


ITEM	PART NO.	DESCRIPTION
1	SEE B-30662 FOR L.C.D. P.C. BOARD SCHEMATIC	THIS DRAWING IS THE PROPERTY OF TELEDYNE ELECTRONIC DEVICES AND CONTAINS CONFIDENTIAL INFORMATION. IT IS NOT TO BE COPIED OR REPRODUCED OR USED WITHOUT WRITTEN PERMISSION
2	SEE C-30169 MAIN P.C. BOARD ASSEMBLY	
3	SEE C-31103 WIRING DIAGRAM	
4	TELEDYNE ELECTRONIC DEVICES	CITY OF INDUSTRY, CALIFORNIA 91748
5	MEDICAL ANALYZER	
6	MODEL 100	
7	MAIN P.C. BOARD SCHEMATIC	
8		
9		
10		
11		
12		
13		
14		
15		
16		
17		
18		
19		
20		
21		
22		
23		
24		
25		
26		
27		
28		
29		
30		
31		
32		
33		
34		
35		
36		
37		
38		
39		
40		
41		
42		
43		
44		
45		
46		
47		
48		
49		
50		
51		


NOTES:

- SEE B-30662 FOR L.C.D. P.C. BOARD SCHEMATIC
- SEE C-30169 MAIN P.C. BOARD ASSEMBLY
- SEE C-31103 WIRING DIAGRAM



REAR VIEW OF FRONT PANEL

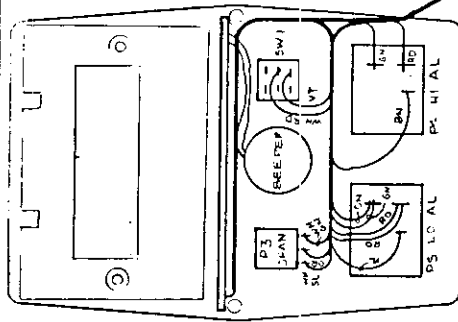


INSIDE VIEW OF CASE

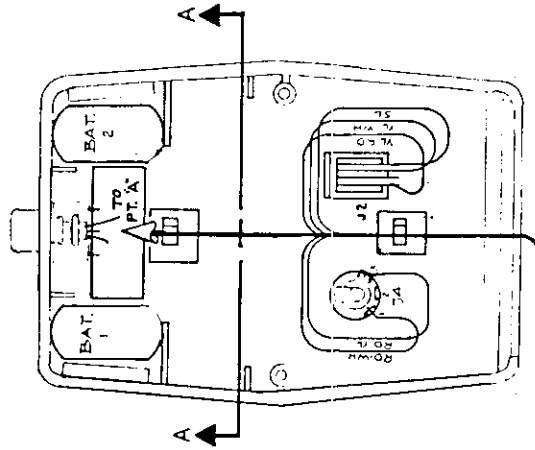

VIEW A-A

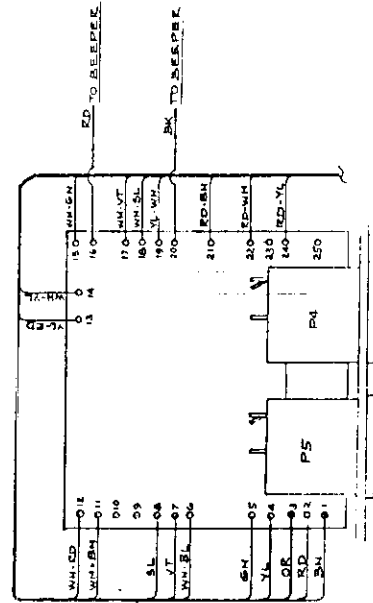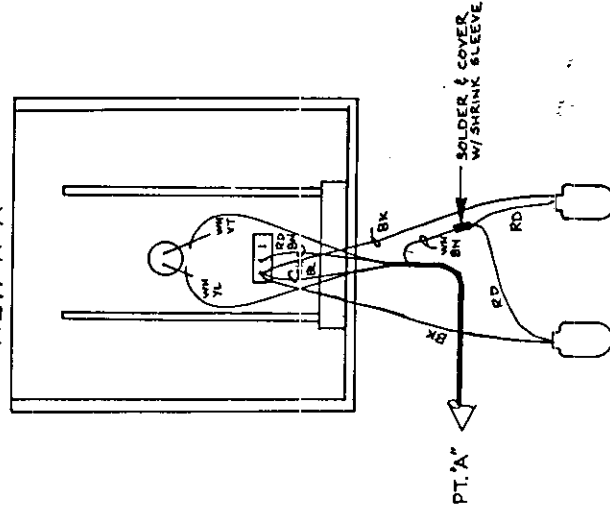


BOTTOM VIEW

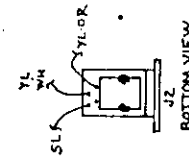


ITEM	PART NO	REQ	DESCRIPTION
1	30661	1	TELEDYNE ELECTRONIC DEVICES
2	30662	1	MEDICAL OR ANALYZER
3	30663	1	MDL-ED 100
4	30664	1	WIRING DIAGRAM
5	30665	1	SCALE
6	30666	1	NOTED
7	30667	1	DATE 3-7-63
8	30668	1	SCALE
9	30669	1	NOTED
10	30670	1	DATE 3-7-63
11	30671	1	SCALE
12	30672	1	NOTED
13	30673	1	DATE 3-7-63
14	30674	1	SCALE
15	30675	1	NOTED
16	30676	1	DATE 3-7-63
17	30677	1	SCALE
18	30678	1	NOTED
19	30679	1	DATE 3-7-63
20	30680	1	SCALE
21	30681	1	NOTED
22	30682	1	DATE 3-7-63
23	30683	1	SCALE
24	30684	1	NOTED
25	30685	1	DATE 3-7-63
26	30686	1	SCALE
27	30687	1	NOTED
28	30688	1	DATE 3-7-63
29	30689	1	SCALE
30	30690	1	NOTED
31	30691	1	DATE 3-7-63
32	30692	1	SCALE
33	30693	1	NOTED
34	30694	1	DATE 3-7-63
35	30695	1	SCALE
36	30696	1	NOTED
37	30697	1	DATE 3-7-63
38	30698	1	SCALE
39	30699	1	NOTED
40	30700	1	DATE 3-7-63
41	30701	1	SCALE
42	30702	1	NOTED
43	30703	1	DATE 3-7-63
44	30704	1	SCALE
45	30705	1	NOTED
46	30706	1	DATE 3-7-63
47	30707	1	SCALE
48	30708	1	NOTED
49	30709	1	DATE 3-7-63
50	30710	1	SCALE
51	30711	1	NOTED
52	30712	1	DATE 3-7-63
53	30713	1	SCALE
54	30714	1	NOTED
55	30715	1	DATE 3-7-63
56	30716	1	SCALE
57	30717	1	NOTED
58	30718	1	DATE 3-7-63
59	30719	1	SCALE
60	30720	1	NOTED
61	30721	1	DATE 3-7-63
62	30722	1	SCALE
63	30723	1	NOTED
64	30724	1	DATE 3-7-63
65	30725	1	SCALE
66	30726	1	NOTED
67	30727	1	DATE 3-7-63
68	30728	1	SCALE
69	30729	1	NOTED
70	30730	1	DATE 3-7-63
71	30731	1	SCALE
72	30732	1	NOTED
73	30733	1	DATE 3-7-63
74	30734	1	SCALE
75	30735	1	NOTED
76	30736	1	DATE 3-7-63
77	30737	1	SCALE
78	30738	1	NOTED
79	30739	1	DATE 3-7-63
80	30740	1	SCALE
81	30741	1	NOTED
82	30742	1	DATE 3-7-63
83	30743	1	SCALE
84	30744	1	NOTED
85	30745	1	DATE 3-7-63
86	30746	1	SCALE
87	30747	1	NOTED
88	30748	1	DATE 3-7-63
89	30749	1	SCALE
90	30750	1	NOTED
91	30751	1	DATE 3-7-63
92	30752	1	SCALE
93	30753	1	NOTED
94	30754	1	DATE 3-7-63
95	30755	1	SCALE
96	30756	1	NOTED
97	30757	1	DATE 3-7-63
98	30758	1	SCALE
99	30759	1	NOTED
100	30760	1	DATE 3-7-63

C-31103

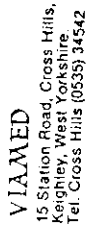

VIAMED
15 Station Road, Cross Hills,
Keighley, West Yorkshire.
Tel. Cross Hills (0535) 34542

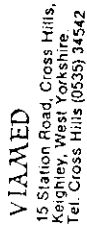

VIAMED
15 Station Road, Cross Hills,
Keighley, West Yorkshire.
Tel. Cross Hills (0535) 34542

VIAMED
15 Station Road, Cross Hills,
Keighley, West Yorkshire.
Tel. Cross Hills (0535) 34542

VIAMED
15 Station Road, Cross Hills,
Keighley, West Yorkshire.
Tel. Cross Hills (0535) 34542

VIAMED
15 Station Road, Cross Hills,
Keighley, West Yorkshire.
Tel. Cross Hills (0535) 34542