

VM3COP43.01 Maxtec Handi+ QA Procedure

- 1) Unbox the Maxtec Handi+ analyser (PN:0111250).



- 2) Check the box contains all the items listed:

Maxtec Handi+ analyser
Operating manual
'T' piece adaptor
Flow diverter
Barbed adaptor
Clear tubing
Lanyard



- 3) Peel back the protective film from the Oxygen sensor. Allow the sensor to be exposed to air for at least 30 minutes before testing.



- 4) Activate the analyser by pressing the power button on the front face.



- 5) Press the ▼ calibration button until CAL is displayed.



- 6) The analyser should display 20.9 when tested in air.



- 7) Recover the Oxygen sensor with the protective film.
- 8) Repack the analyser and peripheral items as they were originally packaged.
- 9) Enter the QA results into the Intrastats system.