

VM3COP43.04 Maxtec MAXO2+AE QA Procedure

- 1) Unbox the Maxtec MAXO2+AE analyser (PN:0111261).



- 2) Check the box contains all the items listed:


Maxtec MAXO2+AE analyser
Operating manual
'T' piece adaptor
Flow diverter
Barbed adaptor
Dovetail bracket and fixing screw
Lanyard




- 3) Peel back the protective film from the Oxygen sensor. Allow the sensor to be exposed to air for at least 30 minutes before testing.
- 4) Connect the MAX250 Oxygen sensor to the analyser cable.



- 5) Activate the analyser by pressing the power button on the front face.

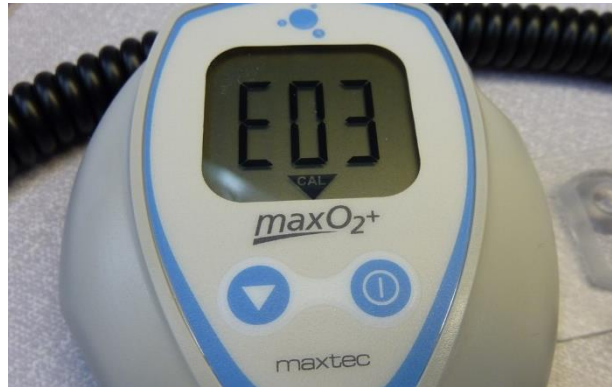

- 6) Press the  calibration button until CAL is displayed.



- 7) The analyser should display 20.9 when tested in air.



- 8) Recover the Oxygen sensor with the protective film.
- 9) Repack the analyser and peripheral items as they were originally packaged.
- 10) Enter the QA results into the Intrastats system.

