

COMPANY OPERATING PROCEDURES

0018790

Artema


P879RA

VM3/COP/32.24

Date: 18-Dec-01











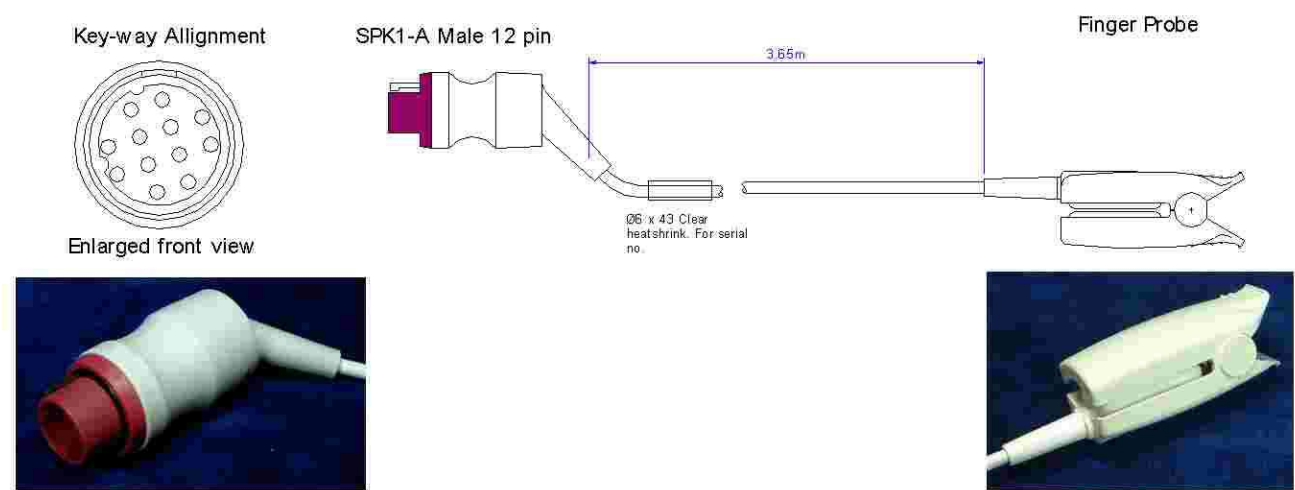





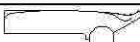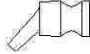


Revision date: 17-May-04

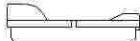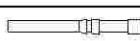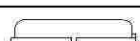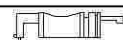Issue: 3

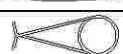

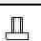Equipment required: Soldering iron (0060120), solder (0050012), Wire stripper (0060030), Flush Cutter (0060010), Snipe nose pliers (0060021), 'helping hand' (0060145), Heat gun (0060100).

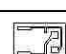Parts list: Kit and parts required. (Continued over page)

SPK1-A male 12-pin Side			Finger Probe Side		
Qty	Description	Part No.	Qty	Description	Part No.
1	SPK1-A male 10-pin kit	0010707	1	 Top Shell (x25)	0010110
(1)	 Rubber housing	kit	2	 Pad Support (white) (x50)	0010160
(1)	 Collar	kit	1	 Top Pad (white)(x25)	0010130
(5)	 Pins	kit	1	 Bottom Pad (white)(x25)	0010131
(1)	 Upper casing	kit	1	 Bottom Shell (x25)	0010111
(1)	 Cable clamp	kit	1	 Spring (x25)	0010140
(1)	 Pin housing	kit	2	 Button (White) (x50)	0010180
(2)	 Screws	kit	1	 Detector	0030901
(1)	 Lower casing	kit	1	 LED/I.R.	0030950
(1)	 Burgandy Collar	kit	1	 Strain Relief	0010150
1	Ø6 x 43mm Clear heat shrink	0032331			

COMPANY OPERATING PROCEDURES

0018790

Artema

P879RA

VM3/COP/32.24

Date: 18-Dec-01


Revision date: 17-May-04

Issue: 3


1	Ø6 x 25mm heat shrink	0032321			
1	Ø1.6 x 20mm heat shrink	0032310			
1	<div><div></div><div>R1</div></div> 1.33 kΩ Resistor	0032040			

ASSEMBLY OPERATIONS

1. Pre Heat soldering iron temperature to 240°c.
2. Collect all required parts and equipment listed above.
3. Cut a 3.75 metre length of standard 6-core cable. (Details shown below).

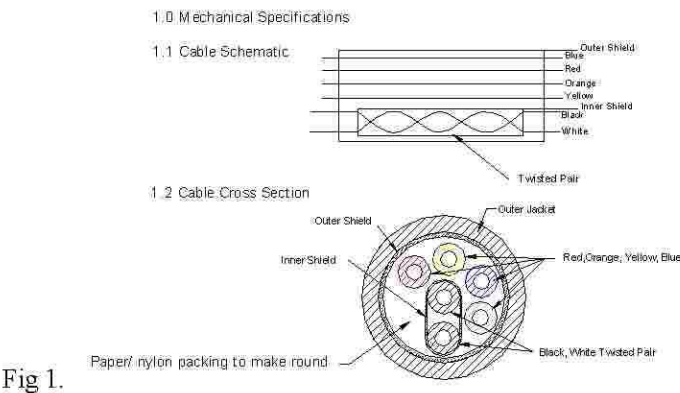

SPK1-A male 12-pin Side :

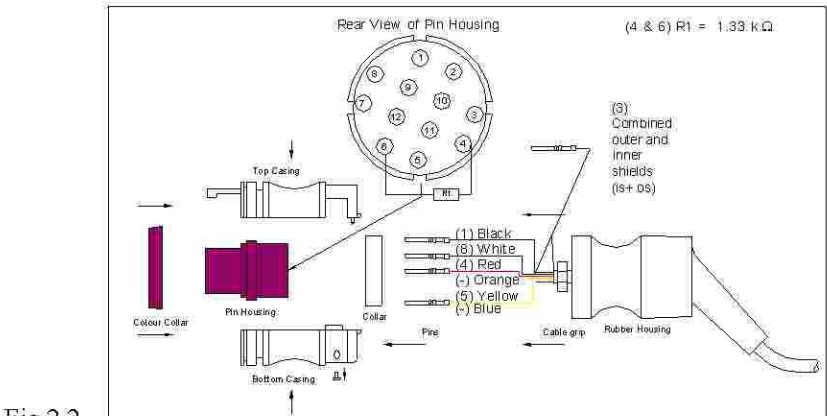

1. Feed Ø6 x 43mm (clear) heat shrink, rubber housing, collar, cable grip and Ø6 x 25mm (black) heat shrink over the end of the cable.
2. Strip 25mm off outer jacket of cable to reveal coloured wires, outer shield, and nylon/paper wire packing.
3. Cut all packing, orange and blue wire to the base. Trim resistor legs to 4mm and 15mm.

COMPANY OPERATING PROCEDURES

0018790

Artema

P879RA

VM3/COP/32.24

Date: 18-Dec-01

Revision date: 17-May-04

Issue: 3

4. Strip 25mm off inner jacket to reveal black and white wires and the inner shield.
5. Twist inner and outer shields together, and heat Ø6 x 25mm heat shrink over excess wire.
6. Trim ends of wires to the same length.
7. Strip jacket of every wire 2mm to reveal copper core, and solder resistor legs and wires to the rear of individual pins.
8. Insert all pins into correct locations.
9. Insert pin housing into the bottom casing, and screw cable clamp over cable.
10. Push top casing onto bottom casing, and push the collar over mating cases.
11. Push rubber housing over cases, and then the maroon collar over the front of the casing.

Finger Probe side:

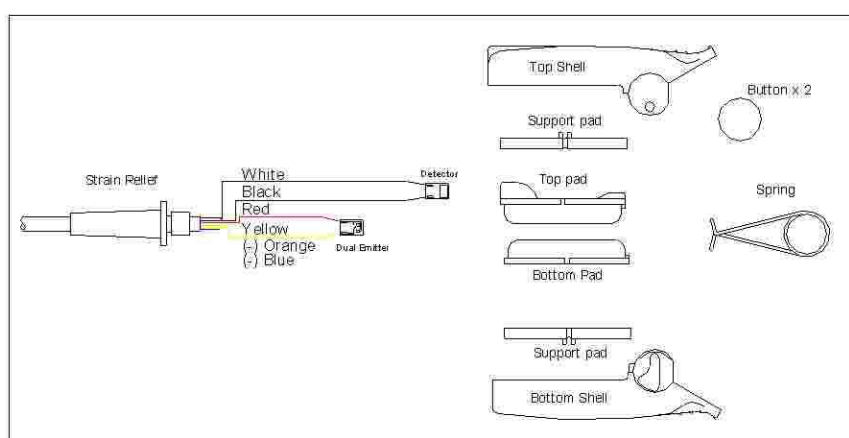


Fig 2.3

1. Apply loctite primer to the cable surface, and slide on the strain relief.
2. Apply a small amount of superglue on to the surface of the cable and push the strain relief over the glue to secure in place leaving approximately 80 mm of cable.
3. Strip outer jacket up to the strain relief and cut packing, outer shield, blue and orange wires.
4. Cut red and yellow wires to 15 mm, strip jackets off 2mm and apply a small amount of solder to the ends.
5. Strip 10 mm off inner jacket and cut off inner shield.
6. Strip jackets 2mm and apply small amount of solder.
7. Solder wires to the detector, LED/I.R as shown in fig 2.3
8. Place the assembly on the drying rack, and apply a small amount of clear silicon to the front of the detector and LED/I.R and mount into the pads (Led/IR in the top pad and Detector in the bottom pad) allowing the sensors to be seen and central, and scrape excess silicon. Then place the drying rack in the drying cabinet and leave to dry overnight.
9. Superglue loose cable to the pads and fill the rears with white silicon.
10. Glue pads onto the pad supports (prime first).
11. Place spring around pads and into place.
12. Clip upper and lower shells (apply a little super glue) into place and glue caps onto the sides.

COMPANY OPERATING PROCEDURES

0018790

Artema

P879RA

VM3/COP/32.24

Date: 18-Dec-01


Revision date: 17-May-04

Issue: 3


TESTING

1. Attach SPK-1-A side to the test cable and then to the test box connector marked 'A'.
2. Check display is showing correct characteristics as shown below. (At correct switch positions)

LED should read approx 1.8v

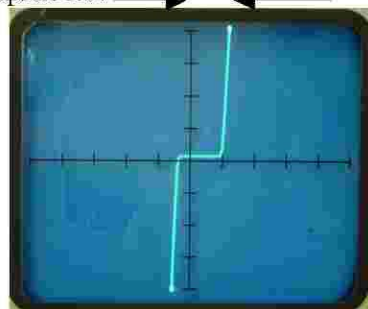

Position 1. IR, LED.

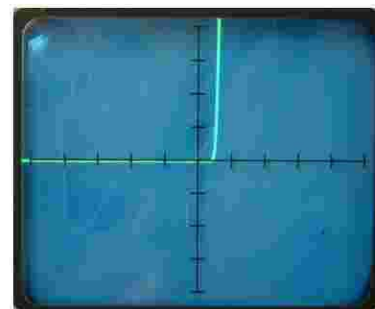


Position 4. Detector

3. If the LED signal is at the bottom then it is wired incorrectly.
4. 'Play' with wire at connections to see if any change in the display (i.e. flickering etc).
5. If there is any movement of signal, the cable must be taken apart and all connections checked and re-soldered. Then tested again until results are satisfactory.
6. Check the cable is of correct quality standard. (See VM/COP/30.11 for details).
7. Attach SPK-1-A side to a test cable then to Nellcor monitor, and the probe on to the finger to check SpO₂ level. (Ideal reading 95-100.)

Labelling

1. Labels: to be attached facing upwards as looking at the top of the probe.
 - 1 x CE Label
 - 1 x Viamed shell label on probe lower shell.
 - 1 x Serial no. Label
 - 1x Orange 'Do Not Throw Away' Label (correct one of two is dependant of country unit is being sold to).

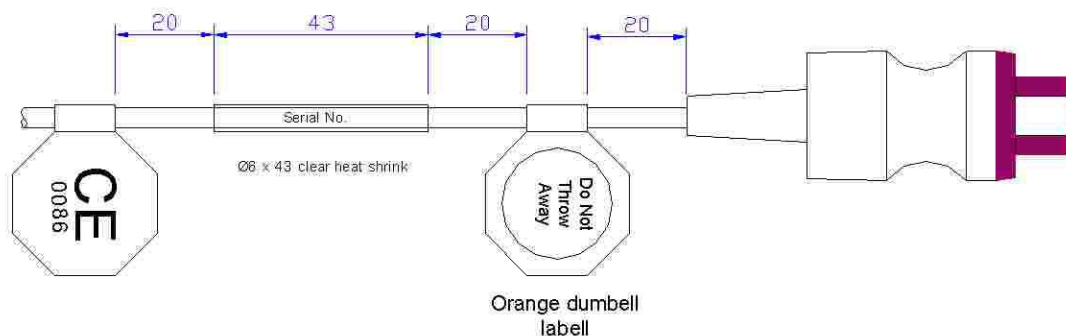

COMPANY OPERATING PROCEDURES

0018790

Artema

P879RA

VM3/COP/32.24

Date: 18-Dec-01


Revision date: 17-May-04

Issue: 3


Quality Assurance (QA)

1. Attach SPK-1-A side to the test cable and then to the test box connector marked 'A'.
2. Check display is showing correct characteristics as shown below. (At correct switch positions)

LED should read approx 1.8v

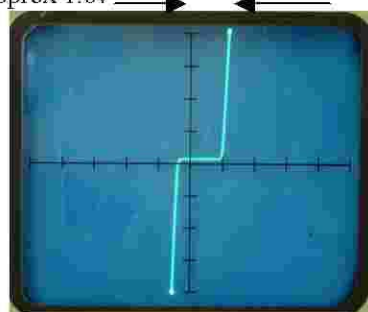

Position 1. IR, LED.

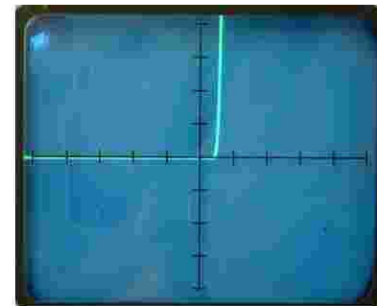

Position 4. Detector

3. If the LED signal is at the bottom then it is wired incorrectly.
4. 'Play' with wire at connections to see if any change in the display (i.e. flickering etc).
5. If there is any movement of signal, the cable must be taken apart and all connections checked and re-soldered. Then tested again until results are satisfactory.
6. Check the cable is of correct quality standard. (See VM/COP/30.11 for details).
7. Attach SPK-1-A side to a test cable then to Nellcor monitor, and the probe on to the finger to check SpO₂ level. (Ideal reading 95-100.)
8. Fill and sign attached paperwork.

Packaging

1. Visually check all labels are attached properly
2. Using a twist tie (bunny clip) wrap the cable and place in a small blue Viamed plastic box, ensuring the cable is inserted in a neat and tidy presentable manor.
3. Place a serial number sticker (supplied with the batch) on the front face of the box.
4. Place a packed and tested sticker (also containing initials of the individual who is packing) on the right hand side top left corner of the box. Do not close box.

Final QA

1. Final inspection. Visually ensure cable sit neatly within the box and is in a presentable state.
2. Boxes are ready to stock in stores.

COMPANY OPERATING PROCEDURES

0018790

Artema

P879RA

VM3/COP/32.24

Date: 18-Dec-01

Revision date: 17-May-04

Issue: 3