



1386340

LOT 2019-07

REF 0120399 T Adapter. Pack of 5

PN 46-006005

*Clean the T-piece by wiping with a soft cloth moistened in a detergent solution or immerse in a bath of the solution. Remove any deposits with a soft disposable cloth. The recommended cleaning agent is an alkaline/enzymatic cleaning Prolystica 2x (1). To disinfect wipe with or immerse in 70% isopropyl alcohol. For high level disinfection, we recommend CIDEX OPA (2).*

*Follow the manufacturers instructions for use.*

*1. Prolystica is a registered trademark of Steris Corporation.*

*2. CIDEX OPA is a registered trademark of Johnson and Johnson Corporation.*

*The T-piece has been tested to meet performance specifications after 50 cycles of cleaning/disinfection with the specified agents.*