

VM 2160 Test / QA procedure 0012160, 0012161, 0012162, 0012164

Attach the bar code to the correct box, at the end in the middle.





Take the back off the VM 2160 and inset the batteries.




Check the on/off switch works, the central button on the front of the unit, to switch on press and hold, when switching off press and hold as the unit counts down.

Attached the test probe (0014750) to the monitor.

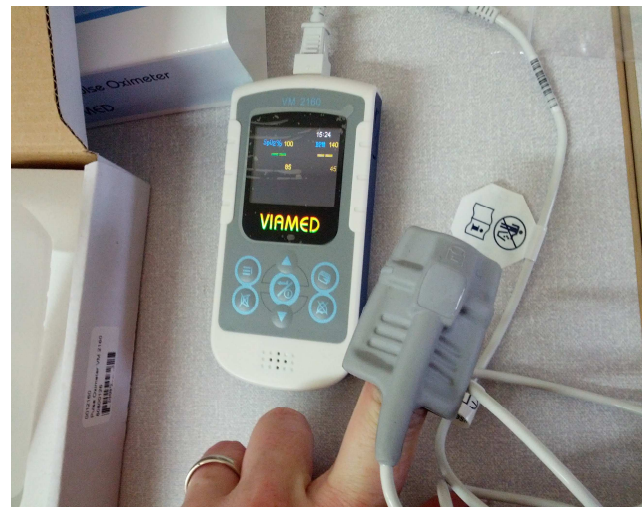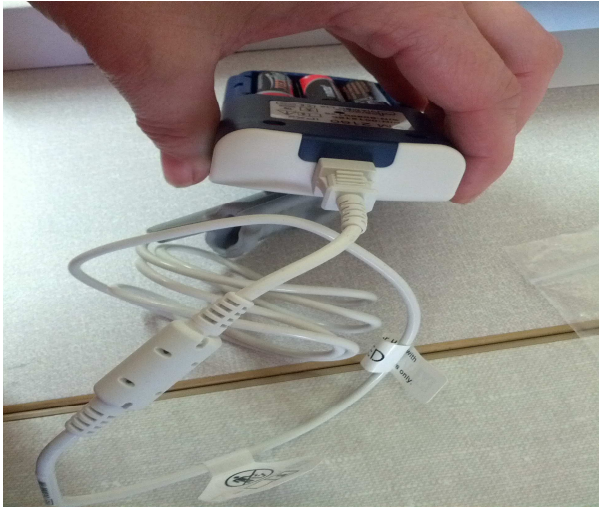

Put on your finger and wait for a stable reading to come up, check that the reading displayed is within normal range (i.e. 94 to 100%) for the SPO2, BPM (i.e. circa 72 bpm resting for adults) and

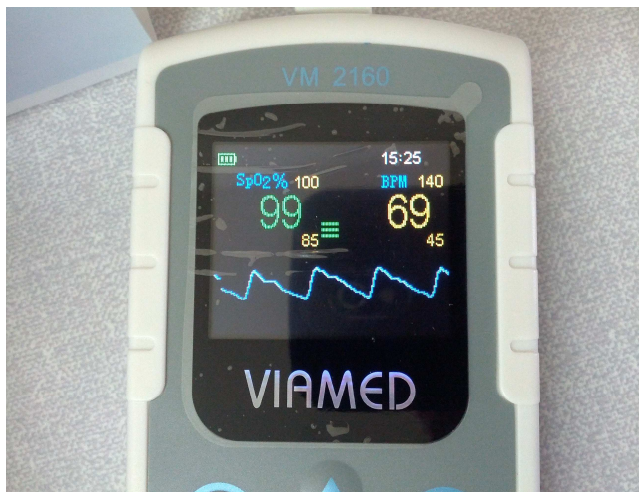


waveform, and that the display itself is working and doesn't have any missing segments.


Remove the probe.

press the Menu button on the front top left.


Setup – OK

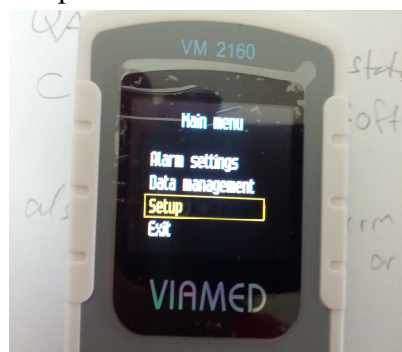


Date time – OK

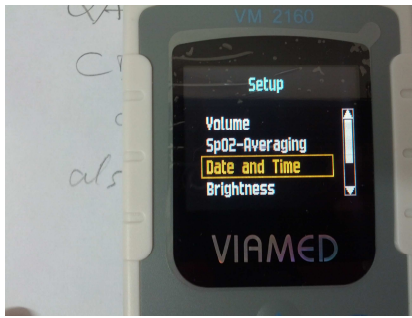

24 Hours – OK

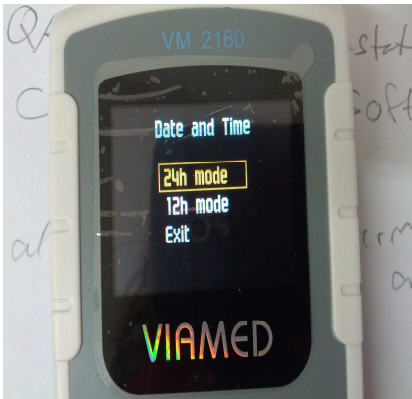


Set/Check Date – press twice for each digit – cycle through each one and correct the date and time if needed. (the time is usually and hour out)

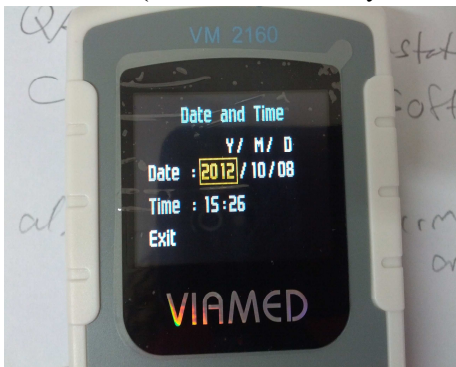

Then exit this menu.


Press Menu

Data Management – OK

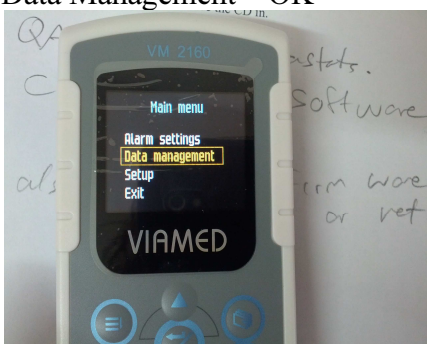


Delete all data – OK



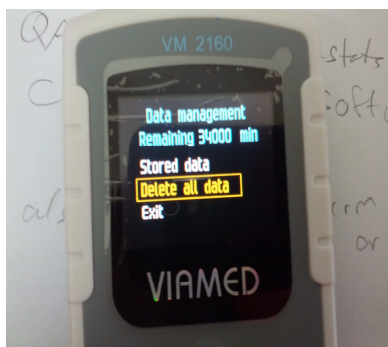


go to Yes – OK

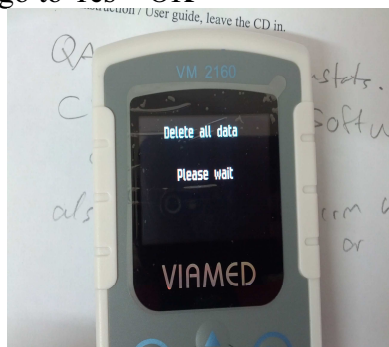

Wait until this finishes then exit.

At this point the time can be seen on the screen at the top right hand corner.

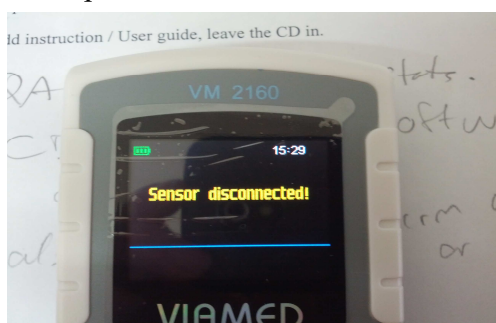

Now you can power off the unit.

Wipe unit carefully with cleaning wipe.

Add instruction / User guide, leave the CD in.



Fill in the QA form on Intrastats. Including the CD version, Software version (found on the unit) and the firmware type (Medical /Vet).

