
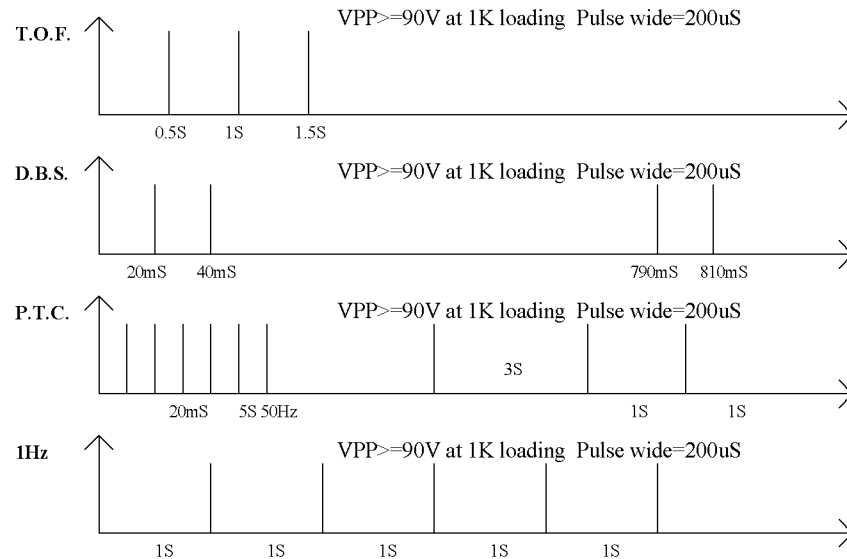


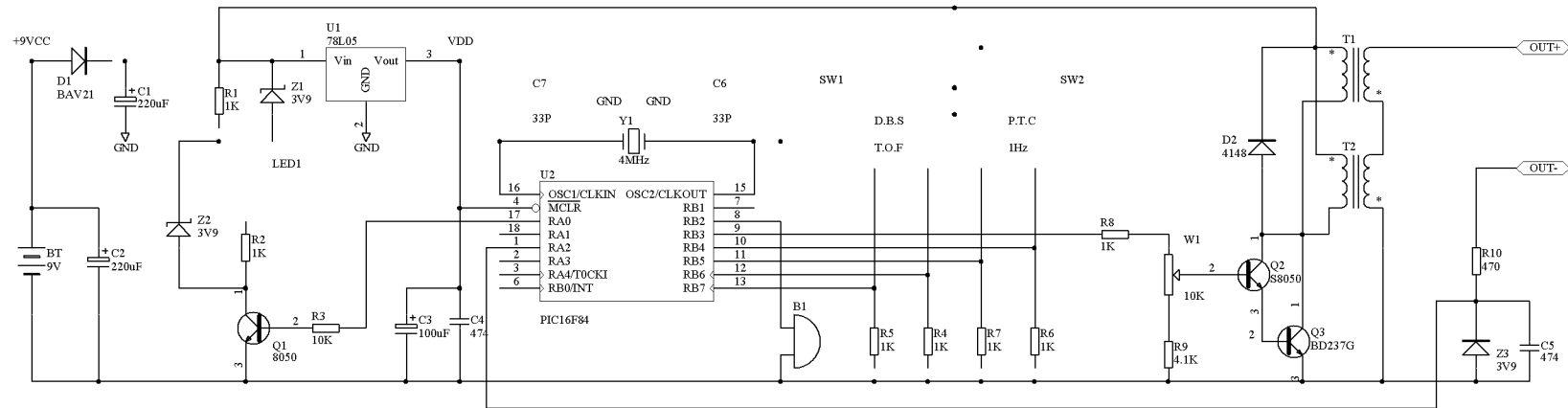

Microstim DB.

T.O.F. : 4 pulse at 2Hz

D.B.S. : 3 pulse at 50Hz, followed by a 0.75 second pause, then 2 pulses at 50Hz.

P.T.C. : 50Hz for 5 second, followed by a 3 second pause, then continuous 1Hz.

1 Hz : continuous 1 Hz.

Set output using intensity control, audio indicates patient current flowing; 100mA maximum, 200us pulses, constant current.

Replace battery when flash is deep red. uses 1 x MN1604, 9V alkaline battery