

L.S. Lamb

1987

1987

1987

THERMOCOT

TC400

Mk I

Appleyard and Sons  
Leeds, Yorkshire

---

MANUFACTURERS  
HANDBOOK

# THERMACOT™ TC400™ Mk 1

## MOBILE COT AND INTEGRAL RADIANT WARMER

APPLEYARD & SONS  
LEEDS, YORKSHIRE  
0113 250 2917

---

Manufacturers Handbook

1987  
V6.

(PAPERPORT)

**THERMACOT™**  
**TC400™ Mk I**

**MOBILE COT AND INTEGRAL RADIANT WARMER**

Appleyard and Sons  
Leeds, Yorkshire  
(0532) 502917

---

**MANUFACTURERS  
HANDBOOK**





**COPYRIGHT 1987 APPLEYARD AND SONS**

The information contained herein is the property of  
**APPLEYARD AND SONS** and must not be reproduced either in  
whole or part without prior written permission of the company









THERMACOT™  
TC400™ Mk I

MOBILE COT AND INTEGRAL RADIANT WARMER

Appleyard and Sons  
Leeds, Yorkshire  
(0532) 502917

---

MANUFACTURERS  
HANDBOOK

V1.1 / 1990



GSM  
Graphic Arts Limited  
Castlegarth Works  
Thirsk  
North Yorkshire  
YO7 1PS

Telephone: 01845 522206

UAL  
United Automation  
237 Liverpool Road  
Birkdale  
Southport  
PR8 4PJ

Telephone: 01704 65713

R. S. Components Limited  
P.O. Box 99  
Corby  
Northants  
NN17 9RS

|                     |               |                   |
|---------------------|---------------|-------------------|
| Telephone: National | 0181-360 8600 | fax 01536 201501  |
| North West          | 0161-477 8400 | fax 0161-480 5428 |

Order code: 2042 0002

Farnell Electronic Components Limited  
Canal Road  
Leeds  
LS12 2TU

Telephone: 0113 263 6311 fax 0113 263 3411

Order code: 709659

## 12.0 SERVICING AND SPARES

All Servicing of the TC400 THERMACOT™ is only to be carried out by suitably qualified personnel, instructed in the operation and maintenance of the TC400 THERMACOT™ and approved by a representative of the manufacturer or his agent.

Servicing information, assistance and spares may be obtained by contacting:-

VIAMED  
15 Station Road, Cross Hills,  
Keighley, West Yorkshire.  
Tel:(0535) 34542

## 12.0 SERVICING AND SPARES

All Servicing of the TC400 THERMACOT™ is only to be carried out by suitably qualified personnel, instructed in the operation and maintenance of the TC400 THERMACOT™ and approved by a representative of the manufacturer or his agent.

Servicing information, assistance and spares may be obtained by contacting:-

VIAMED  
15 Station Road, Cross Hills,  
Keighley, West Yorkshire.  
Tel:(0535) 34542



**TC400 Mk I**

**FREE STANDING RADIANT WARMER**



| <u>Parts List</u> |                     |         |
|-------------------|---------------------|---------|
| Drg. No           | Item                | Part No |
|                   | Radiant Heater unit | 2000    |
| 1                 | Cover               | 2001    |
| 2                 | Screw               | 2002    |
| 3                 | Mounting Plate      | 2003    |
| 4                 | Screw               | 2004    |
| 5                 | Washer              | 2005    |
| 6                 | Nut                 | 2006    |
| 7                 | Insulating spacer   | 2007    |
| 8                 | Power controller    | 2008    |
| 9                 | Knob                | 2009    |
| 10                | Switch              | 2010    |
| 13                | Nut                 | 2013    |
| 14                | Fuse Holder         | 2014    |
| 15                | Terminal Block      | 2015    |
| 16                | Radiant cover       | 2016    |
| 17                | Reflector           | 2017    |
| 18                | 400watt element     | 2018    |
| 19                | Element holder      | 2019    |
| 20                | Grille              | 2020    |

File =TC400F

**TC400 Mk I**

**WALL MOUNTED RADIANT WARMER**

**WITH**

**RESUSCITATION EQUIPMENT**