Single use flow sensor

Type H, 4310003

REF 3030131005

Manufactured for VIAMED LTD-UK

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EN Instructions for use

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bluepoint MEDICAL GmbH & CO. KG
An der Trave 15
23923 Selmsdorf, Germany
www.bluepoint-medical.com

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Single use flow sensor

Intended Use

Single patient use flow sensor used to measure the airflow in the breathing system of the patient for use with Hamilton ventilators. The sensor is compatible to:

- **Hamilton Flow sensor – PN 279331**

Caution:

- Reusing the sensor may affect patient's safety due to the possible contamination of the sensor.

Ventilator connection

- To connect the Flow Sensor to the ventilator attach the sensor tubes underneath the front panel of the ventilator. The blue tube must be attached to the "PROXIMAL" inlet of the ventilator.

Patient connection

- Insert the Flow Sensor between the Y piece of the patient circuit and the patient connection. The side of the Flow Sensor is equipped with the blue tube should face the patient. It is recommended that short flexible tubing is used between the sensor and the patient connection. Position the sensor with the tubing upright to prevent kinking and condensation.

Tubing clip

- The tubing clip is designed to secure the Flow Sensor tubes to the ventilation tubes. Place the wide opening of the clip around the inspiratory ventilation tube. Place the Flow Sensor tube in the narrow opening of the clip, leaving a loop of tubing between the clip and the Flow Sensor to accommodate the full range of patient movement.

Calibration

Caution:

- The Flow Sensor must be calibrated prior to patient use.
- Alignment, operation and sensor change must be carried out in accordance with the relevant device-specific instruction manual.

Disposal

The flow sensor must be disposed of as infectious waste.

Measuring range: 0 to 180 L/min

Accuracy: ± 20%, uncalibrated,
acc. DIN EN ISO 80601-2-13

Sensor connectors: 15M/15M, ISO 5356-1

Recommended operating conditions

- Temperature: 10°C to 40°C
- Relative humidity: 5% to 95%, non-condensing
- Atmospheric pressure: 700hPa to 1060hPa

Recommended storage and transport conditions

- Temperature: -20°C to 50°C
- Relative humidity: 5% to 95%, non-condensing
- Atmospheric pressure: 700hPa to 1060hPa

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