


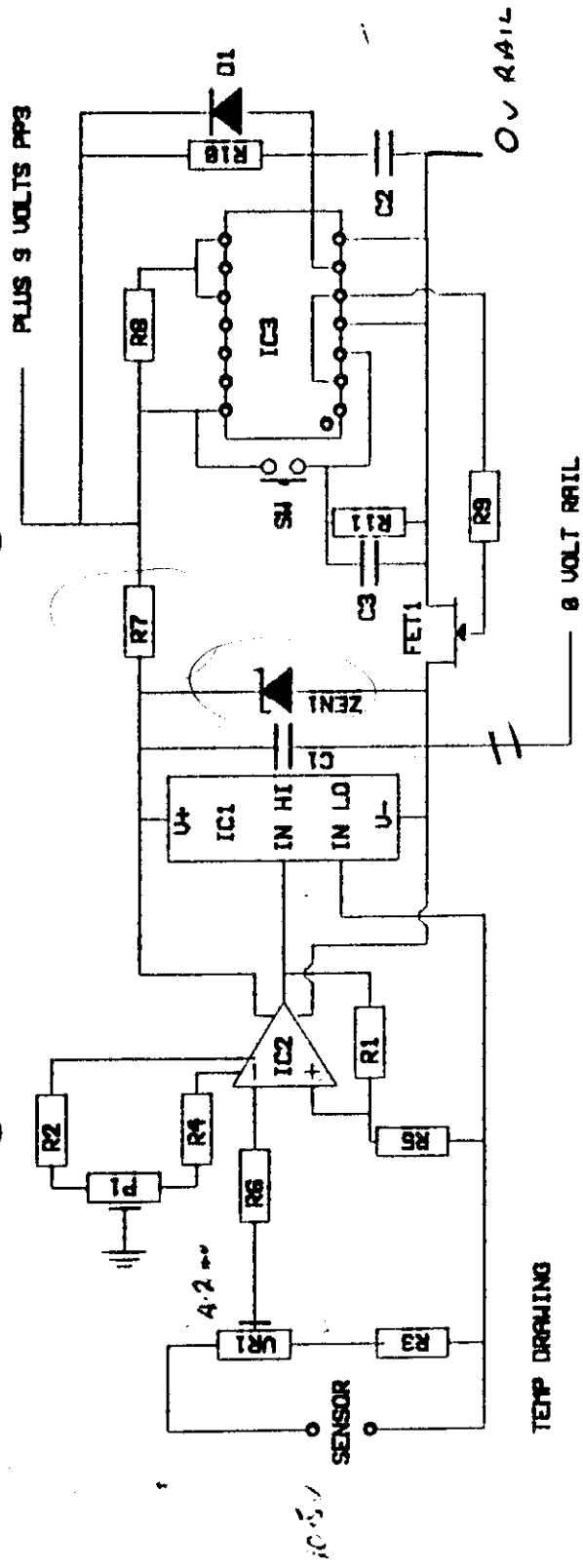

R1	22K	C1	10NF	IC1	OPM400
R2	4.7K	C2	33NF0	IC2	TL071CP
R3	8K2	C3	SEE NOTE	IC3	MC140138CP
R4	4.7K	UR1	10K		
R5	11K	P1	1K MULTI		
R6	10K	DISP	OPM400		
R7	100R	FET1	UN18KM		
R8	10K	ZEN1	8.2 VOLT		
R9	10K	D1	1N418		
R10	5M8				
R11	10K				

R7 - ZEN1 Pinned Jd

COMPANY VANDAGRAPH


DRAWING BY MELMAR PRODUCTS

MODEL DIVOX MK1 1994

DRAWING NUMBER 0801


DATE 1 JUL 1994

ENGINTEER M D T MELMAR

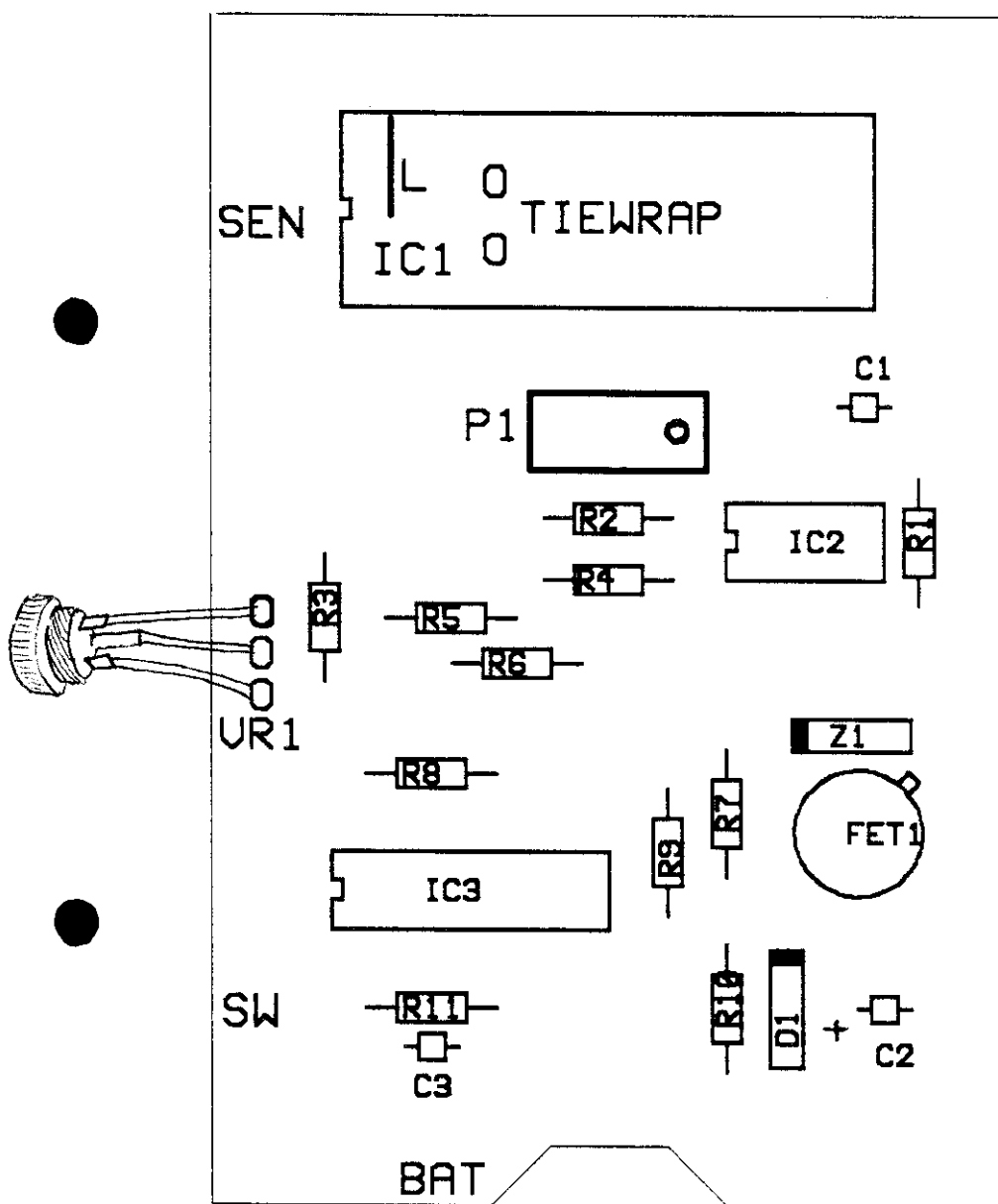




VIAMED

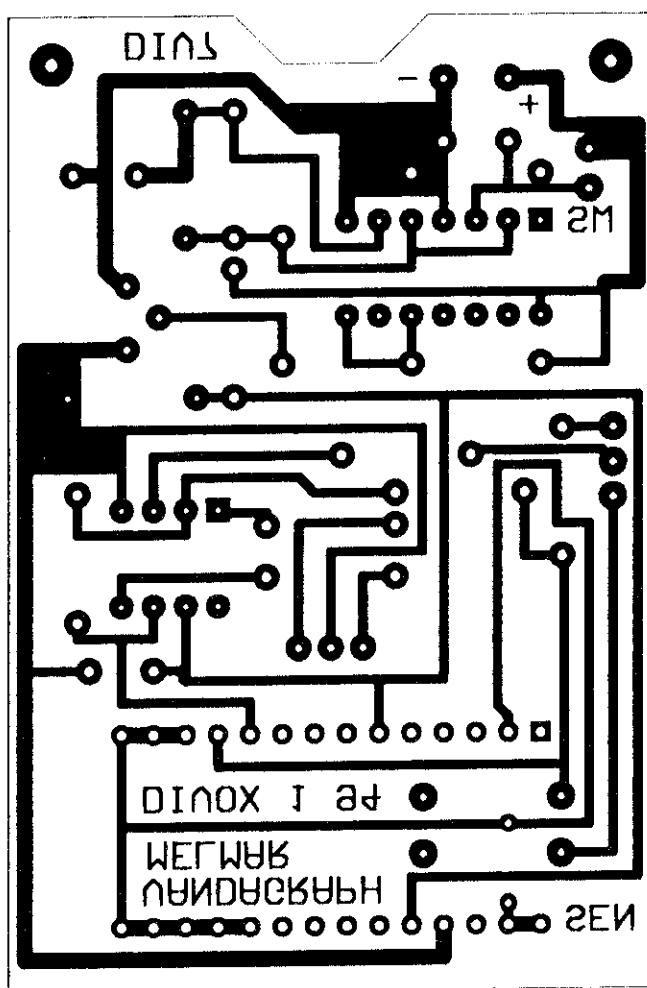

Viamed Limited, 15 Station Road, Cross Hills,
Keighley, West Yorkshire BD20 7DT
Tel: 01535 634542 / 636757 Fax: 01535 635582
Registration No. 1291765 in England

MARK

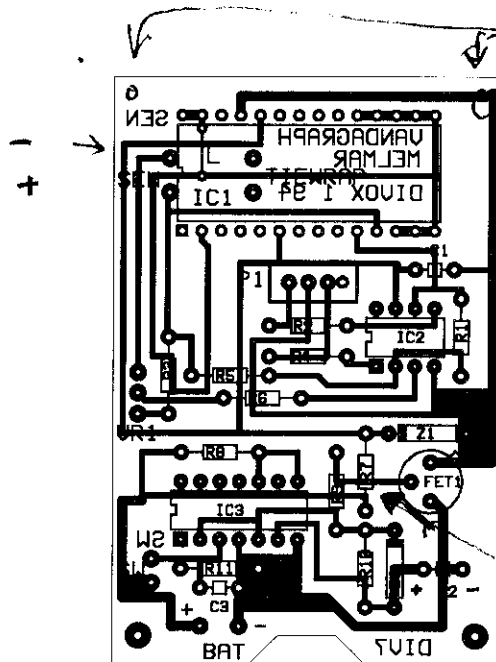

Holes to locate on case

Z1 No longer used
Component holes not required

R7 No longer used
Must Be replaced by
Taper

AUBREY OLDHAM.

1000/1

FAT 0706 818636




VIAMED



P and M. SERVICES
1/2/95

ATTN AUBREY OLDHAM

P.O. 7833/JSC
100- off DINOX PCB'S

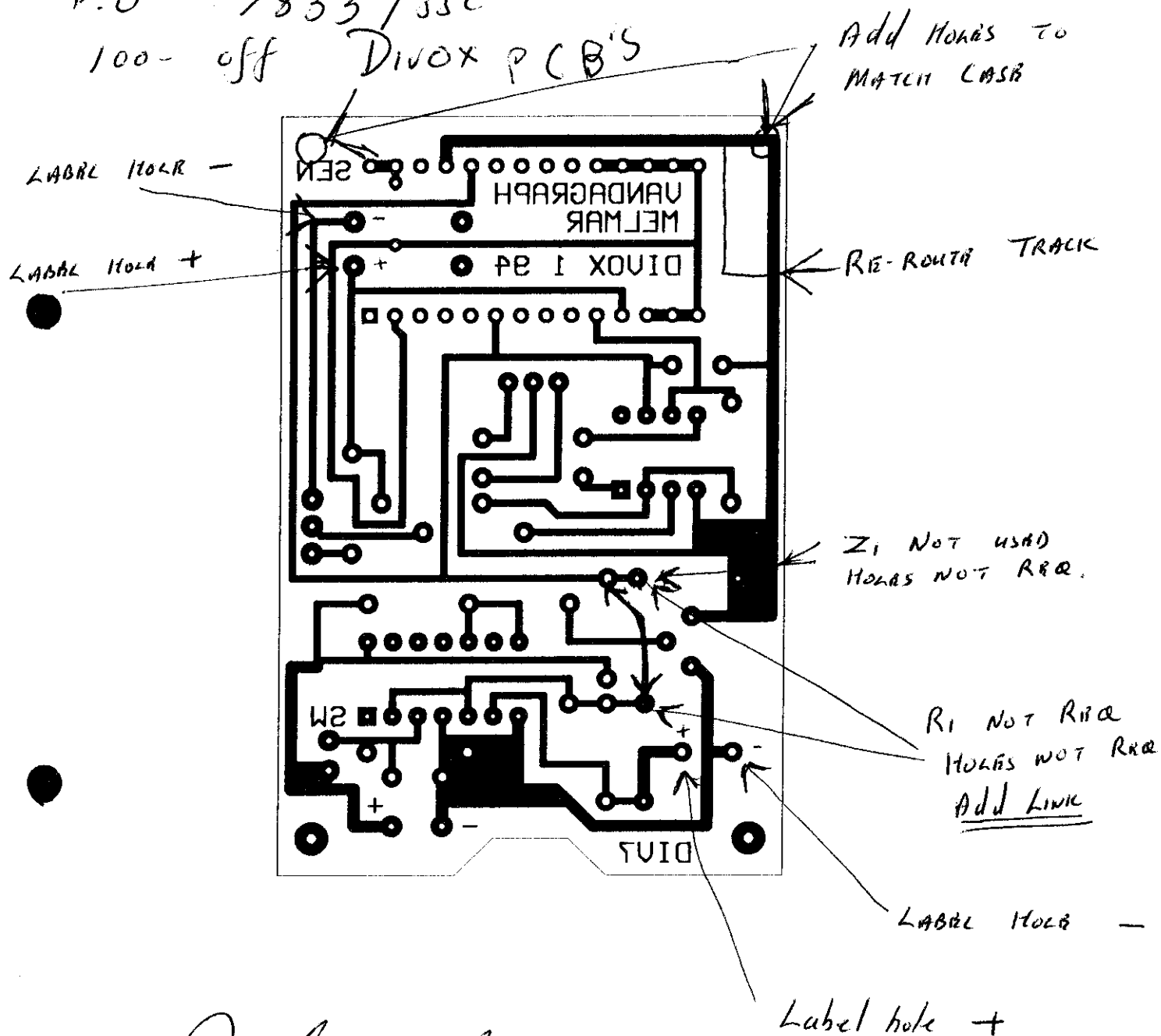

John S. Lamb



Viamed Limited, 15 Station Road, Cross Hills,
Keighley, West Yorkshire BD20 7DT
Tel: 01535 634542 / 636757 Fax: 01535 635582
Registration No. 1291765 in England

P. & M. SERVICES

(Rochdale) Ltd.




ESTABLISHED 1964

PRINTED CIRCUITS : PHOTO-MECHANICAL ENGINEERING

STONESWOOD MILL,
BACUP ROAD,
TODMORDEN,
LANCS., OL14 7HG
Tel: (0706) 815212
Fax: (0706) 818636
Modem: (0706) 819933

FAX MESSAGETO: *VIA-MED.*FROM: *P2M SERVICES / AUBREY..*FOR THE ATTENTION OF: *JOHN LAMB.*DATE: *2nd Feb 95*NO. OF PAGES: *1.*

MESSAGE: *COPIES OF THE MODIFIED NETWORKS. FOR DIVOX
P.C.B. FOR YOUR APPROVAL.*

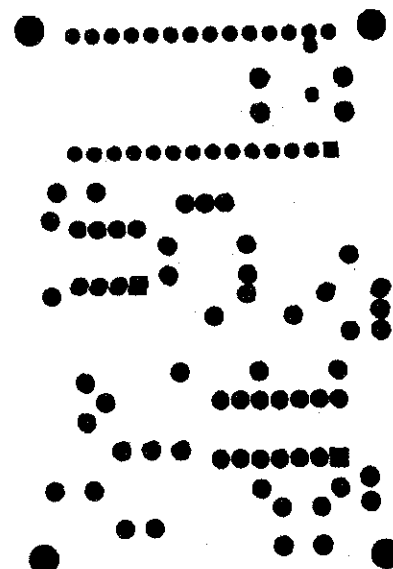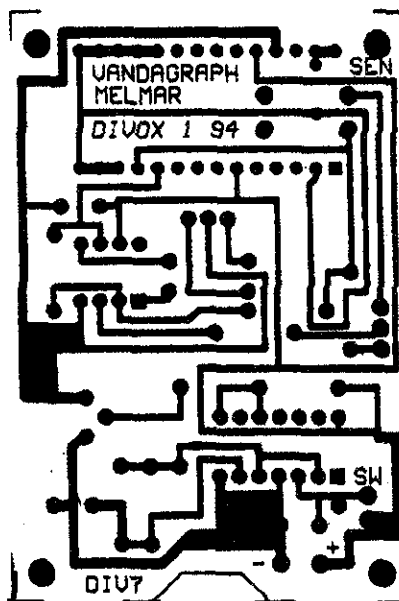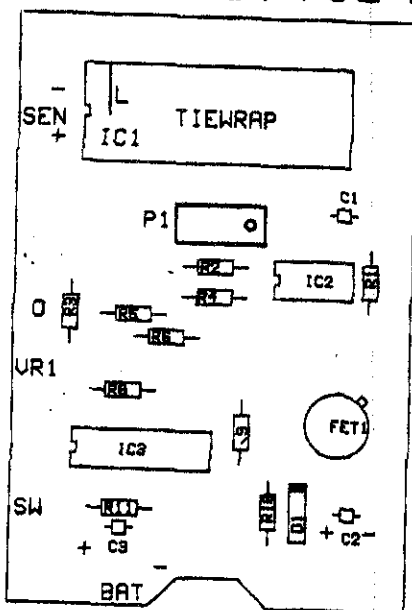

REGARDS AUBREY.

P. & M. SERVICES

(Rochdale) Ltd.

ESTABLISHED 1964

PRINTED CIRCUITS : PHOTO-MECHANICAL ENGINEERING

STONESWOOD MILL,
BACUP ROAD,
TODMORDEN,
LANCS., OL14 7HG

Tel: (0706) 815212



Fax: (0706) 818636

FAX: *VIA-MED.*


FOR THE ATTENTION OF:

*STEVE HARDAKER.*DATE: *29-5-96*NO OF PAGES: *1*

COPIES OF THE MODIFIED NETWORKS FOR DIVOX
P.C.B. FOR YOUR APPROVAL. HOPE ALL'S O.K.
BEST REGARDS.

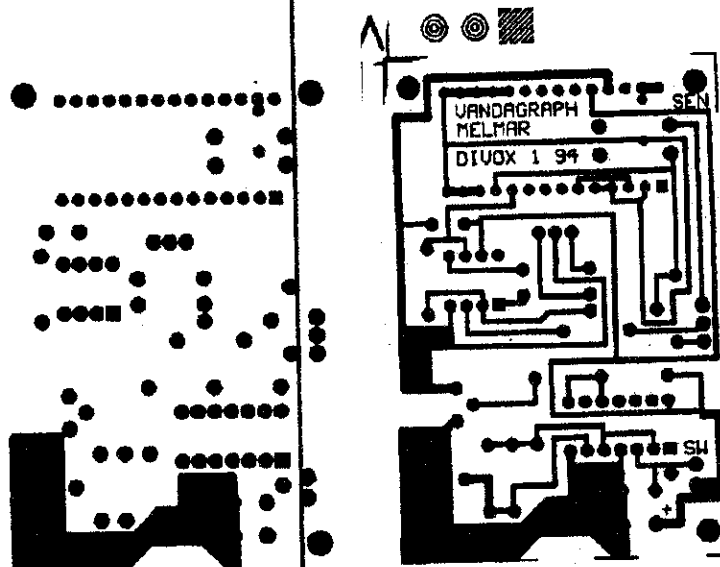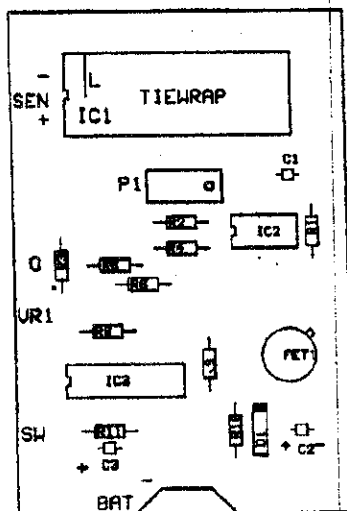AUBREY.

MARK

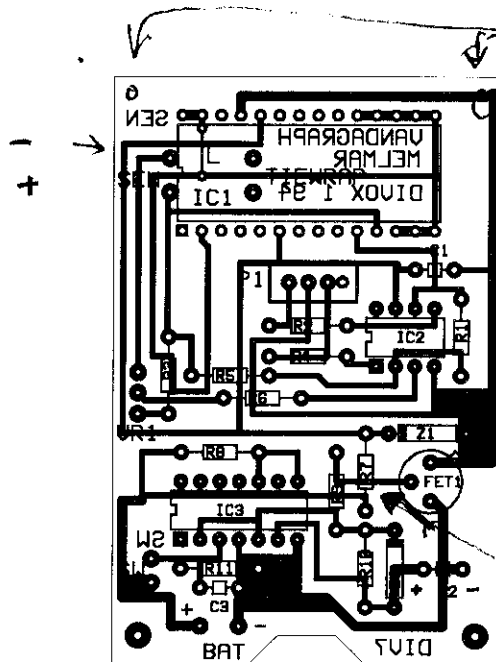

Holes to locate on case

Z1 No longer used
Component holes not required

R7 No longer used
Must Be replaced by
Taper

AUBREY OLDHAM.

1000/1

FAY 0706 818636